

KOSMOS System Software 1.0.73 for Brazil

Date: September 30th, 2019

Distribution: Internal (sales and clinical educators) and External (Brazilian customers)

System Software v1.0.73 Release

The KOSMOS 1.0.73 system software release makes KOSMOS available for customers in Brazil. It is important to note that Wi-Fi and Bluetooth will not be enabled with this software, which impacts the following system functionality:

- PACs, Modality Worklists connectivity will not be available.
- System registration will be disabled - a status of “Not registered” will appear in the About screen.
- Software updates will only be available via USB drive.
- Time and Date settings will need to be changed to manual to ensure accurate information. The setting defaults to automatic. From the Home screen, tap settings > Languages, Date and Time > tap left of the Automatic date and time button to turn it off.

Software Updates Using USB Flash Drive


If the device cannot be connected to the Internet, the software may be updated by a USB flash drive.

For a USB flash drive certified for your system serial number, contact EchoNous Customer Service by telephone at (844) 854-0800 or via email at service@echonous.com.

Note that KOSMOS System Software must be installed from an EchoNous flash drive and is only authenticated for a specific serial number.


Plug the USB Flash Drive directly into the USB-C port on the side of the KOSMOS Bridge.

The system will recognize the USB drive and begin to verify its contents. This can take 2 to 3 minutes to complete.

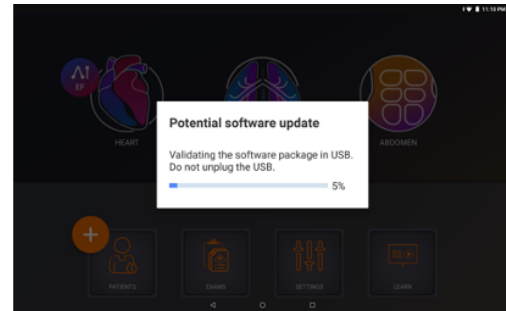


Once the contents of the plugged in USB drive have been verified, you will see a popup message.


Press the INSTALL button.

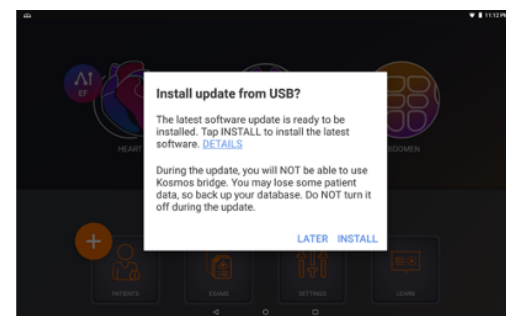

Kosmos Bridge will begin copying the update package from the USB drive. You will be able to see the progress on the screen.

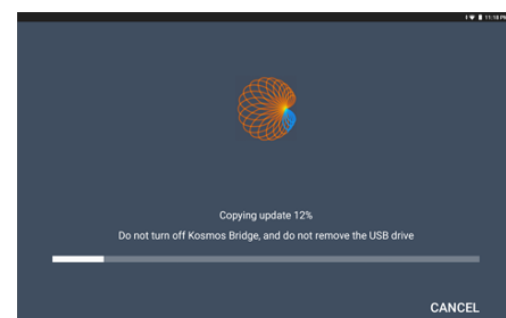

Once the update package has been copied, the installation process will immediately start.

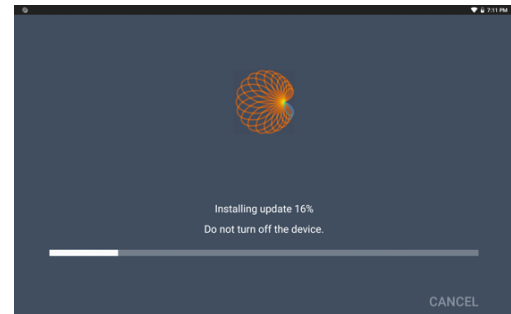


Once the installation is complete, the Kosmos Bridge will reboot and a message will appear saying that the update is complete.


Enabling the Handle Controls

The Handle Controls feature can be enabled from the Home Screen.

Tap **TURN ON HANDLE CONTROLS** then tap **ON**

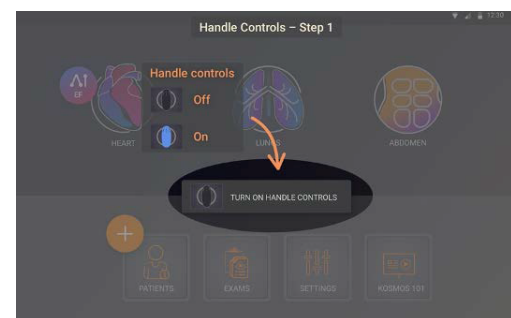

When the Handle Controls are enabled for the first time, KOSMOS Bridge will prompt you to update the Handle Controller firmware

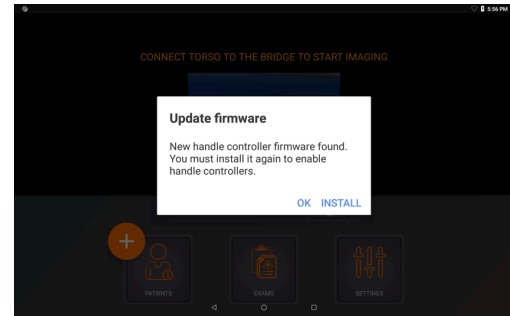

Press the INSTALL button on the popup to update the Handle Controller firmware and KOSMOS Bridge will proceed to install the firmware.

