

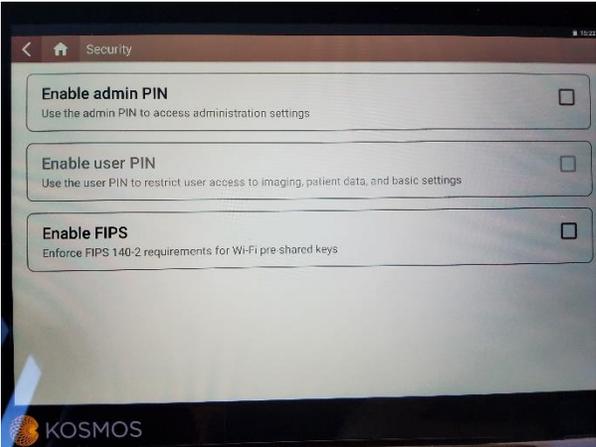
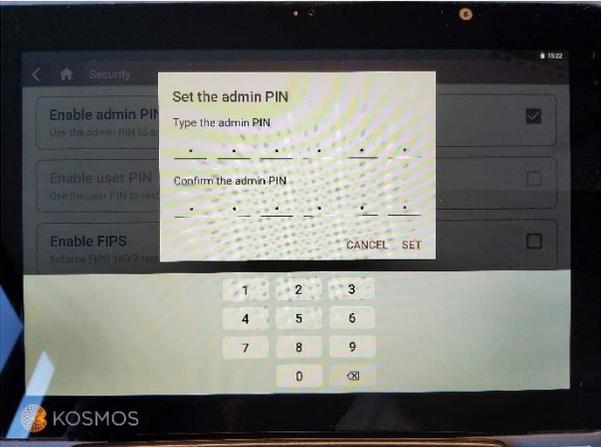
## Summary

This technical bulletin provides instructions for enabling AI FAST workflows on your Kosmos Torso and Torso-One probes via USB flash drive. ***This procedure will need to be performed on every Kosmos bridge tablet where you plan to use the AI FAST enabled probe(s).***

## Contents

One (1) USB Drive containing 7.1.0.x software and Kosmos AI FAST probe license(s).

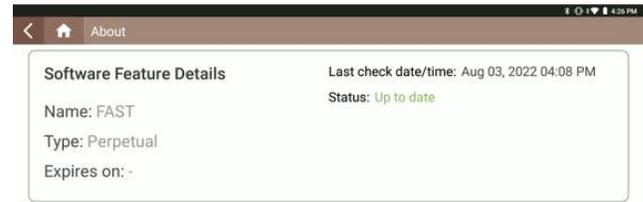
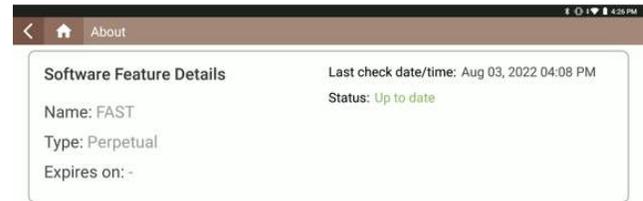
## Software Update and License Installation

<p><b>STEP 1</b></p>	<p><b><i>NOTE: Follow Step 1 only if the device contains software version 7.0.0.125. For software versions 4.x and 6.x, begin at Step 2. The software version can be viewed at the following location: Settings &gt; About</i></b></p> <p>Enable the Admin PIN by selecting the check box at the following location: <b>Settings &gt; Admin &gt; Security &gt; Enable admin PIN</b></p> <p>Enter 000000 as the admin PIN and touch SET.</p>	 
----------------------	---	--

# Technical Bulletin

<p><b>STEP 2</b></p>	<p>Plug the USB flash drive into the USB-C port on the right side of the Kosmos Bridge.</p> <p>The system will recognize the USB drive and begin to verify its contents. This step can take 2 to 3 minutes to complete.</p>	
<p><b>STEP 3</b></p>	<p>Once the contents of the USB drive have been verified, you will be prompted to install the software.</p> <p>Press INSTALL.</p>	
<p><b>STEP 4</b></p>	<p>Kosmos Bridge will begin copying the update package from the USB drive. The progress of the install is indicated by the progress bar on the screen.</p>	

# Technical Bulletin

<p><b>STEP 5</b></p>	<p>Once the update package has been copied, the installation process will immediately start.</p> <p>After installation is complete, the Kosmos Bridge will reboot, and a message will appear stating the update is complete.</p> <p>It is important to leave the USB drive plugged into the bridge until the software has installed and the device has fully rebooted.</p>	
<p><b>STEP 6</b></p>	<p>Verify the FAST software feature is enabled in: <b>Settings &gt; About &gt; Software Feature Details</b></p> <p>The name of the feature will be shown (FAST), as well as the license type and expiration.</p>	
<p><b>STEP 7</b></p>	<p>Disable the Admin PIN feature by deselecting the box at the following location. <b>Settings &gt; Admin &gt; Security &gt; Enable admin PIN</b></p>	

*If you have any questions regarding the software installation or feature implementation process, contact EchoNous Customer Service by telephone at (844) 854-0800 or via email at [service@echonous.com](mailto:service@echonous.com).*