

# Contact Us

For questions, please contact EchoNous®  
Customer Support:

Call 1-844-854-0800  
Fax 425-242-5553  
Email [info@echonous.com](mailto:info@echonous.com)  
Web [echonous.com](http://echonous.com)



**Caution:** Federal (United States) law restricts this device to sale by or on the order of a physician.

**Note:** Not all features are available in all markets. Reference the user guide of your region or country for all available features before use.



# Quick Start Guide

## What's in the Box

### Bridge



Kosmos Bridge



Power Supply



Bridge Stand

### Torso-One




Kosmos Torso-One

### Torso



Kosmos Torso

### Accessories

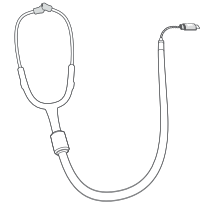


Binaural Headset




ECG Patient Cable

## Set up your KOSMOS


To charge, attach the power supply into the USB slot on Kosmos Bridge and the other end into an electrical outlet.

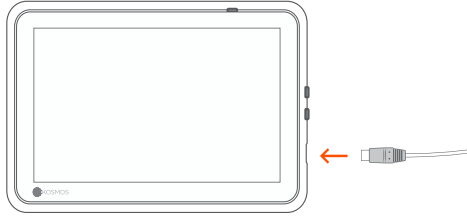

Plug the Kosmos Torso connector into the slot under the Kosmos Bridge Handle.

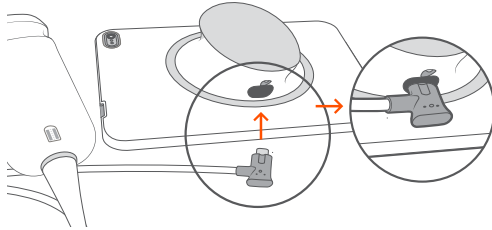


Unfold the stand, put it on a flat surface, and place Kosmos Bridge on it.

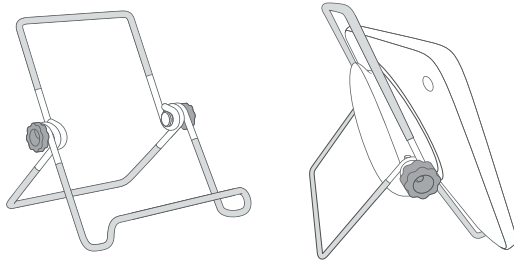


Adjust the position to get the best viewing angle

Press the power button to turn on Kosmos Bridge.

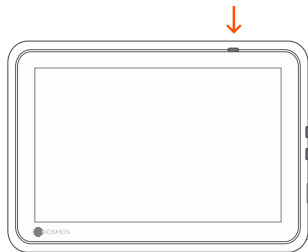


## ECG (Only applicable with Torso)

To view the ECG Signal, connect the magnetic connector end of the ECG patient cable to Kosmos Torso.

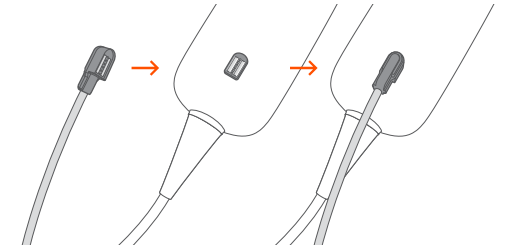

## Digital Auscultation (Only applicable with Torso)

To hear the digital auscultation sound, plug the USB end of the binaural headset into the USB slot on Kosmos Bridge.

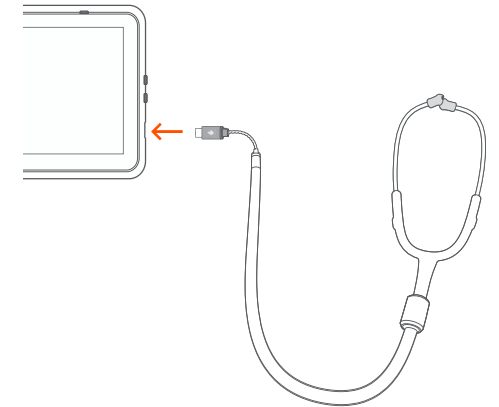

ⓘ Please refer to the Kosmos Interface Guide for next steps.