

# Chemical Compatibility

The cleaning and disinfection chemicals listed here have been evaluated for compatibility with the device materials. We will continue to update the table below as we evaluate additional cleaning and disinfection chemicals for compatibility with the device.

| Cleaning and/or Disinfectant Agent                             | Active Ingredient   | Kosmos Bridge | Kosmos Probes | ECG Cables | Headset |
|--|---|---------------|---------------|------------|---------|
| Warm Soapy water   | None  | A             | A             | A          | A       |
| PDI Healthcare Sani-Cloth® Plus Germicidal Disposable Wipes    | n-Alkyl dimethyl ethylbenzyl ammonium chlorides (0.125%), n-Alkyl dimethyl Benzyl Ammonium Chlorides (0.125%), Isopropanol (10-20%), 2-Butoxyethanol (<5%)  | A             | A             | A          | A       |
| 1PDI Healthcare Super Sani-Cloth® Germicidal Disposable Wipes  | n-Alkyl dimethyl ethylbenzyl ammonium chlorides (0.25%), n-Alkyl dimethyl Benzyl Ammonium Chlorides (0.25%), Isopropyl Alcohol (Isopropanol) (55.5%)  | A             | A             | A          | A       |
| 1PDI Healthcare Sani-Colth® Bleach Germicidal Disposable Wipes | Sodium Hypochlorite(0.63%)  | A             | A             | C          | B       |
| 2Metrex CaviWipes™ (KavoWipes)                                 | Diisobutylphenoxyethoxyethyl dimethyl benzyl ammonium chloride (0.28%), Isopropanol (17.2%)   | A             | A             | A          | A       |
| 1Metrex CaviCide™  | Diisobutylphenoxyethoxyethyl dimethyl benzyl ammonium chloride (0.28%), Isopropanol (17.2%)   | A             | A             | A          | A       |
| 1Metrex CaviWipes1™  | Didecyl dimethyl ammonium Chloride (0.76%), Ethanol(7.50%), Isopropanol (15%)   | A             | A             | A          | A       |
| 1Metrex CaviCide1™   | Didecyl dimethyl ammonium Chloride (0.76%), Ethanol(7.50%), Isopropanol (15%)   | A             | A             | A          | A       |
| ASP Cidex® OPA   | ortho-Phthalaldehyde(0.55%), Dipotassium Hydrogen phosphate, Potassium Dihydrogen Phosphate, Benzotriazole, citric acid, D&C green dye #5, N-(Hydroxyethyl)-ethylenediaminetriacetic acid(HEDTA) (99.45%) | N/A           | A             | N/A        | N/A     |
| ASP Enzol® Enzymatic Detergent                                 | Subtilisins (<5%)   | A             | A             | A          | A       |
| Hydrogen Peroxide 3% solution                                  | Hydrogen Peroxide (3%) , water (97%)  | A             | A             | A          | A       |
| Kyorin Pharmaceutical Milton                                   | Sodium Hypochlorite(1.1%)   | A             | A             | C          | B       |

**Key:** 1Product on List N: Disinfectants for Use Against SARS-CoV-2 2Per Metrex released note, approved for Covid 19

**A:** Compatible

**B:** Caution - Not recommended.  
May cause minor cosmetic changes with extended use.

**C:** Not compatible - Do not use. Extended exposure may lead to product damage.

**N/A:** Compatibility data not available

**Note:** Not all features are available in all markets. Reference the user guide of your region or country for all available features. **⚠ Caution:** Federal (United States) law restricts this device to sale by or on the order of a physician.

Electronic versions of the Kosmos User Guides may be viewed and downloaded at [echonous.com/resources](http://echonous.com/resources).

Printed versions of the Kosmos User Guides may be requested by emailing [service@echonous.com](mailto:service@echonous.com) or by calling 1-844-854-0800.

For questions, please contact EchoNous® Customer Support:

Call: 1-844-854-0800

Fax: 425-242-5553

Email: [support@echonous.com](mailto:support@echonous.com)

Web: [echonous.com](http://echonous.com)



Australia Sponsor: LC & Partners Pty Ltd, North Sydney, NSW, 2060  
Tel: +61 2 9959 2400