Kosmos Power Pack Quick Start Guide

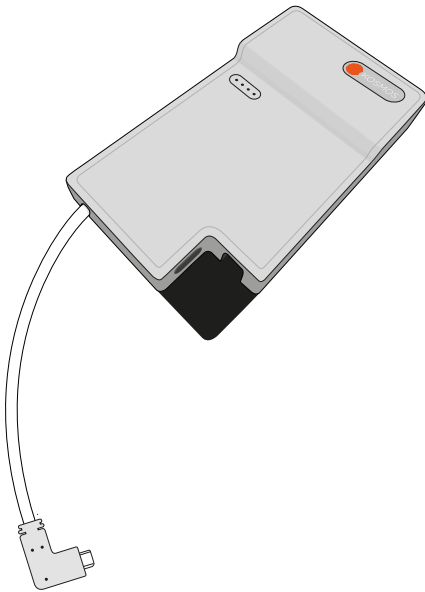
Multilingual Booklet

3	Kosmos Power Pack Quick Start Guide
7	Guide de démarrage rapide Kosmos Power Pack
11	Panduan Memulai Cepat Power Pack Kosmos
15	Guia de início rápido do Kosmos Power Pack
19	Kosmos Power Pack Guía de inicio rápido
23	Kosmos Power Pack Lynstartguide
27	Kosmos Power Pack Kurzanleitung
31	Guida introduttiva a Kosmos Power Pack
35	Snelstartgids Kosmos Power Pack
39	Kosmos Power Pack Hurtigstartveiledning
43	Kosmos Power Pack Snabbstartsguide
47	Kosmos Power Pack Hızlı Başlangıç Kılavuzu
51	Skrócona instrukcja obsługiakumulatora Kosmos Power Pack
55	Kosmos Power Pack Ghid de pornire rapidă
59	Kosmos Power Pack gyors üzembhelyezési útmutató
63	Kosmos Power Pack クイックスタートガイド
67	Kosmos Power Pack Οδηγός γρήγορης εκκίνησης


Kosmos Power Pack Quick Start Guide

What's in the box


Power Pack

Charging Cord

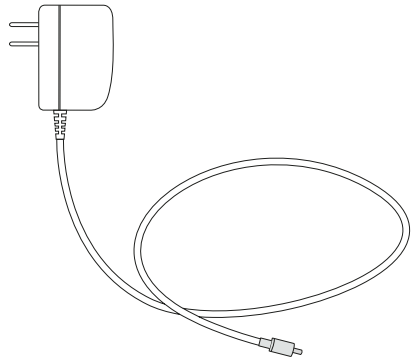Mounting Plate

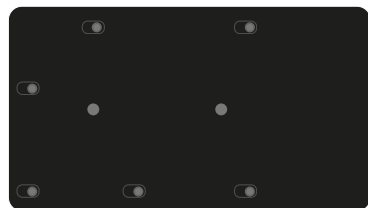
(Pre-attached to Power Pack)

Setting up your Power Pack

Step 1: Make sure Power Pack is charged for one hour prior to use. (See instructions on Charging your Power Pack).

Step 2: Position Power Pack onto device so that it sits comfortably in hand when in use, keeping both port to device and Power Pack are on the same side.

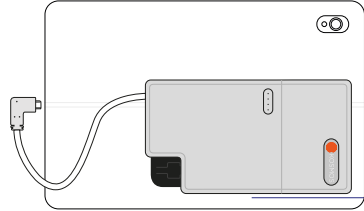
Depending on the size of the tablet, the Power Pack may need to shift to the left or right to get a comfortable fit.

Step 3: Once the Power Pack position is set to your preference, peel off the adhesive backing on each strip and place the Power Pack onto the device in desired position. Press firmly onto the Power Pack and hold for 45 seconds to make sure adhesive is applied tightly to the device.

Notes:

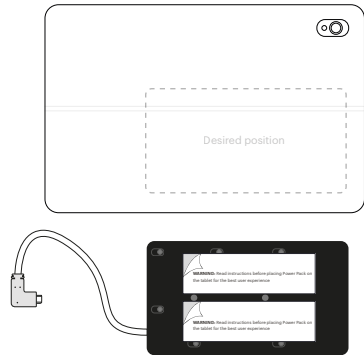- Adhesive strips are not for reuse.
- The adhesive will not adhere to rubber.


Step 4: Connect the Power Pack cable to the tablet as shown

A: Power Pack cable

B: Probe cable connection

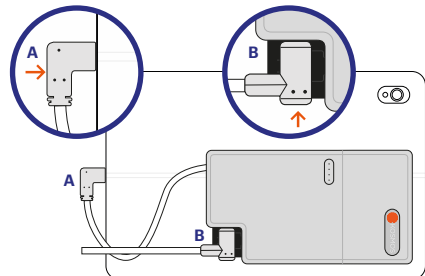Note: Power Pack must be connected to the tablet **first** in order for the probe to be recognized by the Power Pack.


Charging your Power Pack

Step 1: Disconnect the probe if connected.


Step 2: Connect the charger to the Power Pack. Once connected, the lights on the Power Pack will indicate the battery power level.

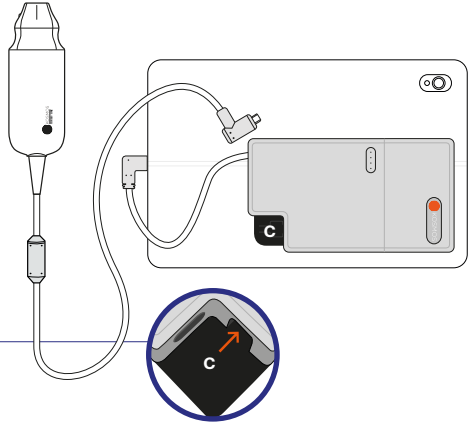C: AC Charger adapter point

Step 3: Disconnect charger once Power Pack is fully charged.

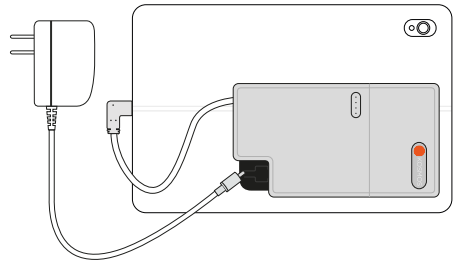
(See Power Pack LED Status).


How to dismantle the Power Pack (optional)

Step 1: To remove the Power Pack from the mounting plate, press down on the release tab (D) and slide the Power Pack down.

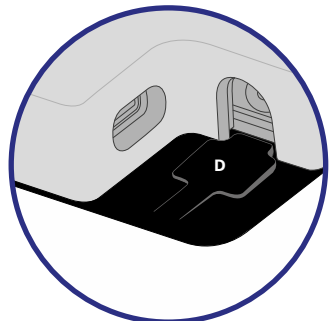Step 2: To mount the Power Pack back on, slide the Power Pack up into place.

Power Pack LED Status

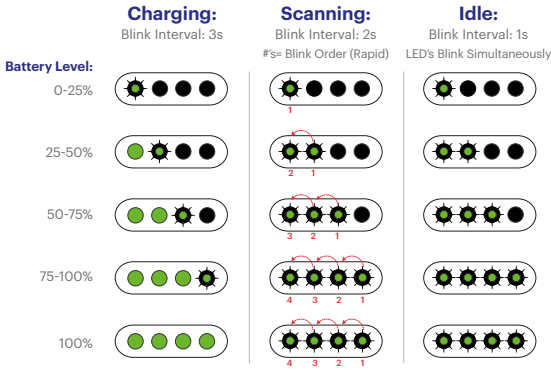Compatibility:

See echonus.com for list of compatible tablets and probes

Contact us

For questions, please contact EchoNous®
Customer Support:

8310 154th Ave NE Ste 200
Redmond, WA 98052
Tel 1-844-854-0800
Fax 425-242-5553
Email support@echonus.com
Web echonus.com

Caution: Federal (United States) law restricts this device to sale by or on the order of a physician.

Note: Not all features are available in all markets. Reference the user guide of your region or country for all available features before use.

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© 2015 to 2023 EchoNous, Inc., or its affiliates. All rights reserved. EchoNous, echonous.com, and the EchoNous logo are trademarks of EchoNous, Inc. or its affiliates. EchoNous, Inc. 8310 154th Ave. NE, Building B, Suite 200, Redmond, WA 98052.