



---

## Setting up your Power Pack

**Step 1:** Make sure Power Pack is charged for one hour prior to use.  
(See instructions on Charging your Power Pack).

**Step 2:** Position Power Pack onto device so that it sits comfortably in hand when in use, keeping both port to device and Power Pack on the same side.

Depending on the size of the tablet, the Power Pack may need to shift to the left or right to get comfortable fit.

**Step 3:** Once the Power Pack position is set to your preference, peel off the adhesive backing on each strip and place the Power Pack onto the device in desired position. Press firmly onto the Power Pack and hold for 45 seconds to make sure adhesive is applied tightly to the device.

**Notes:**

- Adhesive strips are not for reuse.
- The adhesive will not adhere to rubber.

---

## Charging your Power Pack

**Step 1:** Disconnect the probe if connected.

**Step 2:** Connect the charger to the Power Pack. Once connected, the lights on the Power Pack will indicate the battery power level.

**C:** AC Charger adapter point

**Step 3:** Disconnect charger once Power Pack is fully charged.

(See Power Pack LED Status).

---

## How to dismantle the Power Pack (optional)

**Step 1:** To remove the Power Pack from the