



Tablet & Transducer Setup

Kosmos Platform Quick Start Guide
for Kosmos Tablet G1

Torso-One

Quick Start Guide for Kosmos Tablet G1


Getting Started

Step 1 Charge Kosmos Tablet G1

Plug the power supply into the USB-C port on the side of the Kosmos Tablet G1

Step 2 Connect to Wi-Fi (Optional)

From the Kosmos Homescreen, tap **Settings > Admin > Wi-Fi**

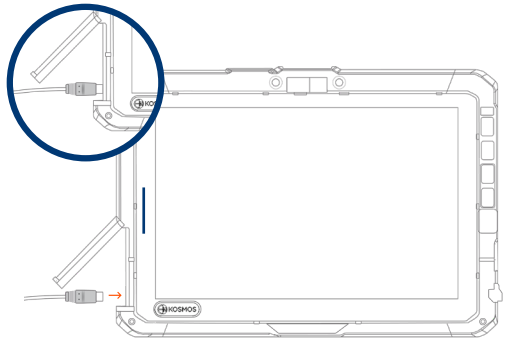


USB-C port for charging

Step 3 **Connect Torso-One**

Plug the Kosmos Torso-One connector into the port on the side of the Kosmos Tablet G1.

Note: To register your transducer and licensed features for the first time, the probe must be connected to the device and your device must be connected to the internet. This step may take a few minutes.

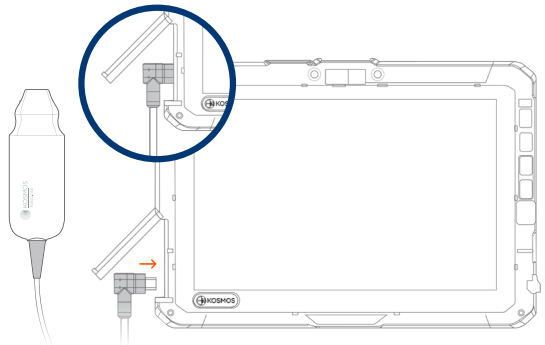


USB-C port for probe connector

Step 4 **Transducer Element Check**

An automatic Transducer Elements Test is initiated every time a Kosmos probe is connected to a device.

With the completion of a successful test, users can begin scanning.

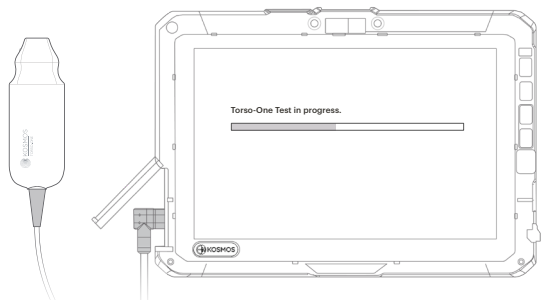

Lexsa

Quick Start Guide for Kosmos Tablet G1


Getting Started

Step 1 Charge Kosmos Tablet G1

Plug the power supply into the USB-C port on the side of the Kosmos Tablet G1

Step 2 Connect to Wi-Fi (Optional)

From the Kosmos Homescreen, tap **Settings > Admin > Wi-Fi**

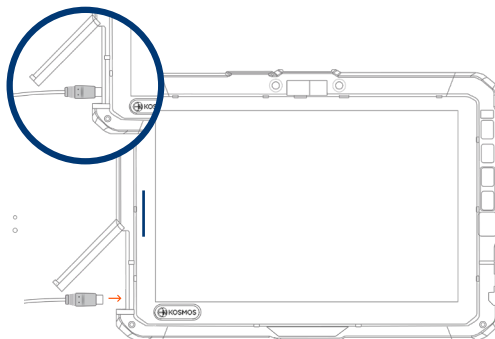


USB-C port for charging

Step 3 **Connect Lexsa**

Connect Lexsa to the USB-C port on the side of the Kosmos Tablet G1.

Note: To register your transducer and licensed features for the first time, the probe must be connected to the device and your device must be connected to the internet. This step may take a few minutes.

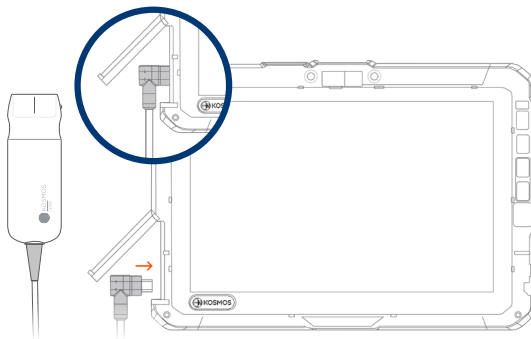


USB-C port for probe connector

Step 4 **Transducer Element Check**

An automatic Transducer Elements Test is initiated every time a Kosmos probe is connected to a device.


With the completion of a successful test, users can begin scanning.

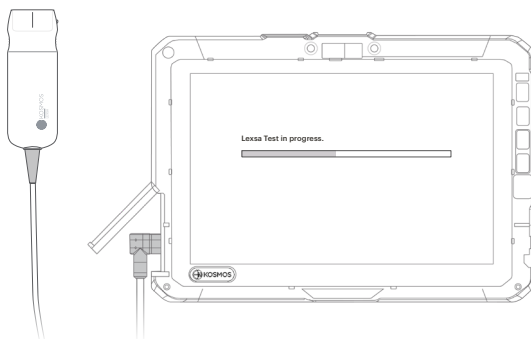

Kosmos Tablet G1 Accessories (Optional)

Kickstand (Optional)

Follow the Kosmos Tablet G1 kickstand instructions provided.

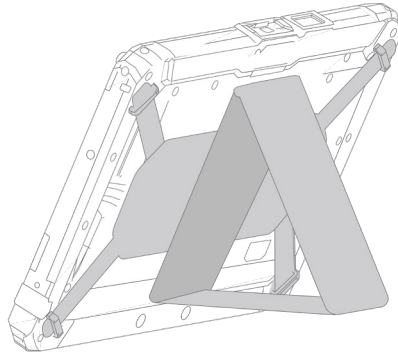

Purchasable Features for Kosmos Tablet G1

Licensed features are optional-purchasable features. To check the status of licensed features, from the Kosmos Home screen, tap **Settings > About > Software Feature Details**.

For more information regarding these features, please contact your EchoNous representative.

Customer Support



EchoNous, Inc.
8310 154th Ave. NE, Building B, Suite 200
Redmond, WA 98052

Call 1-844-854-0800
Email support@echonous.com

Refer to user guide before use.

⚠ **Caution:** Federal (United States) law restricts this device to sale by or on the order of a physician.

Note: Not all features are available in all markets, please check with your local representative for availability in your region.

Please visit www.echonous.com for a current list of compatible tablets.



© EchoNous, Inc., or its affiliates. All rights reserved.

EchoNous, echonous.com, and the EchoNous logo are trademarks of EchoNous, Inc. 8310 154th Ave. NE, Building B, Suite 200, Redmond, WA 98052.

P008972-001_RevB