
Product: Kosmos on iOS Software 5.0.1.4
Distribution: Kosmos on iOS Users

Date: 4/3/2026
D012539

We're pleased to announce the release of Kosmos on iOS 5.0. This major update expands the clinical capabilities of Kosmos by introducing Transcranial Doppler (TCD), Obstetrician (OB), and Gynecologist (GYN) imaging presets. The release also includes measurement packages, and structured reporting for OB/GYN and Vascular assessments.

New Features & Capabilities

- **Transcranial Doppler (Optional Add-On)** Designed specifically for the Torso-One probe, our dedicated TCD preset enables rapid assessment of stroke and vasospasm via the transtemporal window
- **OB & GYN Presets (Optional Add-On)** Now includes fully integrated measurement packages alongside optimized imaging presets.
 - **Capabilities:** Complete biometry for OB (Gestational Sac, CRL, BPD, HC, AC, FL, AFI) and GYN (Uterus, Ovaries, Endometrium, Bladder Volume).
 - **Reporting:** Features comprehensive DICOM Structured Reporting (SR)
 - Contact your local representative or Customer Support to add these capabilities.
- **Vascular Annotation Package (Now Included Standard)** Also in this update is a robust annotation with DICOM SR for Carotid, Lower Extremity (Arterial/Venous), and Abdominal Aorta exams.

Feature Activation

- **TCD, OB & GYN Presets** To purchase these features, please contact your local EchoNous sales representative or Customer Support to discuss licensing.
- **Vascular Annotation Package:** This feature is included in the standard update. To access it, simply follow the installation steps below.

Installation

1. Connect your iPad to the internet.
2. Navigate to the Apple App Store and download or update to the Kosmos Ultrasound 5.0.1.4 app.
3. While connected to the internet, open your Kosmos Ultrasound 5.0.1.4 app and when prompted enable the necessary drivers. [See Enable Drivers video.](#)

This release of Kosmos on iOS 5.0 underscores our dedication to continuous product innovation and providing an exceptional customer experience. We encourage all users to update to this latest version to ensure optimal performance.

If you have any questions regarding installation or feature activation, contact EchoNous Customer Service by telephone at (844) 854-0800 or via email at service@echonous.com.