

Kosmos Software v2.2 for Kosmos on iOS

Date: September 12th, 2024

Upgrading to Kosmos Software v2.2 on iOS

To upgrade your previous iOS Kosmos app to v2.2 please follow these steps:

1. With your currently installed Kosmos app, send any exams you wish to keep to PACS or export them to a local file. Each exam must be exported individually. [See Exam Export video.](#)
 - a) Please note existing exams will not be imported into iOS 2.2 and will be deleted once your current app is deleted.
2. Delete the Kosmos Ultrasound app currently installed on your iPad.
3. Go to the Apple App store and download the Kosmos Ultrasound 2.2 app.
 - a) Please note if you don't delete your previous Kosmos iOS app per step 2 above and you have both your previous version of Kosmos iOS and Kosmos iOS 2.2 installed on your iPad, you will need to delete both Kosmos apps and re-install Kosmos iOS 2.2
4. Open the Kosmos Ultrasound 2.2 app, when prompted, enable the necessary drivers. [See Enable Drivers video.](#)
5. If you have purchased Auto Doppler or Auto Preset licenses, you can enable or disable the features in Settings > Imaging Preferences.

Kosmos Software v2.2 Release Features for Kosmos on iOS

The v2.2 software provides the following features and improvements for Kosmos on iOS

- Auto Doppler: Automatic Doppler gate placement for user specified cardiac valve interrogation
- Auto Preset: Automatic preset adjustment based on anatomy being scanned
- Auto EF workflow improvement
- Auto VTI trace improvement

Do not disconnect the probe during the Transducer Element Check. Disconnecting the probe before the test has been completed may cause a temporary software anomaly while in Exam Review.

Compatible Tablets

For a current list of compatible tablets, please visit the EchoNous website at:
<https://echonous.com/product/device-compatibility/>

*Kosmos Trio is not available in the United States or Australia.

Licensed Features for Kosmos on iOS

Licensed features are optional-purchasable features. To check the status of the licensed features on your device:

Kosmos Home screen → Settings → About → Software Feature Details

For questions regarding purchasing or checking the status of your licensed features, please contact your EchoNous representative.

Table: Features by Kosmos Probe and Licensed Features for Kosmos on iOS

Features	Torso-One	Lexsa	Licensed Features
B-Mode	X	X	
M-Mode	X	X	
Color Doppler	X	X	
CPD		X	
Cardiac Calcs	X		
Vascular Calcs		X	
PW	X	X	X
TDI	X		X
CW	X		X
Trio	X		X
Auto EF	X		X
AI FAST	X		X
Auto Doppler	X		X
Auto Preset	X		X

Visit the Apple App Store to download and install the latest update for the EchoNous Kosmos Ultrasound App. If you have any questions, contact EchoNous Customer Service by telephone at (844) 854-0800 or via email at service@echonous.com.

*Kosmos Trio is not available in the United States or Australia.