



KOSMOS

User Guide



P008457-003 Rev A

January 2025

* Android is a trademark of Google LLC.

* Apple licenses the "iOS" trademark from Cisco.

© 2015 to 2025 EchoNous, Inc., or its affiliates. All rights reserved.

CHAPTER 1

Getting Started 1

- What's new in this release? 1
- Packaged Contents 1
- Intended users 2
- Intended use/indications for use 2
 - Contraindications* 4
- General warnings and cautions 5
- User guide 6
 - Symbols in this user guide* 6
 - User guide conventions* 6
- EchoNous customer support 8

CHAPTER 2

KOSMOS Overview 9

- What is Kosmos? 9
- Kosmos clinical applications 11
- Training 11
- Kosmos classifications 12
- Patient environment 12

CHAPTER 3

Using Kosmos 13

- Kosmos hardware 13
 - Kosmos Bridge* 13
 - Kosmos Torso, Kosmos Torso-One and Kosmos Lexsa* 15
 - Kosmos Power Supply* 16
 - Kosmos ECG Patient Cable** 16
 - Kosmos Digital-to-Analog Converter** 17
- Connecting Kosmos Probes 18
- Connecting the Kosmos power supply 19
- Turning Kosmos Bridge on and off 20
 - Turning on Kosmos Bridge* 20
 - Turning off Kosmos Bridge* 20
- Using the Kosmos Bridge handle controls 20

<i>Switching Probes</i>	21
<i>Turning on the handle controls</i>	22
<i>Ergonomic considerations while using the handle controls</i>	24
General interaction	26
<i>Home Screen: Kosmos Torso and Kosmos Torso-One</i>	26
<i>Home Screen: Kosmos Lexsa</i>	26
<i>Learn</i>	27
<i>On-screen keyboard</i>	27
Understanding the different waveforms	28
<i>ECG & DA *</i>	28
<i>ECG</i>	28
Settings	29
<i>Imaging Preferences</i>	29
<i>Configuring ECG and DA Signals</i>	30
<i>Setting the language, date, and time</i>	30
<i>Adjusting the Sound</i>	31
<i>Setting Brightness</i>	32
<i>Screen mirroring (Miracast)</i>	32
<i>Security</i>	33
<i>Setting the auto power off and auto sleep time interval</i>	35
<i>To change the sleep mode time interval:</i>	35
<i>USB Export</i>	41
<i>To configure USB export preferences:</i>	41
<i>Installing software updates</i>	41
<i>Managing network and internet settings</i>	42
<i>Factory Reset</i>	42
Wireless networking	44
<i>Functions</i>	44
<i>Connection specifications</i>	44

CHAPTER 4

Incorporating ECG and DA Signals 47

Overview	47
<i>ECG</i>	47
<i>DA</i>	48

Benefits of using ECG and DA signals with ultrasound	48
Using the Kosmos ECG patient cable	49
Attaching binaural headset	51
Viewing the ECG and DA signals	52
<i>Signal scrolling</i>	52
<i>ECG signal indicator</i>	52
Preserving the ECG and DA signals when freezing an image or taking a clip	53
Archiving and exporting ECG and DA waveforms	53

CHAPTER 5

Performing an Exam 55

Overview	55
<i>Primary Exam Workflows</i>	56
Exam workflows	57
<i>Standard workflow</i>	57
<i>Quick workflow</i>	58
<i>Kosmos AI-assisted EF workflow</i>	59
Managing exams	60
<i>Starting an exam</i>	60
<i>Searching for an exam</i>	60
<i>Deleting exams</i>	60
<i>Completing exams</i>	61
Managing patient data	61
<i>Adding a new patient</i>	61
<i>Accessing patient information using MWL</i>	61
<i>Searching for a patient</i>	62
<i>Changing to another patient</i>	62
<i>Editing a patient record</i>	62
<i>Merging two patient records</i>	63
<i>Deleting patient records</i>	63
Organ Presets	64
Imaging Modes and Features	64
<i>B-mode (2D)</i>	65
<i>M-mode</i>	65
<i>Color Doppler</i>	66

<i>Pulsed-Wave Doppler</i>	68
<i>Tissue Doppler Imaging</i>	70
<i>Continuous-Wave Doppler</i>	70
<i>Image mode controls</i>	72
Using Kosmos AI-assisted EF Workflow	73
<i>Calculating Ejection Fraction with Kosmos AI-assisted EF Workflow</i>	73
<i>Reviewing/adjusting the ED/ES frames and LV contours</i>	75
<i>Recommendations for acquiring optimal A4C and A2C clips for accurate EF calculations</i>	78
<i>Error conditions and system notifications for Kosmos AI-assisted EF workflow</i>	79
Acquiring images and clips	79
Completing an exam	80
<i>Kosmos Cardiac Measurements</i>	80
Kosmos AI FAST	83
<i>Using Kosmos AI for FAST Exam</i>	83
Us2.ai & Kosmos	84
<i>Getting Started</i>	84

CHAPTER 6

Reviewing an Exam 87

Starting an exam review	87
Annotating images and clips	88
<i>Navigating to the Edit Image screen</i>	88
<i>Annotation tools</i>	89
<i>Measuring with the caliper tool</i>	89
<i>Deleting annotations</i>	90
Managing images and clips	90
<i>Filtering images and clips</i>	90
<i>Selecting images and clips</i>	91
<i>Deleting images and clips</i>	91
Reviewing and editing a report	92
<i>Opening a report</i>	92
<i>Editing a report</i>	92
Exporting images and clips to a USB drive	94
Completing an exam review	95

Archiving an exam to a PACS server 96

Deleting an exam 97

CHAPTER 7

Kosmos Probes 99

Kosmos Probe sheaths 99

Ultrasound transmission gels 100

Kosmos Probe storage 100

Daily storage 100

Storage for transport 100

Transducer Element Check 101

CHAPTER 8

Kosmos Maintenance 103

Cleaning and disinfecting 103

General cautions 103

Kosmos Bridge 104

Kosmos Probes 105

Guidelines for AR (Automated Reprocessors) 110

Kosmos ECG patient cable 110

Binaural headset 113

Recycling and disposal 113

Troubleshooting 114

Preventive inspection, maintenance, and calibration 114

Kosmos Bridge handle controls 114

CHAPTER 9

Safety 117

Electrical safety 117

References 117

Labeling symbols 118

Contact information 126

Biological safety 129

ALARA education program 129

Kosmos Torso and Kosmos Torso-One Acoustic Output Tables 132

<i>Kosmos Lexsa Maximum Acoustic Output Summary</i>	140
<i>Measurement accuracy</i>	142
Measurement accuracy	147
<i>Control effects</i>	148
<i>Related references</i>	148
<i>Transducer surface temperature rise</i>	148
<i>ECG supplemental information</i>	149
Ergonomics	150
Electromagnetic compatibility	151
<i>Electromagnetic emissions</i>	152
<i>Electromagnetic immunity</i>	153
<i>Separation distances</i>	156
<i>Certificate and compliance</i>	156
<i>Intentional radiator</i>	157
<i>Class B device</i>	157
<i>Industry Canadian statement</i>	158
Standards	158
<i>HIPAA</i>	158
<i>DICOM</i>	159

CHAPTER 10

Specifications **161**

System Specifications 161

Environmental operating and storage conditions 162

Operating, charging, transport, and storage condition ranges **162**

Mode of operation **162**

Power supply (charger) **162**

Internal batteries **163**

CHAPTER 11

IT Network **165**

Wireless networking 165

Functions **165**

Connection specifications **165**

Network for connecting the device 166

Specifications for the connection	166
<i>Hardware specification</i>	166
<i>Software specifications</i>	166
<i>Security</i>	166
IT network failure recovery measures	167

CHAPTER 12

Glossary 169

APPENDIX A

Auto EF Clinical Performance and Non-Clinical Testing 175

Auto EF Clinical Performance Testing	175
<i>Results</i>	176
Software Verification and Validation Testing	176
Algorithm Testing	177

APPENDIX B

AI FAST Clinical Performance and Non-Clinical Testing 179

AI FAST Clinical Performance Testing	179
<i>Results</i>	180
Software Verification and Validation Testing	180
Algorithm Testing	181



Intentionally left blank.

Getting Started

What's new in this release?

New features and changes for the 8.0 version of Kosmos include:

- Pulsed Wave Doppler on Lexsa
- Color Power Doppler on Lexsa
- Auto VTI Trace
- M-mode Cardiac Calculations for Torso-One
- Improved EF Workflow
- In-progress archive

	Please visit the website at echonous.com/product/resources for electronic versions of the user guides.
	Not all features are available in all markets. Please check with your EchoNous representative for availability in your region.

Packaged Contents

The Kosmos box contains the following items:

- Kosmos system, comprised of the Kosmos Bridge and Kosmos Torso or Kosmos Torso-One or Kosmos Lexsa
- Kosmos power supply
- Kosmos ECG patient cable (only with Kosmos Torso)
- Kosmos Digital-to-Analog Converter (only with Kosmos Torso)
- Kosmos Platform Quick Start Guide
- Chemical Compatibility
- Kosmos Welcome Letter
- USB flash drive containing:



- Kosmos User Guide
- Kosmos AI Station 2 User Guide

Intended users

Kosmos is intended to be used by qualified and trained healthcare professionals that are legally authorized by law in the country, state, or other local municipality in which they practice to use the device. The list of the potential users includes but is not limited to (based on title/geographical location): Medical specialists, primary care physicians, point-of-care (POC) users, sonographers, medical healthcare technicians, nurses, nurse practitioners, physician assistants, and medical students.

Users may or may not be working under supervision or authority of a physician.

Intended use/indications for use

	To help ensure the diagnostic quality of the images obtained, all patient images must be obtained by qualified and trained healthcare professionals.
	Auto EF and AI FAST have not been evaluated for use in children or pregnant women.

Kosmos is intended to be used by qualified and trained healthcare professionals in the clinical assessment for the following clinical applications by acquiring, processing, displaying, measuring, and storing ultrasound images, or synchronized ultrasound images, electrocardiogram (ECG) rhythms, and digital auscultation (DA) sounds and waveforms.

With respect to its ultrasound imaging capabilities, Kosmos is a general purpose diagnostic ultrasound system used in the following clinical applications and modes of operation:

Clinical Applications and Modes of Operation for Kosmos

Clinical Applications:

- **Torso/Torso-One:** Cardiac, Thoracic/Lung, Abdominal

- **Lexsa:** Lung, Vascular/Peripheral Vascular, Musculoskeletal, Nerve and interventional guidance (includes needle/catheter placement, fluid drainage, and nerve block)

Modes of Operation: B-mode, M-mode, Color Doppler, Color Power Doppler, Pulsed-Wave (PW) Doppler, Tissue Doppler Imaging (TDI), Continuous-Wave (CW) Doppler, Combined Modes of B+M, and B+CD, and Harmonic Imaging

Kosmos is intended to be used in clinical care and medical education settings on adult and pediatric patient populations.

Kosmos includes the AI-assisted automated ejection fraction software, known as Auto EF, which is used to process previously acquired trans thoracic cardiac ultrasound images, to store images, and to manipulate and make measurements on images using the Kosmos. Auto EF provides automated estimation of left ventricular ejection fraction. This measurement can be used to assist the clinician in a cardiac evaluation. Auto EF is indicated for use on adult patients only in healthcare facilities.

The Kosmos includes the Auto Anatomical Structure Labeling and View Identification, also referred to as AI FAST, software, which is intended for use only by qualified and trained medical professionals for automatic real-time detection and labeling of anatomical structures during image acquisition during cardiac, thoracic/lung, or abdominal ultrasound imaging. This feature is only indicated for use on adult patients in healthcare facilities.

The device is non-invasive, reusable, and intended to be used on one patient at a time.



TABLE 1-1. Modes of Operation & Purchasable Features

Mode	Torso	Torso-One	Lexsa	Purchasable Features
B-mode	X	X	X	
M-mode	X	X	X	
Color Doppler	X	X	X	
Harmonic Imaging	X	X		
AI-assisted EF Workflow	X	X		
CW Doppler	X	X		
PW Doppler	X	X	X	X
Tissue Doppler (TDI)	X	X		
Color Power Doppler			X	
AI FAST	X	X		X
Us2.ai	X	X		X














Contraindications

Kosmos is designed for transcutaneous scanning and trans thoracic echocardiography only.

Kosmos is not intended for ophthalmic use or any use causing the acoustic beam to pass through the eye.


	Show care when scanning near a wound to avoid damaging or further injuring the affected area.
	Federal (USA) law restricts this device to sale by or on the order of a physician.

General warnings and cautions


	System users are responsible for image quality and diagnosis.
	Kosmos is not MRI compatible and should not be used in an MRI suite.
	Kosmos is not for use in oxygen-rich environments.
	To avoid the risk of electrical shock, do not allow any part of Kosmos (except for the Kosmos Torso, Kosmos Torso-One or Kosmos Lexsa lens and the Kosmos ECG patient cable) to touch the patient.
	To avoid the risk of electrical shock or injury, do not open the Kosmos Bridge or Kosmos Torso or Kosmos Torso-One or Kosmos Lexsa enclosures for any reason. All internal adjustments and replacements (such as the battery) need to be made by a qualified Kosmos technician.
	To avoid the risk of electrical shock and fire hazard, inspect the power supply, AC power cords, cables, and plugs on a regular basis to ensure that they are not damaged.
	The Kosmos system, including the Kosmos ECG patient cable, is not defibrillation proof. To prevent injury to the operator/bystander, Kosmos Torso, Kosmos Torso-One, Kosmos Lexsa and the Kosmos ECG patient cable/leadwires must be removed from patient contact before the application of a high-voltage defibrillation pulse.
	Before using the system for interventional procedures, you must have training in the applicable interventional procedures in addition to training in the use of ultrasound imaging for needle and/or catheter guidance. Well known limitations of ultrasound physics may lead to an inability to visualize the needle/catheter or differentiate it from acoustic artifacts. Serious injury or complications may result from attempting an interventional procedure without proper training.
	As a precaution, be careful when scanning near a wound or over a dressing.
	Do not use Kosmos for intracavity imaging.
	Kosmos uses Bluetooth wireless communication technology.
	Keep power cords away from trafficked areas.
	Use only with approved EchoNous Power supply (Part number P005974) in any clinical care and medical education settings.

User guide




This user guide is intended to assist you with the safe and effective operation of Kosmos. Before attempting to operate Kosmos, read this user guide and strictly observe all the included warnings and cautions. Also, pay special attention to the information in the chapter called **Safety**.

	Not all software versions include all the features described in this guide. Reference the software version on your device.
---	--

This user guide and any digital media (and the information they contain) is the proprietary and confidential information of EchoNous and may not be reproduced, copied in whole or in part, adapted, modified, disclosed to others, or disseminated without the prior written permission of the EchoNous legal department. This document or digital media is intended to be used by customers and is licensed to them as part of their EchoNous purchase. Use of this document or digital media by unauthorized persons is strictly prohibited. This user guide is also available through the EchoNous website, or a paper copy may be supplied on request.

	Federal (United States) law restricts this device to sale by or on the order of a physician.
---	--

Symbols in this user guide

	Warning	A warning describes precautions to prevent injury or loss of life.
	Caution	A caution describes precautions to prevent damage to the device.
	Note	A note provides supplemental information.

User guide conventions

The following style conventions are used in this guide:

- Numbered and lettered steps must be performed in a specific order.
- Bulleted items are lists in no specific order.
- Kosmos touch screen icons and buttons are indicated in bold, such as **SCAN**.

- The word:
 - **Tap** refers to touching the screen quickly with your finger
 - **Double tap** refers to touching the screen two times in quick succession with your finger
 - **Drag** refers to touching the screen with your finger and then moving your finger across the screen
 - **Swipe** refers to moving your finger across the screen quickly
 - **Pinch** refers to moving two fingers in a pinch motion or pinch release motion across the screen
 - **Check** refers to tapping a check box to enable the associated function
 - **Clear** refers to tapping a check box to disable the associated function
 - **Select** refers to tapping a menu item from a menu list
- Links to other sections within the guide appear bold and colored, such as the cross reference, see **Imaging Modes and Features**.

EchoNous customer support

Contact customer support:

Phone: 844-854-0800

Fax: 425-242-5553

Email: info@echonous.com

Web: www.echonous.com

- End of section -

What is Kosmos?

Kosmos consists of Kosmos Bridge, which runs the EchoNous system software, and is connected by cable to a Kosmos probe.

The following probes are available for the Kosmos System:

- Kosmos Torso:
 - A phased array transducer that combines ultrasound, ECG and digital auscultation in one probe.
 - When Kosmos Torso is connected, Kosmos also provides three lead, single-channel ECG and digital auscultation (DA) signals.
 - Provides portable ultrasound imaging and supports non-invasive Cardiac, Thoracic/Lung and Abdominal imaging.
- Kosmos Torso-One:
 - A phased array ultrasound-only probe with a smaller, more streamlined form factor to help fit in between intercostal spaces.
 - Provides portable ultrasound imaging and supports non-invasive Cardiac, Thoracic/Lung and Abdominal imaging.
- Kosmos Lexsa
 - A linear array ultrasound probe.
 - Provides portable ultrasound imaging and supports non-invasive Lung, Vascular/Peripheral Vascular, Musculoskeletal and interventional guidance (includes needle/catheter placement, fluid drainage, and nerve block).

Kosmos uses pulse-echo ultrasound to generate real-time ultrasound images. This process involves transmitting high-frequency acoustic pulses into the body from the probe and detecting the returned signals and processing the return echoes through analog and digital processing to form real-time images of anatomy (B-mode and M-mode) and blood flow (Color Doppler, Color Power

Doppler, Tissue Doppler Imaging, Pulsed-Wave Doppler, and Continuous-Wave Doppler). Reference Modes of Operation & Features for Kosmos for more information about which modes are applicable for each Kosmos Probe.

Kosmos Bridge is a custom-designed tablet approved, preconfigured, and supplied by EchoNous. Kosmos Bridge is provided with a power supply. When the display is connected to Kosmos Torso, Kosmos Torso-One or Kosmos Lexsa, the combination is configured as a medical electrical system.

Kosmos provides optional wireless connectivity, allowing remote storage. Additionally, The Kosmos Bridge is battery powered.

Kosmos also includes the AI-Assisted EF Workflow, and AI FAST.

Kosmos AI-assisted EF workflow can help guide you through the calculation of left ventricular (LV) ejection fraction (EF). Kosmos uses a guided workflow to record the necessary clips. The recorded clips are then used by AI to provide an initial calculation of the EF, stroke volume (SV) and cardiac output (CO) based on the gender and age of the patient with results that you can review and adjust if you need to.

More specifically, Kosmos AI provides an initial calculation of the EF, which is based on identifying the end-diastolic (ED) and end-systolic (ES) frames along with the corresponding LV contours. Those ED/ES frames and LV contours can then be adjusted (as necessary) or accepted as is.

While reviewing these frames, you can adjust them based on your analysis, while Kosmos (using your adjustments) computes the EF and stroke volume (SV).

Kosmos AI FAST can help guide you through a FAST exam by identifying views and labeling key anatomical structures in real time.

For more information about calculating the EF workflow with Kosmos, see **Using Kosmos AI-assisted EF Workflow**.



- CO is available only with ECG when Kosmos Torso is connected and is calculated by multiplying the SV by the heart rate (HR).
- SV is calculated as ED LV volume minus ES LV volume.

Kosmos clinical applications

Kosmos is for non-invasive imaging of the human body and is intended for the following applications by probe:

Torso/Torso-One:

- Cardiac
- Thoracic/Lung
- Abdominal

Lexsa:

- Lung
- Vascular/Peripheral Vascular
- MSK
- Nerve

Training

Kosmos is intended to be used by clinicians with appropriate professional qualifications and clinical training.

All users should read the generic ALARA education program (see *ISBN 1-932962-30-1*), *Medical Ultrasound Safety* or the Health Canada *Guidelines for the Safe Use of Diagnostic Ultrasound* available on the Health Canada website. This program outlines the guiding principle for diagnostic ultrasound, where the qualified user keeps ultrasound exposure to “as low as reasonably achievable” while performing a diagnostic examination.

In addition to the above, users intending to use the ultrasound imaging function must have appropriate training in ultrasound. Appropriate information on training may be obtained by contacting EchoNous or your local professional body.

Kosmos classifications

- Kosmos has an internal battery which allows operation when AC power is not available.

Patient environment

- The Kosmos power supply classification for protection against electric shock: Class II equipment.
- Kosmos Torso, Kosmos Torso-One and Kosmos Lexsa are Type BF Applied Parts. The Applied Parts include:
 - The lens (front surface) of the probe
 - ECG electrodes, as connected to the Kosmos ECG patient cable
- Kosmos Bridge is IP22
- Kosmos Torso, Kosmos Torso-One and Kosmos Lexsa are IPX7


Patient environment

Kosmos is intended to be used in a medical facility. It is battery powered and is expected to be used in the patient environment. Scanning can also be performed when Kosmos is plugged into the EchoNous-approved power supply. It is important to only use the EchoNous-approved power supply; if you use another power supply, scanning will be disabled (but Kosmos will continue to charge).

- End of section -


Using Kosmos



Kosmos hardware

 Only use accessories recommended by EchoNous. Do not connect any USB accessories to Kosmos Bridge that are not recommended by EchoNous; doing so may cause electric shock and/or compromise the security of the device. Contact EchoNous or your local representative for a list of accessories available from or recommended by EchoNous.



The following drawings point out the buttons and controls on Kosmos Bridge and Kosmos probes.

Kosmos Bridge

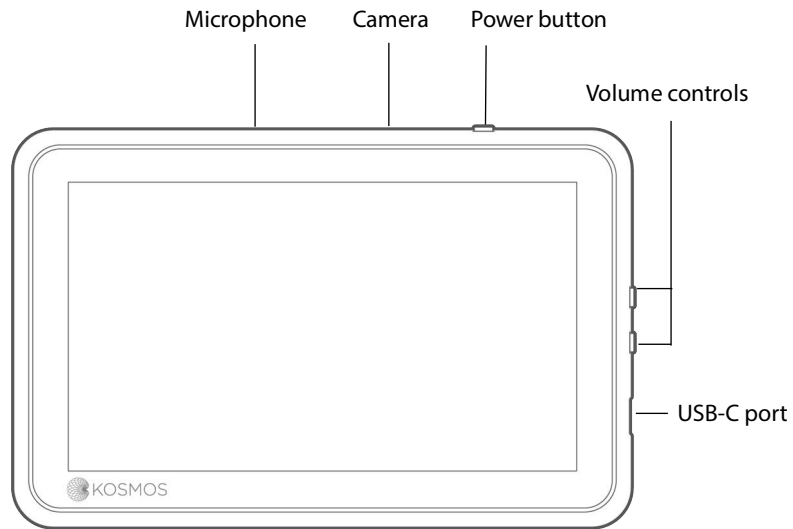


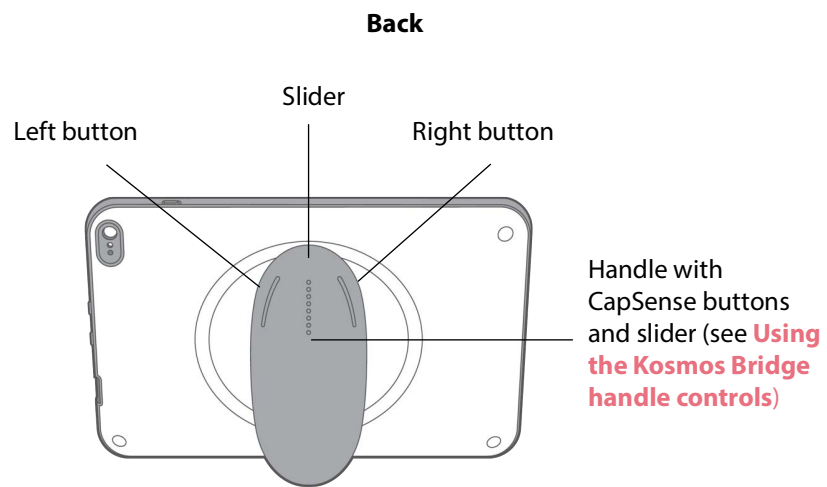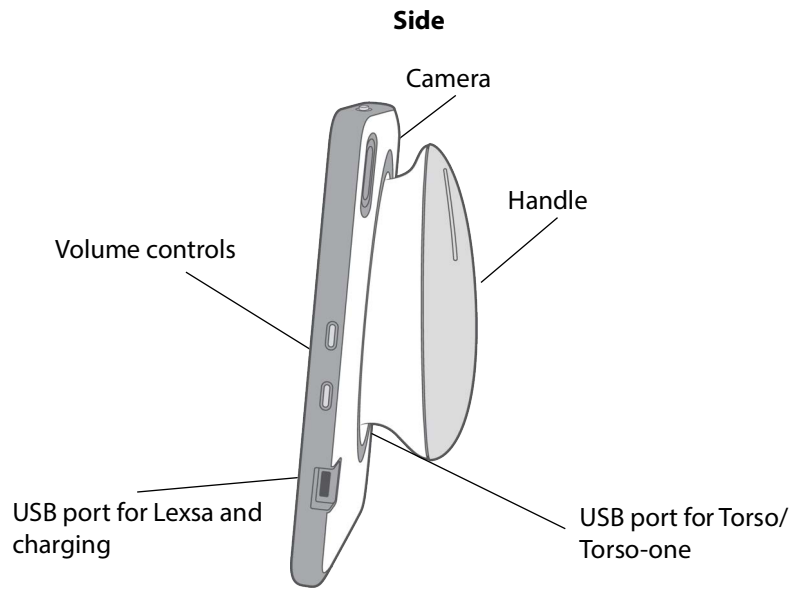


Kosmos Torso, Kosmos Torso-One and Kosmos Lexsa

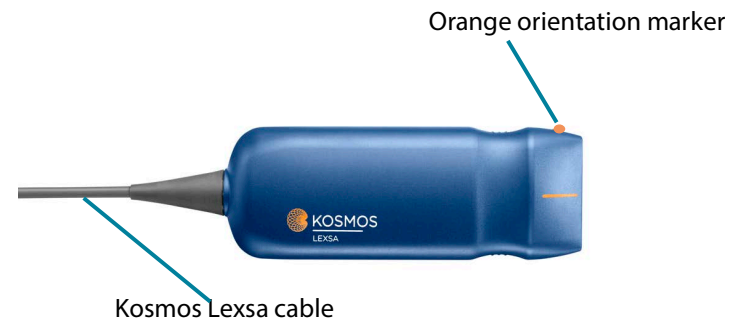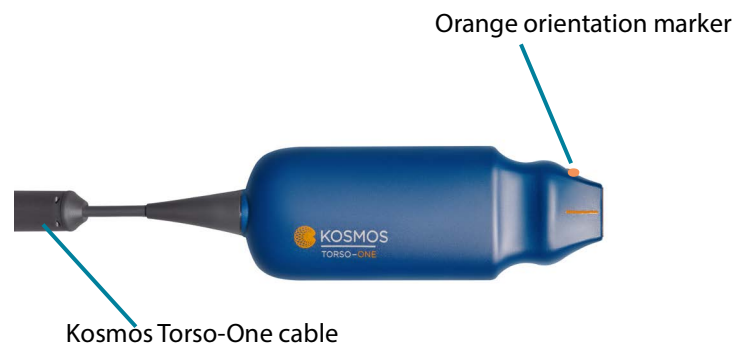

Kosmos Power Supply

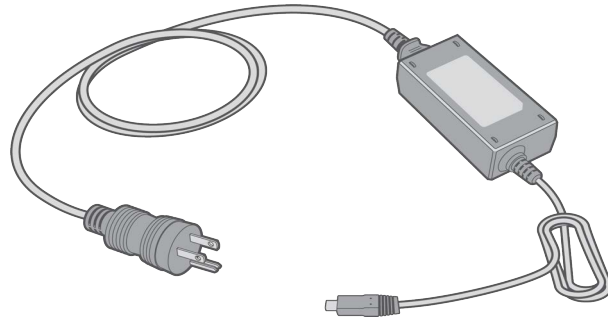


Kosmos ECG Patient Cable*

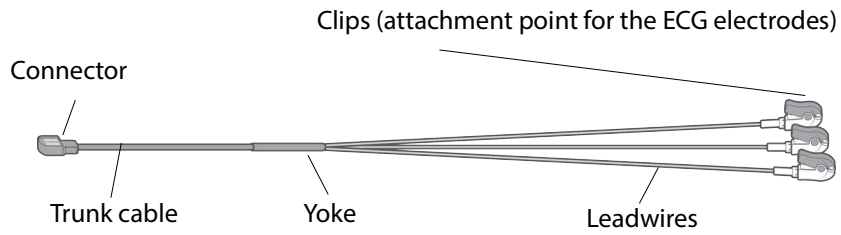

*Only applicable for Torso

Kosmos Digital-to-Analog Converter*

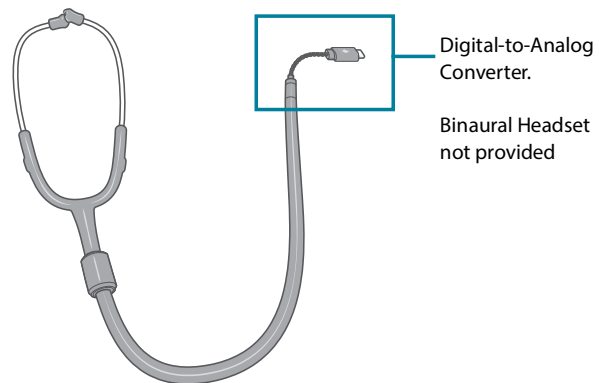

*Only applicable for Torso

■	The detachable USB digital-to-analog converter is provided with the purchase of a new Torso probe. The binaural headset is not included.
⚠	Using a headset that does not meet the headset requirements may result in degraded audio performance when listening to digital auscultation signals.





Binaural Headset Requirements

EchoNous recommended headset requirements:

- Frequency response: 20Hz - 20KHz
- Stereo Male 3.5mm Jack Connector
- Active noise cancellation is preferred


■	EchoNous recommends using wired headphones with closed back for the best performance. Users may experience delay in audio while using Bluetooth headphones.
---	---


Connecting Kosmos Probes

	Before each use, inspect Kosmos Torso, Kosmos Torso-One or Kosmos Lexsa for damage, such as cracks, splitting, or sharp edges. If damage is evident, discontinue using the probe, and contact your EchoNous representative.
	Only use accessories recommended by EchoNous. Do not connect Kosmos Torso, Kosmos Torso-One or Kosmos Lexsa into any device other than Kosmos Bridge.
	Do not attempt to plug Kosmos Torso or Kosmos Torso-One into the side USB port.
	While scanning with Lexsa, do not disconnect the AC power if it is already connected.

To connect Kosmos Torso or Kosmos Torso-One to Kosmos Bridge:


- ★ Plug the Kosmos Torso or Kosmos Torso-One connector into the slot below the Kosmos Bridge handle.

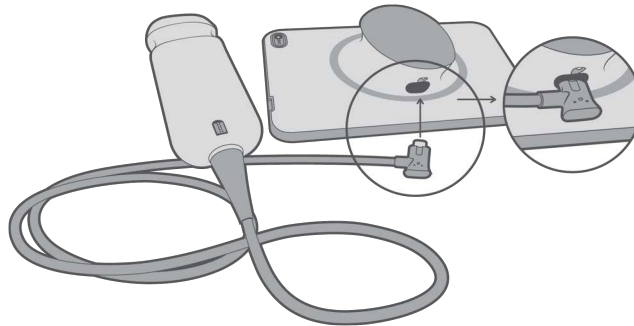

	If Lexsa is connected, disconnect the Lexsa probe to image with Torso or Torso-One.
---	---



To connect Kosmos Lexsa to Kosmos Bridge:

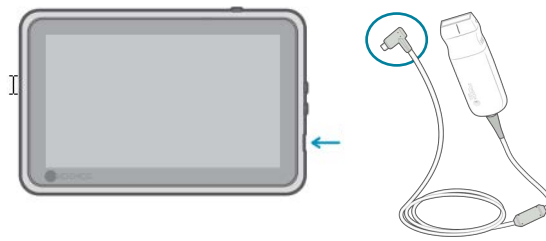- ★ Plug the Kosmos Lexsa connector into the USB port on the side of the Kosmos Bridge.



Connecting the Kosmos power supply


Kosmos Bridge contains an internal rechargeable battery. Recharge Kosmos Bridge using the power supply provided with the device.

	Avoid excessive bending or twisting of the mains power cord.
	Only use Kosmos with power supplies provided by EchoNous. If you try to use a power supply not approved by EchoNous, Kosmos Bridge will continue to charge properly but will disable scanning.

To connect the power supply to Kosmos Bridge:

1. Attach the Kosmos power supply into the USB slot on the side of the Kosmos Bridge.
2. Then plug the other end into an electrical outlet.

Turning Kosmos Bridge on and off

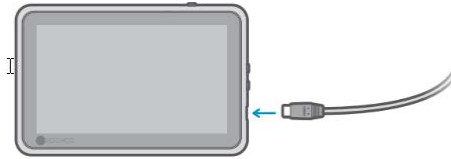

Turning on Kosmos Bridge

To turn on Kosmos Bridge:

3. Press the **Power** button. Connect the probe (s). Select the appropriate probe on the Home screen.
4. Tap the organ of your choice to start scanning.



- If the administrator has set a PIN for security purposes, type it when prompted. However, if you need to start scanning right away, tap **EMERGENCY**.
- To save patient data after scanning, type the PIN to log on to the device, then you can save the exam.

Turning off Kosmos Bridge


To turn off Kosmos Bridge:

1. Press the **Power** button.
2. Do one of the following:
 - When prompted, tap **OK**.
 - Wait the few seconds for Kosmos to turn itself off.

Selecting the Kosmos App


To select the Kosmos App:

- ★ Tap the orange Kosmos icon from the **Select Kosmos Experience** screen.

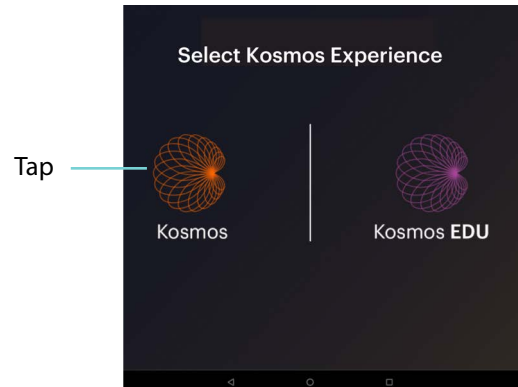

To navigate back to the App Launcher Screen:

- ★ When in the Kosmos home screen, tap **Exit** located in the lower right hand corner of the screen.

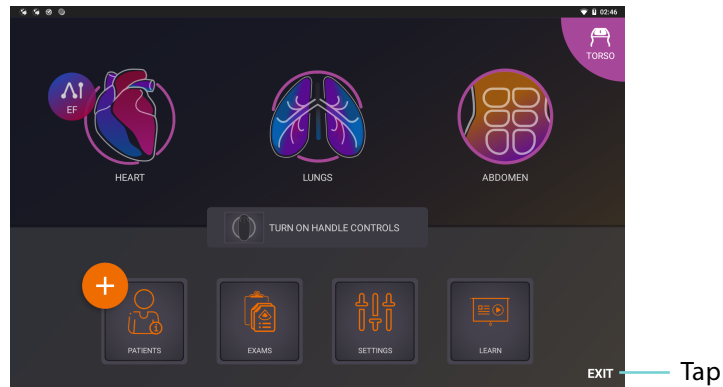

Using the Kosmos Bridge handle controls

The Kosmos Bridge handle is equipped with two buttons and one slider using CapSense technology. These buttons are protrusions on the handle that make it

easier to find them while scanning. The buttons do not move when touched, but are sensitive to light touch, just like the touchscreen on the front of the Bridge.

The handle controls respond to a single tap, double tap, and up-and-down sliding gestures. Once enabled, these controls allow you to control key imaging functions, without lifting your scanning hand from the patient, such as:

- Freezing/unfreezing an image
- Saving an image
- Saving a clip
- Adjusting the gain
- Adjusting the depth




The handle controls work only during live imaging and while an image is frozen.

If you have problems with the handle controls (such as one or more buttons not working), see [Troubleshooting](#).

Switching Probes

If multiple probes are connected to the Kosmos Bridge, easily switch between the probes by tapping the desired icon in the top right corner of the Home screen. The selected probe will appear bigger than the other probe icon.

Tap to select probe

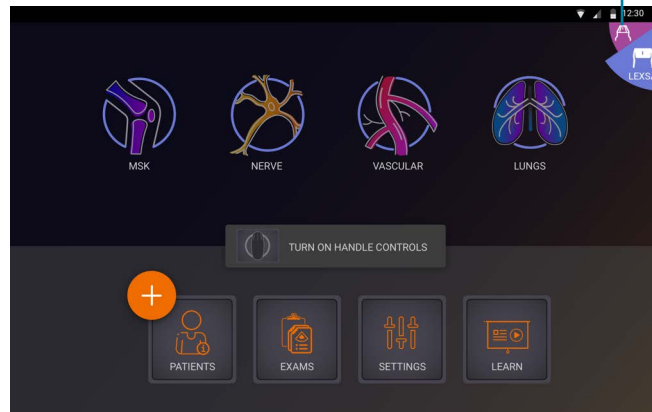

Turning on the handle controls


By default, the Kosmos Bridge handle controls are turned off. The handle controls are available only during imaging and that can be directed by the handle (B-mode, M-mode, B+C mode, EF workflow).

To turn on the handle controls:

- ★ From the Home screen, tap **TURN ON HANDLE CONTROLS** and tap **On**.




To see the handle control mappings from B mode imaging, tap the handle icon.

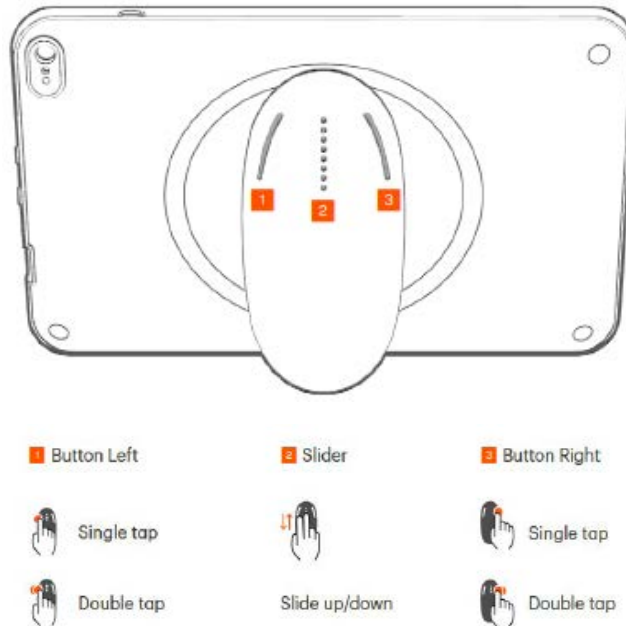

The imaging functions that can be controlled by the handle have teal and purple boundaries.

A single boundary means a single tap and double boundaries mean double tap.

In B-mode imaging, single tap the left button to select between Depth and Gain. The selected control has a purple boundary. You can slide up and down to adjust the selected control.

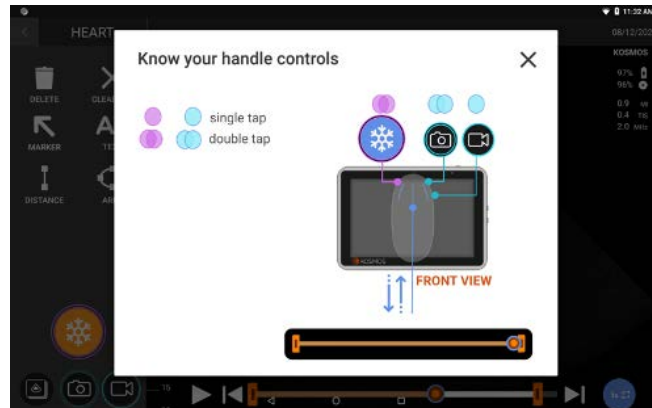


Similarly, on the cine review screen, you can use handle controls to freeze/unfreeze, save image and save clip. Use the slider to move the cine knob between the cine fences.

Ergonomic considerations while using the handle controls


	If using the handle controls cause you discomfort or pain, try adjusting your grip to a more comfortable, neutral position to minimize strain; otherwise, use the on-screen controls instead. Long-term strain can lead to a repetitive stress injury.
--	--

To hold Kosmos Bridge so there is minimal risk of repetitive stress injury:

- Hold Kosmos Bridge in a relaxed position, so that you do not bend your wrist.

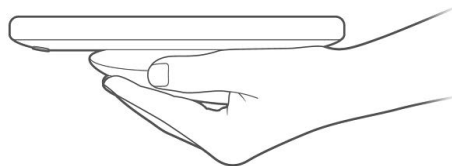


- Place your index and middle fingers on all three controls so they are easily accessible.

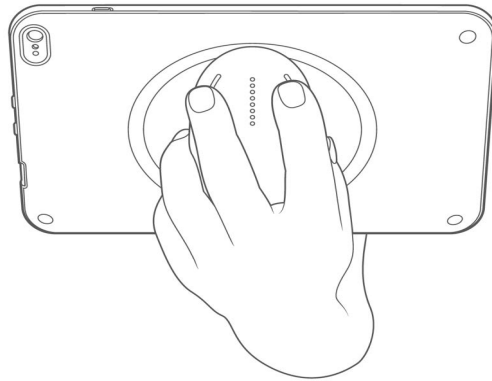


General interaction

Home Screen: Kosmos Torso and Kosmos Torso-One

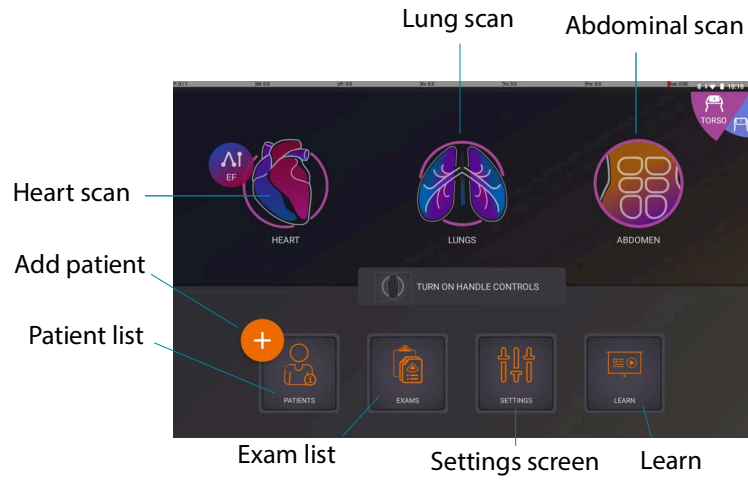

Home Screen: Kosmos Lexsa

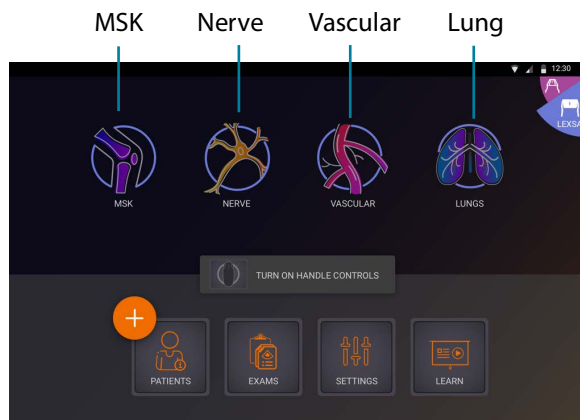

Learn


Tap **Learn** to access how-to-videos and quick guides.

On-screen keyboard

When filling out patient forms or configuring settings in Kosmos, you can type text by tapping the text field you want to edit. An on-screen keyboard appears.

Understanding the different waveforms

ECG & DA *

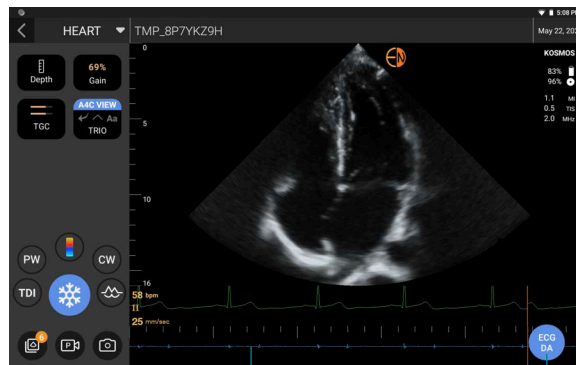

The DA audio plays in synchronization with the visualization of the DA waveform. You can adjust the volume of the audio (and mute the audio) with the physical buttons on Kosmos Bridge.

Tap ECG/DA button to adjust gain and lead.

*The ECG wave form and DA wave form are only available when Kosmos Torso is connected.

ECG

ECG Gain refers to the amplitudes of the ECG waveform. Modify the amplitudes of the ECG waveform by increasing and decreasing the ECG gain.

ECG speed determines the number of waveforms displayed. Choose the appropriate sweep speed (shared between ECG and DA). A lower sweep speed displays more waveforms, while a higher sweep speed displays fewer waveforms but provides greater details of individual waveforms.



Settings

Once you've configured your system settings, they remain as you set them whenever you log back on to Kosmos Bridge.


Imaging Preferences

The Imaging Preferences screen is where you can customize the information Kosmos Bridge displays on the Imaging screen.

To set the imaging preferences:

1. From the Home screen, tap **SETTINGS**.
2. Tap **Imaging Preferences**.
3. To have certain information display in the top bar of the Imaging screen, tap one of the following options under **Customize information**:
 - **Name of facility**—Displays the name of your organization in the top bar of the imaging screen.
 - **Patient name**—Displays the patient name in the top bar of the imaging screen.
 - **Patient ID**—Displays the patient ID in the top bar of the imaging screen.
4. To configure the way Kosmos records clips, tap one of the following options under **Record clip**:
 - **Retrospective**—Captures frames from the cine buffer when you tap the Clip  icon. Kosmos captures cine buffer frames for the number of seconds.
 - **Prospective**—Captures frames after you tap the Record Clip  icon. Kosmos captures frames for the number of seconds.
5. To set how long the clips record, select a time from the **Clip duration** area.



During an exam, if you tap the Record Clip  icon again, you can finish the recording earlier than the clip duration defined here.

6. To adjust the horizontal screen split between M-mode and B-mode, select from the following options under **M-mode layout**:
 - **1:2**—Tap this option to adjust the screen split so the M-mode area is twice as big as B-mode.
 - **1:1**—Tap this option to adjust the screen split so that the M-mode and B-mode areas are equal.
7. From the **Thermal index display** area, select from the following:
 - **TIS**—Thermal index for soft tissue.
 - **TIB**—Thermal index with bone near the focus.
8. Select the **cardiac imaging orientation** preset:
 - Select Left or Right orientation.
9. For PW and CW modes, select one of the following:
 - Synchronized focal point/gate and color box
 - Decoupled focal point/gate and color box

Configuring ECG and DA Signals

Ultrasound is always configured with DA, ECG or DA and ECG when Kosmos Torso is connected to Kosmos Bridge.

To configure the horizontal screen split between the ultrasound, ECG, and DA signals:

1. Connect Kosmos Torso probe to Bridge.
2. From the Home screen, tap **SETTINGS**.
3. Tap **ECG & DA Signals**.
4. Select the layout that best suit your needs.

Setting the language, date, and time

Turning on the automatic date and time will not automatically select time zone. You have to manually adjust the time zone.

To set the language, date format, time format and time zone for Kosmos:

1. From the Home screen, tap **SETTINGS**.
2. Tap **Language, Date, and Time**.
3. From the **Language** drop down list, select the language of your choice.
4. Make a selection from the Date Format drop down list
5. If you would like the time to display in 24-hour format, tap the switch to toggle on.
6. To automatically set date and time, tap the switch to toggle on or off. If selected off, you can manually set the date and time.

Adjusting the Sound

There are two ways to adjust the media volume:

1. From the Home screen, tap **SETTINGS**.
2. Tap **Sound**.
3. Move the slider to adjust the media volume.

or

1. Slide your finger down from the top of the screen.
2. From the drop down box, move the slider to adjust the media volume.

There are two ways to adjust the notification volume

To adjust the volume:

1. From the Home screen, tap **SETTINGS**.
2. Tap **Sound**.
3. Move the slider to adjust the notification volume.
4. For touch sounds, use the toggle switch to turn touch sounds on or off.
5. To change the notification sound, tap Default notification sound and make a selection from the drop down list.

or

1. Use the buttons on the side of the Bridge to adjust the notification volume.

Setting Brightness



There are two ways to adjust the brightness of the display screen:

1. From the Home screen, tap **SETTINGS**.
 2. Tap **Brightness**.
 3. Move the sliders to adjust the brightness level
- or
1. Slide your finger down from the top of the screen.
 2. From the drop down box, move the slider to adjust the brightness level

Screen mirroring (Miracast)

You are able to cast the Bridge screen to another supported device.

1. From the Home screen, tap **SETTINGS**.
2. Select **Connected Devices**.
3. Select **Cast**.

	The Wi-Fi setting must be turned ON to be able to cast to another screen.
	Do not use as a diagnostic tool while using Miracast feature. Image quality may vary based on the screen you are sharing to.

About

The About section is where you will find essential information regarding your device, such as the serial number, HW ID, system software version, registration status and licensed features. Your probe must be connected to view the probe's serial number, firmware version, UPS version, licensed features and to run the Transducer Element Check. Additionally, legal information, regulatory certification, contact information and system logs can be found in the About section.

1. From the Home screen, go to **SETTINGS --> About**.
2. If you have not registered Kosmos, tap **Register**. This will connect your Kosmos device to the EchoNous cloud. Make sure your device is connected to

the internet.

3. To run the transducer element check, tap **Check**.

Admin

The Admin section is where you will find:

- Security (Admin and user PINs)
- Automatic power off and sleep settings
- Us2.ai measurements (if applicable)
- DICOM settings
- Exam preferences
- Export preferences
- Software updates
- Wi-Fi preferences, VPN preferences
- Factory reset
- System Security (System screen lock, encryption and credentials)


Admin settings may only be configured by the Kosmos Administrator.

Security


For additional security, you have the option of setting up an administrator PIN, a clinical user PIN, or no PIN at all. If you do choose to set up PINs and then forget your PIN, you can still scan using the emergency feature (but you won't be able to save the exam).

If Kosmos is only used by one person, then you may not want to set up a PIN. However, if the device is going to be used by more than one person, we recommend setting up both administrator and clinical user PINs. The administrator PIN provides access to all of the Kosmos screens, and the clinical

user PIN provides access to all of the Kosmos screens, with the exception of the administration settings screens.

 It's very important to keep track of the PINs you create and store them in a safe place. If you forget your PIN, you must contact EchoNous Customer Support, and they will send you a one-use USB stick so you can change your PIN.

SETTING UP A PIN

 It is important to enable device PIN and Admin PIN for maximum security of patient data stored on the device.

To set up a PIN:

1. From the Home screen, tap **SETTINGS**, then **Admin**.
2. Tap **Security**.
3. Tap to select the **Enable Admin** PIN check box.
4. Type a six-digit numeric PIN, and click **OK**.
5. You now have a choice of how you would like to set up your PINs.

If you choose...	Can scan in Emergency mode?	Can save & review patient data?	Can access admin settings?
No PIN	Anyone	Anyone	Anyone
Admin PIN only	Anyone	Anyone	Administrators enter Admin PIN
Admin PIN & Restrict access to Home screen	Anyone	Administrators enter Admin PIN	Administrators enter Admin PIN
Admin PIN & basic PIN	Anyone	Administrators enter Admin PIN; users enter user PIN	Administrators enter Admin PIN

CHANGING A PIN

To change a PIN:

1. From the Home screen, tap **SETTINGS**, then **Admin..**
2. Tap **Security**.
3. To change the administrator PIN, tap **Change admin PIN**, and type the new PIN number.
4. To change the user PIN, tap **Change user PIN**, and type the new PIN number.

REMOVING A PIN

To remove a PIN:

1. From the Home screen, tap **SETTINGS**, then **Admin**.
2. Tap **Security**.
3. Tap to clear the check box.

Setting the auto power off and auto sleep time interval

During periods of inactivity, Kosmos automatically switches to sleep mode to preserve battery life.

If Kosmos is in sleep mode, briefly press the **Power** button to wake it up; the display does not indicate activity when Kosmos is asleep.

To change the sleep mode time interval:

1. From the Home screen, tap **SETTINGS --> Auto power off & sleep**.
2. Select the sleep mode time interval: 5, 10, or 30 minutes.
3. (Optional) Check the box to have your device automatically power off when in sleep mode and select the time interval: 5, 10, or 30 minutes.
4. (Optional) Check the box to automatically complete the in-progress exam when Kosmos Bridge goes to sleep.

Us2.ai Measurements


License(s) must be purchased to use Us2.ai's Us2.v1 software with Kosmos. To purchase a license, please contact your EchoNous representative.

To connect to the Us2.ai Cloud or to configure settings when using Us2.ai and Kosmos:

1. From the Home screen, tap **SETTINGS**.
2. Tap **Admin --> Us2.ai Measurements**.
3. Enter credentials to connect to Us2.ai Cloud.
4. Check the box to anonymize patient information.
5. To auto send images/clips, toggle on.

DICOM Settings


Manage your modality worklist (MWL) and picture archiving and communication system (PACS) from DICOM settings.

- | | |
|---|--|
|  | <ul style="list-style-type: none">• New systems do not come with any configured profiles.• You cannot have two PACS profiles active at the same time; when you add a new profile, the current one is deactivated. |
|---|--|

Managing PACS

To add a PACS profile:

1. From the Home screen, tap **SETTINGS**.
2. Tap **Admin --> DICOM --> PACS archive**.
3. Tap **ADD PROFILE**.

- | | |
|---|--|
|  | <p>If you are adding a new PACS-SCP profile and already have an existing one, the system deactivates the existing profile. However, all the jobs in the existing queue and any scheduled archives must first be completed.</p> |
|---|--|

4. Type the following information in the **DICOM connection** area:
 - **Station AE title**—Kosmos' Application Entity title.
 - **Server AE title**—Archive server's Application Entity title.
 - **Server IP address**—Archive server's unique identifier.
 - **Server port number**—Archive server's port number.

5. To make sure the connection is working on an active profile, tap one of the following:
 - **PING** to test the network connection between Kosmos and the PACS archive.
 - **Verify** to check the availability of the active PACS archive.
Kosmos Bridge displays the results on-screen.
6. In the **Profile nickname** box, type a unique name to display in the PACS profile list.
7. In the **Archival options** area, you have three options:
 - **Prompt options every time** - Switched on by default; each time you tap the **Archive** button from the Exam review screen, a pop-up menu with different options displays. If you turn the switch off, Kosmos does not display the pop-up menu.
 - **Attach report**—Switched off by default. If you turn it on, Kosmos attaches a report to the archive.
 - **Attach the DICOM SR report** — Switched off by default. If you turn it on, Kosmos attaches a DICOM SR report to archive.



The three Archival options are also available in the USB export options.

8. In the **Auto archive** area, select from the following options:
 - **On/Off**—The auto archive is switched off by default. This means that all the controls (except the on/off switch) are disabled and cannot be edited. If you turn the switch on, all the controls are enabled and can be edited.
 - **Archival frequency**
 - In-progress archive- Images are archived upon acquisition.
 - **Completion of exam**—The archival time selector is disabled.
 - **Daily**—Only the time section of the archival time selector is enabled.
 - **Weekly**—The complete archival time selector is enabled.
 - **Archival time**—Select a daily time and day to archive exams.
9. In the **Retry interval (in seconds)** area, select **60,300**, or **600**.


10. In the Maximum retries area, select 1, 2, or 3. To have the system automatically retry failed jobs, keep the switch set to **On**; otherwise, slide it to **Off**.


DEACTIVATING A PACS PROFILE

To activate or deactivate a profile, in the PACS archive list, tap the switch to toggle between Active and Inactive.


DELETING A PACS PROFILE

To delete a PACS profile:

	Deleting a PACS profile also deletes all configurations of the profile. There must be an active PACS profile before you can archive any exams.
---	--

1. From the Home screen, tap **SETTINGS**.
2. Tap **Admin --> DICOM --> PACS archive**.
3. From the list of profiles, tap to slide the arrow to the left of the profile you would like to delete.
4. Tap the Delete  icon.

MANAGING MWL

	New systems do not come with any configured profiles. You cannot have two MWL profiles active at the same time; when you add a new profile, the current one is deactivated.
---	---

ADDING MWL PROFILE:

1. From the Home screen, tap **SETTINGS**.
2. Tap **Admin --> DICOM --> MWL**.
3. Tap **ADD PROFILE**.



If you are adding a new MWL profile and already have an existing one, the system deactivates the existing profile.

4. Type the following information in the DICOM connection area:
 - **Station AE title**- Kosmos' Application Entity title.
 - **Server AE title**- Archive server's Application Entity title.
 - **Server IP address**- Archive server's unique identifier.
 - **Server port number**- Archive server's port number.
5. To make sure the connection is working on an active profile, tap one of the following:
 - **PING** to test the network connection between Kosmos and the MWL server.
 - **Verify** to check the availability of the active MWL server.
 - Kosmos Bridge displays the results on-screen.
6. In the **Profile nickname** box, type a unique name to display in the MWL profile list.

DEACTIVATING MWL PROFILE


To activate or deactivate a profile, in the **MWL** list, tap the switch to toggle between **Active** and **Inactive**.

DELETING MWL PROFILE

To delete a MWL profile:



Deleting MWL profile also deletes all configurations of the profile.

1. From the Home screen, tap **Settings**.
2. **Tap Admin --> DICOM --> PACS archive.**
3. From the list of profiles, tap to slide the arrow to the left of the profile you would like to delete.
4. Tap the **Delete**  icon.

Exam Preferences


You have the option to limit the number of patients that can be stored on the device. This also includes temporary patient records with no patient name. Once the device reaches the patient cap limit, it will ask you to delete patients to continue scanning. When you delete a patient, all associated exams associated with the patient are also deleted.

PATIENT CAP OR LIMIT PATIENT COUNT

1. From the Home screen, tap **SETTINGS**.
2. Tap **Admin** --> **Exam Preferences**.
3. Swipe to turn the Limit Patient Count **ON**.
4. Select the number of patients allowed on the device.

ENABLE AUTOMATIC EXAM DELETION PREFERENCE

1. From the Home screen, tap **SETTINGS**.
2. Tap **Admin** --> **Exam Preferences** --> **Auto Delete**.
3. Swipe to turn **ON** the option.

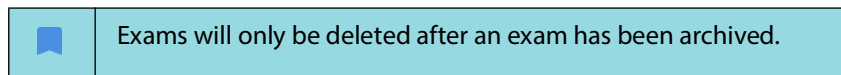

USB Export

To configure USB export preferences:

1. From the Home screen, tap **SETTINGS**.
2. Tap **Admin** --> **USB export**.
3. Check the box to enable exporting exams to USB drive.
4. Check to box for Kosmos to prompt export options every time.
5. Select the file type.

Installing software updates



Before updating the software, back up all patient data.

You can manually check for software updates or configure Kosmos to automatically check to see if there is a new update available. You can also choose to have Kosmos automatically download and install any updates.

To manually check to see if there is a software update available:

1. Make sure you are connected to your network (see **IT Network**).
2. From the Home screen, tap **SETTINGS**.
3. Tap **Admin--> Software Updates**.
4. Tap **CHECK FOR UPDATES**.

To install an update:

1. From the Home screen, tap **SETTINGS**.
2. Tap **Admin--> Software Updates**.
3. Tap **Updates NOW**

Managing network and internet settings

For more information about functions, security, and recovery, refer to the chapter **IT Network**.

To manage network and internet settings:

1. From the Home screen, tap **SETTINGS**.
2. Tap **Admin**.
3. Tap **Wi-Fi**.
4. Toggle the switch to turn Wi-Fi on or off

VPN


To add a VPN:

1. From the Home screen, tap **SETTINGS**.
2. Tap **Admin** --> **VPN**.
3. Tap the plus icon to add a VPN. A lock screen PIN or password will need to be set before you can use credential storage.

Factory Reset

You can restore Kosmos to its factory settings; however, be aware that this will erase all the data from internal storage including:

- Patient data
- PACS settings
- Device settings
- Other user data

	To avoid losing data, be sure to save any data prior to completing the factory reset.
---	---

To complete the factory reset on your device:

From the Home screen, tap **Settings**.

1. Tap **Admin**.
2. Tap **Factory Reset**.
3. Tap **RESET**.

System Security

To add a screen lock to your device:

1. From the Home screen, tap **SETTINGS**.
2. Tap **Admin** --> **System Security**.
3. Tap Screen lock and select screen lock type from the drop down list.

Report Settings

To customize the report settings:

1. From the Home screen, tap **SETTINGS**.

2. Tap **Report Settings**.

3. To customize how the measurements will show up on the Cardiac Report, for each measurement select one of the following:

- **Last** measurement taken
- Average (**Avg**) measurement
- Maximum (**Max**) measurement

4. Select the metrics for distance and velocity.



Regardless of the report settings, users can select Last, Avg or Max during an exam.

Wireless networking

Functions

You can connect Kosmos to an IT network to perform the following:

- Storing exam data (static images and clips) acquired by Kosmos in Picture Archiving and Communication System (PACS) by DICOM communication.
- Setting Kosmos time correctly by inquiring the network time service.

Connection specifications

Hardware specification

802.11 a/b/g/n/ac, Bluetooth 4.0 or later.

Software Specification

Kosmos is connected to PACS by the DICOM standard. For details, refer to the DICOM Conformance Statement that is on the EchoNous website..

-- End of Section --

Use Restriction

This device is restricted to indoor use when operating in the 5150 to 5350 MHz frequency range. This restriction applies in: AT, BE, BG, CH, CY, CZ, DE, DK, EE, EL, ES, FI, FR, HR, HU, IE, IS, IT, LI, LT, LU, LV, MT, NL, NO, PL, PT, RO, SE, SI, SK, TR, UK.

-- End of Section --

Intentionally left blank.

Incorporating ECG and DA Signals

Overview

When Kosmos Torso is connected to Kosmos Bridge, ECG and DA signals can be synchronized with ultrasound in real-time. ECG and DA signals are only available with Kosmos Torso.

ECG

Electrocardiography is the process of recording the electrical activity of the heart over a period of time using electrodes placed over the skin. These electrodes detect the tiny electrical changes on the skin that arise from the heart muscle's electro physiologic pattern of depolarizing and re-polarizing during each heartbeat. The graph of voltage versus time produced by this noninvasive medical procedure is an electrocardiogram (ECG). The horizontal axis represents time, and the vertical axis represents voltage.

With respect to the Kosmos ECG capability, the Kosmos ECG feature uses a three-lead, single-channel ECG that allows the acquisition and display of a single ECG lead, which can be any of Lead I, Lead II, or Lead III.

The Kosmos ECG feature is used with the Kosmos ECG patient cable. One end of the Kosmos ECG patient cable connects to Kosmos Torso, and the other end has three RA/LA/LL leadwires. The leadwires and associated clips are connected to the patient using the standard RA/LA/LL configuration. This allows, at any one time, for a single ECG lead (either Lead I, Lead II, or Lead III) to be acquired and displayed by Kosmos. The user can select which lead to acquire and display by using Kosmos Bridge.

ECG has been traditionally used in ultrasound to provide a timing reference for the cardiac cycle, and it can do the same for digital auscultation (DA). Kosmos ECG serves as a timing reference for both ultrasound and DA signals, and it can also be used to look at the acquired and displayed ECG lead for HR measurement and rhythm assessment by qualified and trained healthcare professionals.

DA

Auscultation is achieved by listening to the internal sounds of the body, usually using a stethoscope, for the purpose of examining the circulatory and respiratory systems (heart and lung sounds), as well as the gastrointestinal system (bowel sounds).

When auscultating the heart, clinicians listen for abnormal sounds, including heart murmurs, gallops, and other extra sounds coinciding with heartbeats. HR is also noted. When listening to lungs, breath sounds such as wheezes, crepitation, and crackles are identified. The gastrointestinal system is auscultated to note the presence of bowel sounds. Digital auscultation (DA) is a digital form of auscultation. It includes the recording, visualization, storage, analysis, and sharing of digital recordings of heart, lung, or abdominal sounds.

The visualization of sounds in DA is accomplished with waveforms that are presented to the user in real-time while the acquisition is taking place. In the case of heart sounds, these waveforms are also known as phonocardiograms.

Benefits of using ECG and DA signals with ultrasound

Ultrasound imaging, ECG, and DA are all integrated into Kosmos Torso in a time-synchronized manner. Being able to view the real-time, synchronized signals of ultrasound, ECG, and DA is a valuable cross reference between different views of the same physiological event.

- **Ultrasound** provides an anatomical view of the motion of the heart.
- **DA** provides auditory and visual (through the phonocardiogram waveforms) feedback regarding the heart valves.
- **ECG** provides information about the electrical activity that drives the heart contractions.

Using the Kosmos ECG patient cable



The Kosmos ECG patient cable connects to Kosmos Torso by means of coupling magnets. Kosmos Torso contains a small permanent magnet where the Kosmos ECG patient cable connects. Do not use Kosmos on patients with cardiac pacemakers or other electronic implantable devices.



The Kosmos ECG patient cable connects to Kosmos Torso by means of coupling magnets. The Kosmos ECG patient cable contains a small permanent magnet at the device connector. Do not use Kosmos on patients with cardiac pacemakers or other electronic implantable devices.



The Kosmos ECG patient cable is not defibrillation proof.



The Kosmos ECG functionality is a Type BF. Kosmos ECG functionality is not for use in situations, such as patient monitoring, where the patient has exposed leadwires that are in direct cardiac contact. Conductive parts of electrodes and associated connectors for Type BF Applied Parts, including the neutral electrode, should not contact other conductive parts including earth.



Kosmos may not accurately report HR in the case of irregular rhythms.



Kosmos is not a substitute for diagnostic ECG. This device does not detect or measure all HR, heart rhythm, and heart waveform changes.



Conducted RF energy may cause noise in the ECG waveform. If noise is detected on the ECG waveform, disconnect Kosmos from AC power.

To use the Kosmos ECG patient cable:

1. Place the ECG electrodes of your choice (this is where the ECG clips will be attached to) on the patient, making sure they are placed symmetrically opposite from each other and match the color coding.


USA recommendation (American Heart Association):

- **RA:** Right arm (white clip)
- **LA:** Left arm (black clip)
- **LL:** Left leg (red clip)


IEC recommendation:

- **R:** Right arm (red clip)
- **L:** Left arm (yellow clip)
- **F:** Left leg (green clip)

ECG Electrode Placement Guide

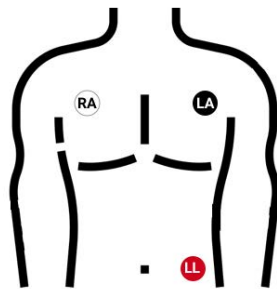

AHA (American Heart Association)

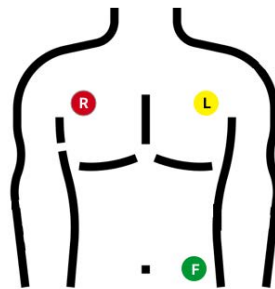

IEC (International Electrotechnical Commission)

2. Plug the connector end of the Kosmos ECG patient cable into the magnetic slot on Kosmos Torso.




Attaching binaural headset

- The DA signal is built into Kosmos; there are no external microphones/sensors to hook up.
- To listen to the DA signal, you must plug the binaural headset into the USB port on Kosmos Bridge. Use the digital-to-analog converter as necessary.

The DA microphones and signal processing are built into Kosmos. Users must provide their own binaural headset. EchoNous provides the digital-to-analog converter.

To attach a binaural headset:

1. Connect binaural headset to digital-to-analog converter.
2. Plug the USB-C end of the converter into the USB-C port on the side of the Bridge.

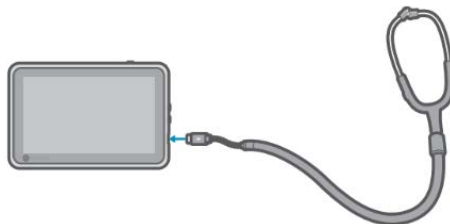

3. Put the headset on.
4. On Kosmos Bridge, tap the **ECG/DA Signals** tab.

5. Tap **DA** to turn it on.
6. From the top of the screen, slide your finger down to see the volume control.
7. Adjust the volume.

Viewing the ECG and DA signals



The ECG and DA signals are only available in B-mode and Color-mode.

1. Tap the **ECG/DA** tab to display the two signal controls. By default, only the ultrasound image displays.
2. To view the ECG signals, tap **ECG** on; tap again to turn it off.
3. To view the DA signal, tap **DA** on; tap again to turn it off.
4. To select which ECG lead is to be acquired and displayed, tap **Lead**.

Signal scrolling

The ECG and DA signals scroll from left to right. The newest signals appear on the left and are indicated by the orange cursor. When the scrolling begins, the area to the right of the cursor is blank, while the new scrolling overlaps the old signals from the second round of scrolling. The DA audio is synchronized with the DA waveform scrolling.

ECG signal indicator

If the signal is weak or you cannot read it on-screen, check to make sure:

- You are holding Kosmos Torso still.
- The patient is not moving.
- The connection of leadwires to Kosmos Torso is not loose.

Preserving the ECG and DA signals when freezing an image or taking a clip

You can freeze an image or take a clip with the ECG and DA waveforms so you can review them in the Editing screen. What you see in the Imaging screen is what gets saved, so if you turn off any of the signals while in live imaging mode and you save an image or clip, only the signals displayed on the screen are saved.

For more information on viewing the ECG and DA signals when reviewing a saved exam or clip, refer to **Reviewing an Exam**.

Archiving and exporting ECG and DA waveforms

When you archive exams to the PACS server, the ECG and/or DA waveforms are embedded into the ultrasound image or clip.






When you export exams to a USB drive, the ECG waveform and the DA waveform and audio signals are embedded into the ultrasound image or clip. However, you cannot archive or export ECG or DA as a separate file, because the ECG and DA data is not archived separately; they are all part of the ultrasound image or clip.

- End of section -

INTENTIONALLY LEFT BLANK

Performing an Exam

Overview

	Before using Kosmos Bridge for a critical procedure, such as needle guidance, make sure it is fully charged and/or plugged into AC power. You do not want the procedure interrupted by a drained battery, which may cause harm to the patient.
	Under certain circumstances, the Kosmos Bridge enclosure can reach temperatures that exceed safe (IEC 60601-1) limits for patient contact. Make sure that only the operator handles the system. Avoid placing Kosmos Bridge on the patient during use.
	The maximum temperature of a Kosmos probe scan head may be greater than (41C) but is less than (43C) when in contact with the patient for normal use. Special precautions should be considered when using the transducer on children or on other patients who are sensitive to higher temperatures.
	To reduce the risk of infection, use sterile sheaths when conducting needle procedures.
	To avoid a mix-up of patient data, complete the exam before examining a new patient.


Primary Exam Workflows

With Kosmos, there are three primary workflows; click one of the links to go to that workflow:

- **Standard workflow** starts with either creating a patient or searching for an existing patient.
- **Quick workflow** starts with scanning a patient.
- **Kosmos AI-assisted EF workflow** uses AI to perform initial EF calculations.

Exam workflows

Standard workflow

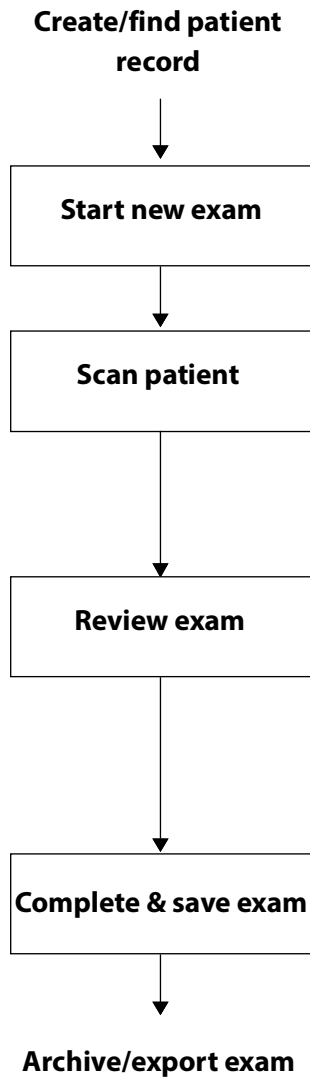

Optional step:

Start scanning immediately, then go back and connect the exam to the correct patient.

Things you can do while scanning:

- Add and delete images and clips
- Add, edit, and delete annotations and notes


Things you can do while reviewing:

- Delete images and clips
- Add, edit, and delete annotations and notes
- Generate report

Optional steps

- Archive exam to PACS
- Export exam to USB

Quick workflow

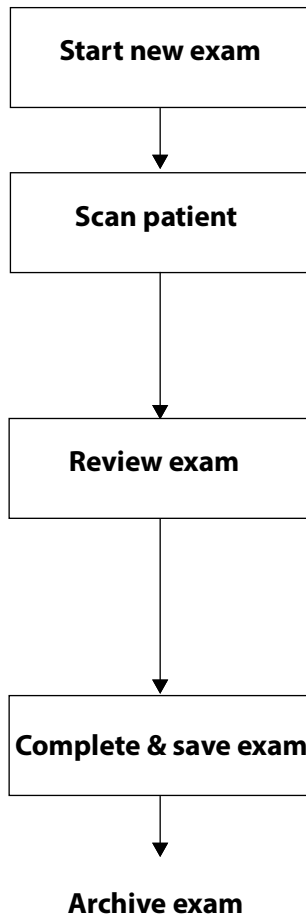

Things you can do while scanning:

- Add and delete images and clips
- Add, edit, and delete annotations and notes


Things you can do while reviewing:

- Delete images and clips
- Add, edit, and delete annotations and notes
- Generate report

Optional steps

- Archive exam to PACS
- Export exam to USB

Kosmos AI-assisted EF workflow

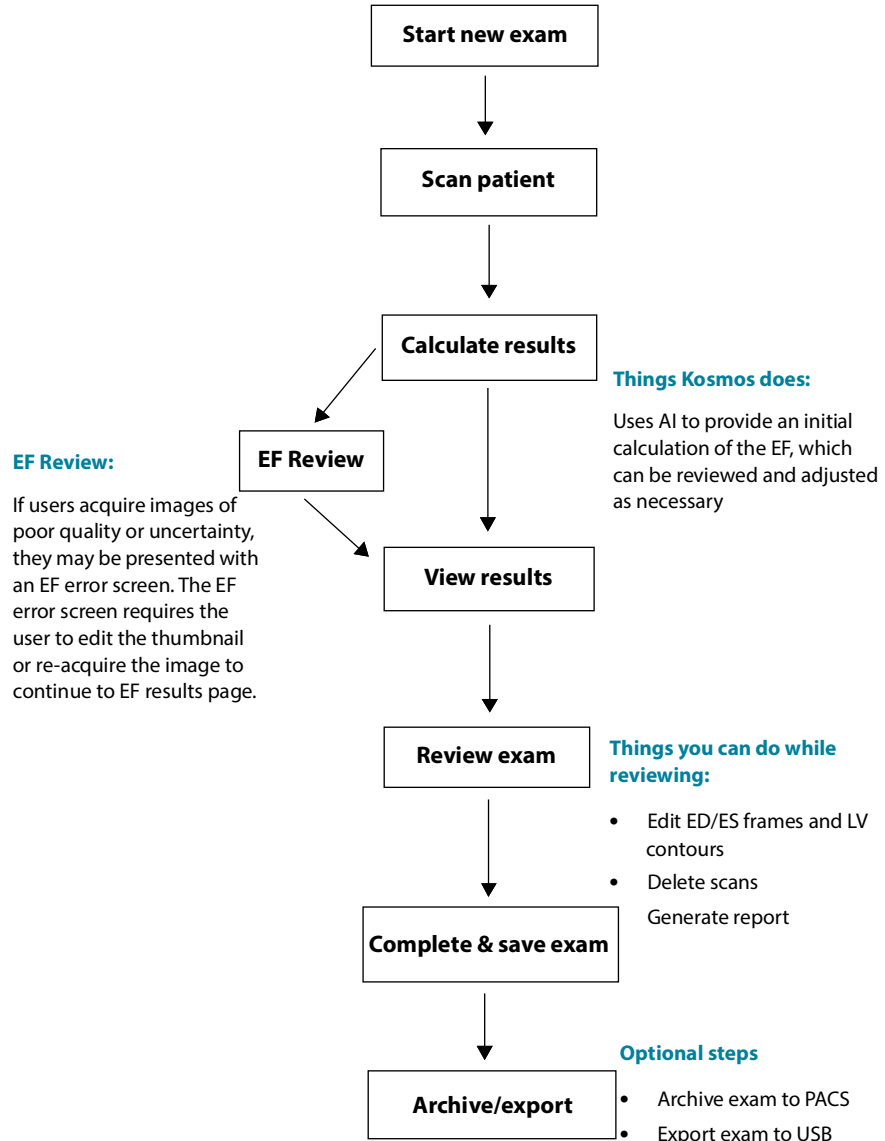

Managing exams


Starting an exam

There are several ways you can start an exam:

- To start scanning immediately, from the Home screen, tap a preset and begin scanning.

When you save the exam, Kosmos automatically generates a temporary ID and saves the images/clips to the **temporary ID**.

•**From the Home screen --> PATIENTS --> NEW PATIENT --> SCAN.**


•Use the Add  icon as a shortcut to add a new patient.

•For existing patients, from the Home screen --> **PATIENTS** --> Select a patient from the patient list --> tap **SCAN**.

•From the Home screen --> **EXAMS** --> **NEW PATIENT** or look up and existing patient --> tap **SCAN**.

Searching for an exam


To search for an exam:

1. From the Exam screen, tap the Search  icon.
2. Type the search criteria, such as date, patient name, DOB, or MRN.
3. From the list of search results, tap the exam you want to view. Each exam listed shows the number of scans taken as shown in the example below.




Deleting exams

To delete one or more exams:

1. From the list of exams, tap one or more circles to the left of the exam. The circle turns into a check mark, showing it is selected.
2. Tap the Trash  icon.

3. At the prompt, tap **OK**.


To delete all the empty exams (those without images/clips):

1. From the list of exams, tap the More options  icon.
2. Tap **Delete all empty exams**.
3. At the prompt, tap **OK**.

Completing exams

To avoid mixing up images and clips saved from multiple patients, make sure to complete an exam.


To complete an exam:

1. From the Imaging screen, tap the Exam review  icon.
2. Tap **Complete**.
3. At the prompt, tap **OK**

.Managing patient data

Adding a new patient



To add a new patient from the Home screen:

1. From the Home screen, tap the Add  icon on the **PATIENTS** button.
2. Enter the patient information.
3. Optionally, you can enter exam information.
4. Tap **SCAN** when you are done.

Accessing patient information using MWL


If you are connected to a healthcare information system and MWL is set up on your Kosmos, you can access patient information.

1. From the Home screen, tap the **PATIENTS** button.

2. Tap the MWL button. Tap the  icon to see the entire list.
3. Tap the  icon to search for a specific patient.
4. Tap **SCAN** to start scanning.

Searching for a patient

To search for a patient:

1. From the Home screen, tap **PATIENTS**.
2. Tap the Search  icon.
3. Type the search criteria for the patient you are looking for, such as name, date of birth, or medical record number.
4. Select the patient from the search result list, and tap **DONE**.

Changing to another patient

To change to or add a new patient when you have already started an exam:

1. From the New Exam screen, tap **CHANGE**.
2. Do one of the following:
 - To change to another patient, tap **ADD NEW**, and complete the patient form.
 - To look for an existing patients, tap **SEARCH HISTORY**, use the search tool to find the patient, and tap the patient name from the list.


Editing a patient record

To edit a patient record:

1. From the Home screen, tap **PATIENTS**.
2. From the Patient list, double-tap the patient record you want to edit.
3. Enter the patient information, and tap **SAVE** when you are done.

Merging two patient records


If you have saved multiple patients with the same name, and they are actually the same patient, you can merge all the exams of that patient into one patient record so it is easier to keep track of that patient.

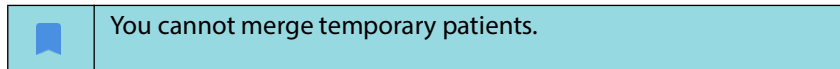

In order to merge two patients, make sure the following fields are complete:


- First name
- Last name
- DOB
- Gender

To merge two patient records:


1. From the Home screen, tap **PATIENTS**.
2. Tap to select one of the patients.
3. From the Patient review screen, tap the More options  icon.
4. Tap **Merge to patient**.
5. From the list, tap the other patient you want to merge.
6. Tap **NEXT**.
7. Tap the fields to keep for the patient.
8. Tap **MERGE**, then tap **OK**.

Deleting patient records

To delete all patient records without exams:

1. From the Home screen, tap **PATIENTS**.
2. Tap the More options  icon.
3. Tap **Delete all patients without exams**.

To delete selected patient records:

1. From the Home screen, tap **PATIENTS**.
2. Tap one or more patient names from patient list.
3. Tap the Trash  icon.

Organ Presets

Table 5-1, “Organ Presets by Kosmos Probe,” on page 64 provides an overview of the organ presets that are available for each Kosmos probe.

TABLE 5-1. Organ Presets by Kosmos Probe

Organ	Torso	Torso-One	Lexsa
Heart	X	X	
Lung	X	X	X
Abdomen	X	X	
Vascular			X
Nerve			X
MSK			X

Imaging Modes and Features

For an overview of the applicable imaging modes and features for each Kosmos probe reference to Table 5-2, “Modes of Operation & Features for Kosmos,” on page 65

TABLE 5-2. Modes of Operation & Features for Kosmos

Mode	Torso	Torso-One	Lexsa
B-mode	X	X	X
M-mode	X	X	X
Color Doppler	X	X	X
Harmonic Imaging	X	X	
AI-assisted EF Workflow	X	X	
CW Doppler	X	X	
PW Doppler	X	X	X
Tissue Doppler (TDI)	X	X	
AI FAST	X	X	
Color Power Doppler (CPD)			X
Us2.ai	X	X	
Cardiac Calculations	X	X	

B-mode (2D)

2D/B-mode is the system's default imaging mode. The system displays echoes in two dimensions by assigning a brightness level based on the echo signal amplitude.


The 2D/B-mode controls are hidden when in Doppler modes. You can switch between 2D/B-mode and Doppler mode controls.

To view the 2D/B-mode controls, tap **2D**.

M-mode

M-mode is also known as Motion Mode. It provides a trace of the image displayed over time. A single beam of ultrasound is transmitted, and reflected signals are displayed as dots of varying intensities, which create lines across the screen.

When M-mode is turned on, the screen splits to show B-mode as well as M-mode. You can adjust depth, and gain (similar to B-mode) along with M-mode specific controls like M-line and sweep speed.

	While scanning with Lexsa probe, M-mode is only available in the Lung preset.
---	---

- ★ To start M-mode, tap the M-mode  icon.

M-Line

- ★ To move the M-Line, use your finger to change to m-mode, drag the M-Line to the location you want.

Sweep speed


You can change the sweep speed to isolate individual motions.

- ★ To change the M-mode sweep speed, tap **Speed**, and select 25, 50, 75 or 100 mm/sec.

Color Doppler

Color Doppler is used to visualize the presence, velocity, and direction of blood flow in a wide range of flow states.

When using Kosmos, you can turn Color Doppler on and off without it interfering with the system's color acquisition.

- ★ To turn Color Doppler on and off, tap the Color  icon.

Color box

You can move and resize the color box during imaging. The maximum axial and lateral size of the box may be limited depending on the organ, depth, or other settings.

- To move the color box, drag it to another position.

To resize the color box, move one of the corners to make it either taller or wider.

Scale

Scale changes the pulse repetition frequency that defines the velocity scale with the range shown at the top and bottom of the color map.

- ★ To change the scale, tap **Scale**.

Color Gain

Color Gain amplifies the return of the color signals.

- ★ To adjust the color signals, tap **Color Gain**.

Sensitivity

Three sensitivity range selections are available to optimize for low, medium, and high range.

- ★ To change the sensitivity, tap **Sensitivity**, and select an option.

Wall filter

With the wall filter, the higher the level, the more it blocks the low-frequency flow.

- ★ To change the wall filter, tap **Wall filter**, and set the appropriate low-frequency flow.

Steer


Steer changes the steering angle of the color ROI. There are 5 angles to choose from.

- ★ To select desired angle, tap **Steer**.

Artery


Artery enables Artery/Vein selection. Artery should be selected for arterial flow and Vein should be selected for venous flow.

- ★ For Artery/Vein selection, tap **Artery**.

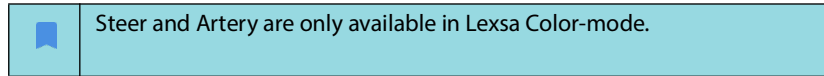

Color map


To change the heart color map:

1. Tap the  icon next to the color map on the right side of the screen.
2. Select the color map you like.
3. To invert the color map, select the check box, and tap **OK** to save the changes.

Pulsed-Wave Doppler

Pulsed-Wave Doppler (PW) mode uses short bursts of ultrasound with a process called range gating to facilitate signal analysis from a small area at a specified depth from the transducer.

To start PW Doppler, tap the **PW mode icon**.

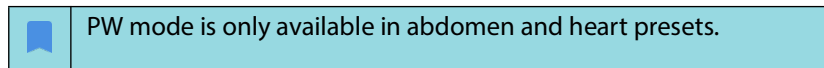

Duplex screen

- ★ Tap the **Update button** for the duplex screen. The frozen B-mode image will be displayed on the top with the live Doppler trace on the bottom.

Gate location and Doppler line

- ★ Adjust the **Gate location** and the **Doppler line** by moving the dotted circle. In the abdomen preset, you can tap the Gate to see and set the angle adjust line. If Color mode is on moving the circle will also move the color box. The

circle and color box can be decoupled by going to **SETTINGS** --> **Imaging preferences**.

Baseline

- ★ Tap and move the **baseline** up and down in the Doppler trace.

Live display

- ★ Tap the **Live display** to toggle between PW live and B live modes. In the B live mode, Doppler trace is frozen.

Wall filter

Wall filter helps to filter out echoes from low frequency signals.

- ★ Tap icon to select the strength of the **filter**: Low, Medium, High.

Invert

- ★ To invert the Doppler spectrum, tap on the invert button.

Scale

Scale changes the velocity scale.

- ★ To change the scale, tap **Scale**.

Doppler Gain

Gain controls the brightness/strength of the Doppler spectrum.

- ★ To adjust Doppler gain, tap **GAIN**.

Audio gain

Audio Gain controls strength of the audio volume.

- ★ To adjust Audio gain, tap **Audio gain**.

Sweep speed

Four sweep speed selections are available.


- ★ To change the sweep speed, tap **Speed** and select 25, 50, 75 or 100 mm/sec.

Save clips and images

- ★ Tap Freeze to review or directly save images and clips. Audio will also be saved in clips.

Tissue Doppler Imaging



Tissue Doppler Imaging (TDI) mode uses Doppler to measure the velocity of myocardial motion throughout the cardiac cycle.

	TDI mode is only available in abdomen and heart presets.
---	--

- ★ To start TDI mode, tap on the **TDI mode icon**.

Continuous-Wave Doppler

Continuous-Wave Doppler (CW) mode uses continuous transmission and reception of ultrasound waves to measure blood velocities.

	When CW is used for a prolonged period, auto freeze comes into effect to manage probe temperature. A 60-second timer appears every time before the auto freeze.
	CW mode is only available in the abdomen preset and heart preset.

- ★ To start CW Doppler, tap the **CW mode icon**.

Duplex screen

- ★ Tap the **Update button** for the duplex screen. The frozen B-mode image will be displayed on the top with the live Doppler trace on the bottom.

Focal point and Doppler line

- ★ Adjust the **focal point** and the **Doppler line** by moving the dotted circle. In the abdomen preset, you can tap the focal point to see and set the angle adjust line. If Color mode is on, moving the circle will also move the color box. The circle and the color box can be decoupled by going to **Settings --> Imaging preferences**.

Baseline

- ★ Tap and move the **baseline** up and down in the Doppler trace.

Live display

- ★ Tap the **Live display** to toggle between CW live and B live modes. In the B live mode, Doppler trace is frozen.

Wall filter

Wall filter helps to filter out echoes from low frequency signals.

- ★ Tap icon to select the strength of the filter: Low, Medium, High.

Invert

- ★ To invert the Doppler spectrum, tap on the invert button.

Scale

Scale changes the velocity scale.

- ★ To change the scale, tap **Scale**.

Doppler Gain

Gain controls the brightness/strength of the Doppler spectrum.

- ★ To adjust Doppler gain, tap **Gain**.

Audio gain

Audio Gain controls strength of the audio volume.

- ★ To adjust Audio gain, tap **Audio gain**.

Sweep speed

Four sweep speed selections are available.

- ★ To change the sweep speed, tap **Speed** and select 25, 50, 75 or 100 mm/sec.

Save clips and images

Tap Freeze to review or directly save images and clips. Audio will also be saved in clips.

Image mode controls

Flipping an image

You can only flip an image when you are scanning the heart.

- ★ To flip the image, double-tap the orientation marker.

Adjusting depth and gain

To adjust depth:

- ★ To increase or decrease the displayed depth, tap **Depth**, and swipe up or down to adjust the depth in one motion.

To adjust gain:

- To adjust gain in Color-mode and B-mode, tap **Gain**, and swipe up or down to adjust the gain in one motion.
- To adjust near and far gain, tap **TGC**, and move the sliders left and right. Notice the gain values automatically update as you adjust the sliders.

Zooming in and out

- While scanning, use two fingers to pinch and expand the image area.
- To return to the default image size, tap the magnifying glass.
- Notice that the zoom factor is shown near the magnifying glass as well as the orange color of the depth scale along the side image area.
- You can freeze while zoomed (and can zoom out and zoom in while frozen).

Freezing an image

- ★ To freeze an image, tap the Freeze  icon.
The **annotation tools** automatically display on the left side of the screen.

Using Kosmos AI-assisted EF Workflow



Calculating Ejection Fraction with Kosmos AI-assisted EF Workflow

Kosmos AI-assisted EF Workflow guides you through the steps of data acquisition followed by an AI-based initial EF calculation which is based on the American Society of Echocardiography (ASE)-recommended modified Simpson's method of disks (Lang 2005, 2015). The initial LV contours are produced with AI trained, expert-annotated LV contours (Ronneberger 2015). You can then review the initial AI results (which include the ED/ES frames along with the corresponding LV contours), and adjust them, as necessary. To calculate EF:


1. From the home screen, tap the AI icon.

Tap to start the AI-assisted EF workflow



	When you tap the Heart AI icon, Kosmos creates a new exam that includes this EF scan.
	Do not rely upon EF calculation as the sole diagnostic criteria. Whenever possible, use EF calculation in conjunction with other clinical information.

2. After you have a good A4C view of the patient, tap **A4C** to acquire a clip.. After you have a good A4C view of the patient, tap A4C to acquire a clip.
3. If you are not satisfied with the recorded clip, tap **Try again** to acquire a new clip, or tap **Accept** to proceed (after four seconds, Kosmos automatically accepts the clip).
4. Tap **SKIP** to see the A4C results, or continue with the A2C acquisition.

	We recommend that you take both A4C and A2C clips for more accurate calculations.
---	---

5. After you acquire images, the algorithm will assess the quality and uncertainty of the clip and users may be presented with the EF error screen. To proceed to your results, the EF error screen requires that you edit the thumbnail or re-acquire the image.
6. After you have a good A2C view of the patient, tap **A2C** to acquire a clip.
7. If you are not satisfied with the recorded clip, tap **Try again** to acquire a new clip, or tap **Accept** to see the A4C/A2C (biplane) results (after four seconds, Kosmos automatically accepts the clip).


Now that the A4C and A2C clips are recorded and accepted, the system selects the ED and ES frames, draws the corresponding LV contours, and calculates the biplane EF using the modified Simpson's method of disks (20 disks are used in the calculation). Kosmos Torso must be the probe used and ECG must be connected to get the CO and HR values.

Reviewing/adjusting the ED/ES frames and LV contours

When reviewing the initial AI calculations for ED/ES frames and LV contours, you can adjust just the frames, the LV contours, or both before saving the results. If you do not make any changes, the AI calculations become the final result.

To adjust the ED/ES frames:


1. From the Results screen, tap **Edit** or one of the thumbnail images. You can also tap **REVIEW** to review previously acquired scans.

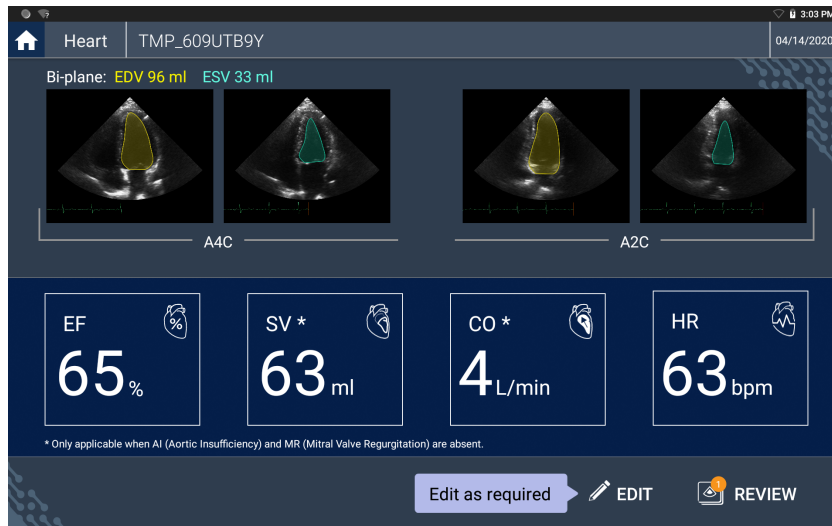

* CO and HR are only available with Kosmos Torso.

2. Depending on which clip you'd like to edit, tap the **A4C clip** or **A2C clip** tab.

- To set a different ED or ES frame, move the orange Seek button to the desired location, and tap **SET ED** or **SET ES**.

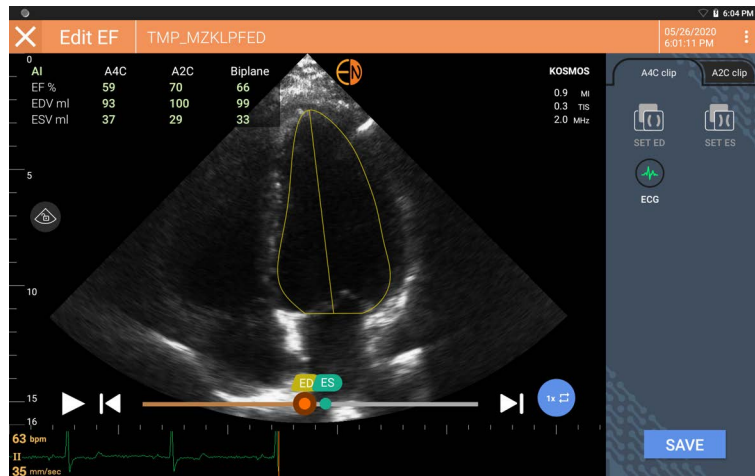


- To return to the original AI calculations, tap the More options \therefore icon and then **Reset**.
- If desired, make changes to the other clip (A4C or A2C), and tap **SAVE**.

To adjust the LV contours:

	If you are wearing gloves while editing the LV contours, make sure they are snug against your fingertips/nails.
	Having gel on your fingers may hinder using the touchscreen effectively. Make sure to wipe the touchscreen regularly.

- From the Results screen, tap one of the four images to go to that image. If you don't specify which image you want, Kosmos defaults to the A4C frame.
- Depending on which clip you'd like to adjust, tap the **A4C clip** or **A2C clip** tab.
- Tap the **A4C clip** or **A2C clip** tab to select an ED or ES frame.
- Tap the LV contour. The LV contour becomes adjustable, and color changes to orange.

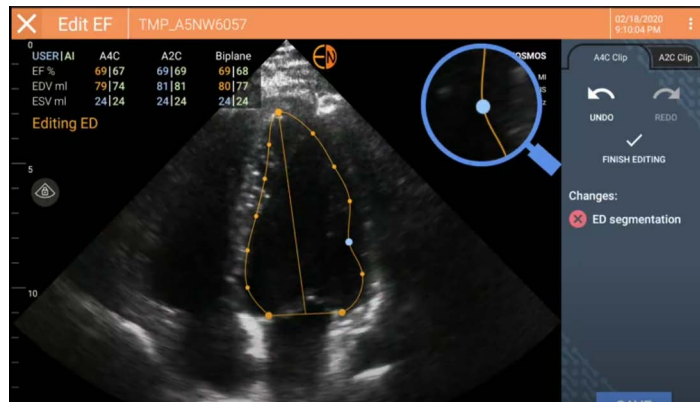

5. Select one or more control points and move them.
 - Notice the calculations are updated as you change the contour.
6. After you are done editing, tap **Finish editing**.
7. If desired, make more changes.
8. Tap **SAVE**.

Recommendations for acquiring optimal A4C and A2C clips for accurate EF calculations

EchoNous recommends the following:

- The patient should be lying on their side in the left lateral position (the left side of the patient is touching the scanning table). For an A4C clip, ensure all four cardiac chambers (left ventricle, left atrium, right ventricle, and right atrium) are captured in the ultrasound image).
- For an A2C clip, ensure both left ventricle and left atrium are captured in the ultrasound image
- Ensure the endocardial border of the LV is clearly visible with the best possible contrast. Use the Gain settings to achieve a clear definition of the LV endocardial border.
- Adjust the depth so that the atria are near the bottom of the ultrasound image yet still visible).

- Avoid truncating the LV.
- Avoid foreshortening the LV.
- For an A4C clip, ensure the intra ventricular septal wall (the wall between the left and right ventricles) is vertical (see the A4C reference image above).
- For an A4C clip, ensure that the orange marker on Kosmos Torso or Kosmos Torso-One is pointed towards the scanning table to avoid acquiring a mirrored view.
- Once you have obtained a proper A4C view, rotate the probe 90 degrees counterclockwise to find the A2C view.
- Ask the patient to hold their breath while recording the clip.
- Make sure to review the results for correctness of ED/ES frames and LV contours and, using the Kosmos editing tool, adjust as needed.

Error conditions and system notifications for Kosmos AI-assisted EF workflow


- If the resulting EF scan (initial and/or with edits) is out of the 0%-100% range, you will not be able to save the EF result in the report or export/archive the scan.

You will first need to edit the ED/ES frames and corresponding LV contours to produce a valid EF. Then you will be able to save the results and export/archive the scan.


- Kosmos will prompt you to edit the results or scan again if any of the following conditions are met:
 - $ESV > 400$ ml
 - $EDV > 500$ ml
 - Difference between A4C and A2C EF is more than 30%

Acquiring images and clips


To acquire an image:

- ★ From the Imaging screen, tap the Save image  icon.

To acquire a clip:

- ★ From the Imaging screen, tap the Save clip  icon.

Completing an exam

1. From the Imaging screen, tap the Exam review  icon.
2. Tap **COMPLETE**.

If you do not tap **COMPLETE** from the Exam review screen, Kosmos automatically completes the exam:

- When you start a new exam
- When you archive the in-progress exam
- After a few minutes
- When you turn off Kosmos Bridge

Kosmos Cardiac Measurements



Do not rely upon Kosmos cardiac measurements as the sole diagnostic criteria. Whenever possible, use Kosmos cardiac measurements in conjunction with other clinical information.


Kosmos Cardiac Calculations package provides the tools to assess cardiac structure and function. Kosmos cardiac measurements are available in B-mode, Doppler and M-mode. While in Exam Review, cardiac calculations and annotation tools can be used to perform cardiac measurements.

Reference Table 5-4 Cardiac Measurements by Mode for a list of cardiac measurements by mode.


While reviewing the Doppler cine, you can:

1. Perform Doppler measurements.
 - VTI - When you tap on VTI, you will have the option to select Auto or Manual VTI trace.

- If you select Auto, tap the signal that you want to trace and the device will trace the signal automatically.
- If you select Manual, you will be prompted to manually trace the signal with your finger.
- Edit the VTI trace by moving the control points.
- Choose a different peak by double tapping it.

 Please note that auto tracing is not available for Mitral Valve VTI in PW and CW tracing. Auto tracing is only available in Annotations or for LVOT VTI (PW) and AV VTI (CW).

- PHT and Delta Velocity - Move the two end points of the calipers to the appropriate location on the Doppler spectrum.
- Velocity and PG - move the cursor to the desired location.
- You can perform 3 PHT, 3 Velocity and 3 VTI measurements per image/clip.
 - Only 3 frames in 2D cine loops can be placed.
 - Can only do 3 VTI measurement at a time.

 You will receive a notification that the measurement is full in the report if you try to place a 4th measurement. You can delete a measurement in the report to make room for a new measurement.



2. Add Annotations:
 - Text
 - Marker
3. Move the baseline
4. Invert the Doppler spectrum
5. View measurements by tapping the Report icon 
6. When viewing the report, the last measurement taken is the default measurement, however, by clicking Last, the device will calculate the average value or provide the maximum value of each measurement.

TABLE 5-3. Cardiac Measurements by Mode

2D Measurements	
PLAX	RVIDd, IVSd, LVIDd, LVPWd, LVIDS, LA diam, LVOTd
Right Heart	RV basal, RV mid, RV length
Mitral Valve	MV Annulus diameter
Aortic Valve	Annulus, Sinus, ST junction, Ascending AO, Vena Contracta, LVOT diameter
IVC	IVC min, IVC max, RAP
Doppler Measurements	
PW	Right Heart: PV AcT (Acceleration time) Mitral Valve: MV VTI (PW), E wave velocity, Deceleration time, A wave velocity Aorta: LVOT VTI (PW) Diastology: E wave velocity (PW), A wave velocity, Deceleration time (PW) Aortic Valve: LVOT VTI (PW)
CW	Right Heart: TR (CW), PAEDP (CW), PR (CW) Mitral Valve: MV VTI (CW), Pressure Half Time (CW) Aortic Valve: AV VTI (CW), Peak AV Velocity, Pressure Half Time (CW) Diastology: TR (CW)
TDI	Right Heart: TV annulus s' Mitral Valve: e' -point (m/s), a' -point (m/s) Diastology: e' -point (m/s), a' -point (m/s)
M-mode Measurements	
M-mode	EPSS, TAPSE, MAPSE, IVC min, IVC max, HR, RAP
PLAX	RVIDd, IVSd, LVIDd, LVPWd, LVIDS, LA diam, AO

Kosmos AI FAST

	Do not rely solely on the AI FAST tool for diagnostic purposes. Kosmos AI FAST assists users by providing a quick orientation to the anatomy of the abdomen. Users should exercise their judgment to ensure annotations are correct.
---	--

Using Kosmos AI for FAST Exam

Kosmos AI FAST provides automated anatomical labeling and view identification for the FAST exam in real time. The labels that appear while scanning are only there while you are scanning; after you save the image or clip, the labels will no longer be there.


Reference Table 5-5 for a list of anatomical structures in each FAST Exam imaging views.

FAST View	Anatomical Structures
RUQ	Liver, right kidney, diaphragm, gallbladder, IVC Potential fluid space: hepatorenal space, pleural space
LUQ	Spleen, left kidney, diaphragm Potential fluid space: splenorenal space, pleural space
SUP	Bladder Potential fluid space: rectovesical space (male), rectouterine space (female)
SUB	Heart, diaphragm, liver Potential fluid space: pericardium
AS	Liver, transverse aorta, transverse IVC
IVC	Liver, sagittal IVC
Aorta	Liver, sagittal aorta
A4C A2C PLAX	Heart Potential fluid space: pericardium

PSAX	Heart
SUB2	Liver, heart, IVC, aorta Potential fluid space: pericardium

To enable Kosmos AI FAST:

- ★ In Abdominal preset, tap **AI**.

	When scanning with Torso/Torso-One probe, Kosmos AI FAST feature is only available in the Abdomen preset.
---	---

Us2.ai & Kosmos

Integration of Us2.ai's Us2.v1 software with Kosmos provides automatic clinical workflow that recognizes and analyzes 2D and Doppler modality images for automated cardiac measurements and the diagnosis, prediction and prognosis of heart disease.

Getting Started




Licenses to use Us2.ai's Us2.v1 software are available for institutional purchase.

1. To purchase a license to use Us2.v1 with Kosmos, please contact an EchoNous representative.
2. Once licensed, users will be provided login credentials to connect their Bridge to the Us2.ai Cloud.

To use Kosmos and Us2.ai's Us2.v1 software on Kosmos Bridge:

1. Use credentials to log in to Us2.ai on your Bridge.
2. Plug in probe, select preset and begin scanning.
 - Plug Torso/Torso-One probes in the port underneath the Bridge handle.
 - Plug Lexsa into the USB-C port on the side of the Bridge.
3. After completing an exam, under Exam Review, tap the US2 icon to send images and clips to the Us2.ai Cloud.
4. Tap the Report icon to view the cardiac measurements.

Us2.ai's Us2.v1 software is registered as an independent medical device and may not be available in your region. For further information regarding Us2.ai, please contact your EchoNous representative.

	An electronic version of the Kosmos & Us2.ai's Quick Start Guide is available on the EchoNous website: echonous.com/product/resources .
	Us2.ai is a third-party provided product. EchoNous disclaims liability, express/implied warranties, for Us2.ai services. Customers acknowledge EchoNous has made no representation as to fitness of third-party services for intended purpose. Us2.ai services are governed by Us2.ai Terms of Use, warranties and disclaimers. For more information, please reference http://us2.ai/terms-conditions/ .
	All Us2.ai owned software installed is governed by the terms and conditions of the Us2.ai End User License Agreement ("Us2.ai EULA") and incorporated agreements. Any questions or claims regarding Us2.ai Software should be brought pursuant to the Us2.ai EULA.

- End of section -


Intentionally left blank.

Reviewing an Exam

Once you have completed an exam, you cannot add any images to it; however, before archiving the exam, you can add, edit, and delete any annotations you have saved.

Once the archive process begins, you will not be able to make edits to the exam.

Starting an exam review

- To start a review during an exam, tap the Exam review  icon.
- To start a review for a completed exam, do one of the following:
 - From the Home screen, tap **EXAMS**, then tap the exam you would like to review.
 - From the list of patients, find the patient, then tap the exam you would like to review.

Annotating images and clips

You can add annotations during the exam when the image is frozen or after you have completed the exam. All annotations are saved as overlays on the image or clip.






Once you have archived an image or clip, you cannot annotate it.



Navigating to the Edit Image screen

To navigate to the Edit Image or Edit Clips screen:


While scanning a patient:

1. Tap the Freeze  icon.
2. Add your annotations.
3. Tap the Save image  or Save clip  icon.

After scanning a patient:


1. Tap the Exam review  icon.
2. Tap the image/clip you want to annotate.
3. Tap the Edit  icon.

From the Home screen:

1. Tap **Exam**.
2. Tap the exam row that you want to edit.
3. Tap the clip you want to annotate.
4. Tap the Edit  icon.

From the Patient screen:


1. Tap a patient from the list.

2. Tap the exam.
3. Tap the image/clip you want to annotate.
4. Tap the Edit  icon.

Annotation tools

Annotations allow the user to mark and label images while frozen and in exam review. It also allows the user to do generic measurements that will not be saved in the report. Annotations can be added to individual images and clips.

When you add an annotation (text, measurements, arrow, area) to a clip or a cine, they persist through all frames.

You can also hide the overlay of the annotations you make by tapping the Hide overlay  icon on saved images and clips.

Measuring with the caliper tool

You can add up to two calipers per image/clip.

When a caliper is not selected and you start dragging one of the two end points of the caliper, the caliper will become selected and will resize based on where you are dragging it.

To place a measurement:

1. From the Edit image or Edit clip screen, tap **DISTANCE**, and a caliper appears in the center of the image or clip.
2. Tap to select the caliper.



Notice that the distance of the caliper displays in the legend on the upper left side of the screen. If you have multiple calipers, they display in different colors.

3. To resize the caliper, tap and drag one of its end points.
4. To move the caliper, tap anywhere on the caliper except the two end points.
5. To clear the caliper, tap an empty area outside it.

Zooming in and out

Use two fingers to pinch and expand the image area. To return to “normal”, tap the magnifying glass. Also, zoom factor is shown near magnifying glass as well as orange color of depth scale along the side. You are able to freeze the image while zoomed (and can zoom in/out in frozen state).

Deleting annotations

★ To delete one annotation, tap the annotation to select it, then tap **DELETE**.


To delete all the annotations you have made, tap **CLEAR ALL**.

Managing images and clips


Filtering images and clips

When you review an exam, all the images and clips, regardless of the scan type (lung, heart, abdomen) are visible in the thumbnail list.

You can filter images and clips in the following ways:


- Drag and pull the thumbnail list down to reveal the filter options.
- Tap the Filter icon on top of the thumbnail list to reveal the filter options.
- Tap the More options  icon in the title bar, and tap **Filter images and clips**.
When the filter options are visible, a blue check icon will be shown next to **Filter images and clips**.


When you select a filter, only the tagged images/clips are visible in the thumbnail list. You can tag images/clips by tapping the star icon under each image/clip in the thumbnail list so the star turns yellow.

To dismiss the filters you have selected, tap the More options  icon, then tap the **Filter images and clips** again to remove the filters.

Selecting images and clips


To select images and clips:

1. Tap the More options  icon, and tap **Select images and clips**.
2. Select the images and clips you want. A gray check will appear in the top right corner of the thumbnail.
3. Optionally, tap the check on the thumbnail; it turns red, and a numbered circle displays to indicate how many images and clips you have selected. To clear the red check, tap it again.


To clear the selections, tap the More options  icon, and tap **Select images/clips**.

Deleting images and clips

To delete selected images and clips:

1. Tap the More options  icon, and tap **Select images/clips**.
2. Select the images and clips you want to delete.
3. Tap **DELETE** and, when prompted, tap **OK**.

Reviewing and editing a report

 Reports are not yet encapsulated in the DICOM file; you can only see images and clips at this review step.

The exam report lets you review patient and exam information, text notes, audio notes, pictures that were taken, images, and clips in the exam report.

Opening a report

To open a report, tap **REPORT**.

Editing a report


Once you've opened the report, each section is expanded for your review. You can collapse each section by tapping the arrow button. Just tap the arrow button to expand the section again.

You can edit each section of the report with the exception of the patient information. This is read-only and cannot be changed.

Editing exam information

The exam information section displays the exam related information that was entered before the scan.

To edit the exam information:

1. Tap the Edit  icon.
2. Make any necessary updates to the section.

Adding a text note

You can add text notes that will display under each scan.

To add a text note:

1. Tap the Add text note icon. A text box, date and time label appear under the last text note.
2. Using the keyboard, type the note.
3. Tap **DONE**.

Editing a text note

To edit a text note:

1. Tap an existing text note. A text box containing the existing note and the keyboard displays.
2. Using the keyboard, edit the text note.
3. Tap **DONE**.

Deleting a text note

To delete a text note:

1. Long press an existing text note. A delete button displays.
2. Tap **DELETE** and, when prompted, tap **OK**.

Exporting images and clips to a USB drive

When exporting an images and clips, use a micro USB or adapter.

You can export images and clips from one exam or multiple exams.




To protect patient data, take appropriate precautions when exporting patient data to a USB drive.





To export images and clips from one exam to a USB drive:

1. From the Home screen, tap **EXAMS**.
2. Tap a row to select an exam.
3. Tap the bookmark icon under each of the thumbnails you would like to export. (This is an optional step and only useful if you would like to export some but not all images and clips.)
4. Connect the USB drive using the USB-c adapter.
5. Tap **EXPORT**. A dialog box appears.
6. Select the file type and whether you want all images and clips exported or only the tagged images and clips.
7. Tap **OK** to start exporting to USB drive.

To export images and clips from multiple exams to a USB drive:

1. From the Home screen, tap **EXAMS**.
2. Tap the circles next to each exam you would like to export.
3. Connect the USB drive using the USB-c adapter.
4. Tap the Export  icon on the top of the screen. A dialog box appears.
5. Select the file type and whether you want all images and clips exported or only the tagged images and clips.
6. Tap **OK** to start exporting to USB drive.

The following table is a legend for the exporting icons.

	Exam is waiting to be exported.
	Export is in progress
	Export is complete.
	Export failed.

Completing an exam review

To complete an exam:

1. Tap **COMPLETE**.
2. When prompted, click **OK**.





Archiving an exam to a PACS server

After completing an exam, you can archive it to a PACS server. Once an exam is archived, you cannot edit it.

For more information about setting up a PACS server, see [Managing PACS archives](#).

For each EF scan, multiple images/clips are archived and exported.


The following table is a legend for the archiving icons.

	Exam is waiting to be archived.
	Archive is in progress.
	Archive is complete.
	Archive failed.

You can archive an exam either from the Exam list or the Exam review screens.

To archive an exam from the Exam list screen:

1. From the Exam List screen, tap to select the completed exam(s) you want to archive.



Tap the Archive  icon. The complete exam is archived according to the default archive options. For more information, see [Managing PACS archives](#).

To archive an exam from the Exam review screen:


1. From the Exam review screen, tap **ARCHIVE**.
2. From the Archive exam to PACS server screen, select which images and clips you want to archive and if you would like to include a report.
3. Click **OK** and, when prompted, click **OK** again.

Deleting an exam

To delete an exam from the Exam list:

1. Tap the left icon next to the exam you would like to delete. The icon turns into a check mark .
2. Tap the Trash  icon.
3. When prompted, tap **OK**.



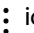



To delete an exam while reviewing it:

1. Tap the More options  icon.
2. Tap **Delete the exam**.
3. When prompted, click **OK**.



Intentionally left blank.

Kosmos Probe sheaths

Where fluid contamination is possible, cover the probe being used (Kosmos Torso, Kosmos Torso-One, or Kosmos Lexsa) with an appropriate sterile sheath from CIVCO, which will promote sepsis and minimize cleaning.

	Be aware that some patients have a latex allergy. Some commercially available Kosmos probe covers contain latex.
	To prevent cross-contamination, use sterile transducer sheaths and sterile coupling gel for clinical applications contacting compromised skin.
	Some sheaths contain natural rubber latex and talc, which can cause allergic reactions in some individuals.
	Use market-cleared sheaths for clinical applications when a Kosmos probe is likely to be splashed or splattered with blood or other bodily fluids.
	Use market-cleared, sterile sheaths and sterile coupling gel to prevent cross-contamination. Do not apply the sheath and coupling gel until you are ready to perform the procedure. After use, remove and discard the single-use sheath, and clean and disinfect the Kosmos probe using an EchoNous-recommended high-level disinfectant.
	After inserting the Kosmos probe into the sheath, inspect the sheath for holes and tears.


Ultrasound transmission gels

	Some ultrasound gels may cause an allergic reaction in some individuals.
	To prevent cross-contamination, use single-use gel packs.

EchoNous recommends the use of:

- Aqua sonic 100 Ultrasound Gel, Parker
- Aqua sonic Clear Ultrasound Gel, Parker
- SCAN Ultrasound Gel, Parker

Kosmos Probe storage

	To prevent cross-contamination or unprotected exposure of personnel to biological material, containers used to transport contaminated Kosmos probes should carry an ISO biohazard label.
---	--

The Kosmos battery can only be replaced at an EchoNous facility; however, for shipping/storage, the battery is Li-Ion 3.6V, 6.4 Ah.

Daily storage

Kosmos is intended to be used and stored in normal ambient conditions inside a medical facility. In addition, the packaging provided with the device may be used for long-term storage.

Storage for transport

Kosmos is intended to be hand held for easy transport. Users may use the packaging supplied with the device for transport. Consult your EchoNous sales representative for information on approved bags and other accessories.

Transducer Element Check

To check the integrity of the transducer elements, an automatic test is run every time a Kosmos probe is connected.. The test reports to the user whether all transducer elements are functioning properly (successful test), or whether failures were detected.









The same test runs automatically when Kosmos Bridge boots up with a Kosmos probe connected.



- End of Section -

Intentionally left blank.



Cleaning and disinfecting

General cautions

	The provided cleaning instructions are based on requirements mandated by the U.S. Food and Drug Administration. Failure to follow these instructions may result in cross contamination and patient infection.
	Cleaning and disinfection instructions must be followed when using a transducer cover or sheath.
	Some reprocessing chemicals may cause an allergic reaction in some individuals.
	Ensure that cleaning and disinfecting solutions and wipes are not expired.
	Do not allow cleaning solution or disinfectant into the Kosmos Bridge or Kosmos probe connectors.
	Wear the appropriate personal protective equipment (PPE) recommended by the chemical manufacturer, such as protective eye wear and gloves.
	Do not skip any steps or abbreviate the cleaning and disinfecting process in any way.
	Do not spray cleaners or disinfectants directly on Kosmos Bridge surfaces or on Kosmos Bridge and Kosmos probe connectors. Doing so may cause solution to leak into Kosmos, damaging it and voiding the warranty.

	Do not attempt to clean or disinfect Kosmos Bridge, Kosmos probe or the Kosmos probe cable using a method that is not included here or chemical not listed in this guide. Doing so can damage Kosmos and void the warranty.
	Do not pull the cable of the Kosmos probe while holding or disinfecting the device. Pulling on the cable may cause damage to the probe.


Kosmos Bridge

	Kosmos Bridge is not sterile when shipped; do not attempt to sterilize it.
	To avoid electrical shock, before cleaning, turn off Kosmos Bridge and disconnect it from the power supply.

Cleaning

Avoid spraying the cleaning and disinfection solutions directly onto Kosmos Bridge. Instead spray onto a non-abrasive cloth and then gently wipe. Ensure that all excess solution is wiped off and not left on the surface after cleaning. The following cleaning and disinfection method must be followed for Kosmos Bridge.

1. After each use, disconnect the USB cable from the Kosmos probe.
2. Remove any accessories, such as the headset or power supply.
3. Using a wipe from an approved presaturated disinfectant wipe, carefully wipe the screen and all other areas of Kosmos Bridge. Choose an EchoNous-approved wipe from the list in **Presaturated wipes**.
4. If necessary, clean Kosmos Bridge with additional wipes to remove all visible contaminants.

	After disinfection, examine the display for cracks, and if damage exists, discontinue use of the system and contact EchoNous Customer Support.
---	--

Presaturated wipes




Product	Company	Active Ingredients	Contact Condition
Sani-Cloth Plus	PDI Inc.	n-Alkyl (68% C12, 32% C14) dimethyl ethylbenzyl ammonium chlorides. 0.125% n-Alkyl (60% C14, 30% C16, 5% C12, 5% C18) dimethyl benzyl ammonium chlorides. 0.125%	5 minutes wet contact time for disinfection
CaviWipes (KavoWipes)	Metrex	Diisobutylphenoxyethoxyethyl dimethylbenzyl ammonium chloride (0.28%), Isopropanol (17.2%)	5 minutes wet contact time for disinfection




Kosmos Probes

Cleaning

The following cleaning instructions must be followed for Kosmos Torso, Kosmos Torso-One and Kosmos Lexsa. Kosmos probes must be cleaned after each use. Cleaning Kosmos probes is an essential step before effective disinfection.

Before cleaning Kosmos Torso, Kosmos Torso-One and Kosmos Lexsa, read the following warnings and cautions.

	Always disconnect the USB cable from the Kosmos probe before cleaning and disinfecting.
	After cleaning, you must disinfect Kosmos probes by following the appropriate instructions.
	Always wear protective eye wear and gloves when cleaning and disinfecting any equipment.

	Use only EchoNous-recommended wipes. Using a non-recommended wipe can damage the Kosmos probe and void the warranty.
	When cleaning and disinfecting Kosmos probes, do not allow any fluid to enter electrical connections or metal portions of the USB connector.
	The use of a cover or sheath does not preclude proper cleaning and disinfecting of a Kosmos probe. When choosing a cleaning and disinfecting method, treat Kosmos probes as if a cover was not used in the procedure.






To clean Probes:

1. After each use, disconnect the USB cable from the Kosmos probe.
2. Remove any accessories attached to, or covering the Kosmos probe, such as a sheath.
3. At point of use, wipe Kosmos probe with an approved presaturated wipe.
4. Prior to disinfecting the Kosmos probe, remove all ultrasound gel from the Kosmos probe face by using an approved presaturated disinfectant wipe. Choose an EchoNous-approved wipe from the list in **Presaturated wipes**.
5. Using a new wipe, remove any particulate matter, gel, or fluids that remain on the Kosmos probe using a new presaturated wipe from **Presaturated wipes**.
6. If necessary, clean the Kosmos probe with additional wipes to remove all visible contaminants.
7. Before continuing with disinfection, ensure the Kosmos probe is visibly dry.

Disinfecting (intermediate-level)


Use the following steps to disinfect a Kosmos probe whenever it has not come into contact with non-intact skin, or intact mucous membranes (non-critical

use). Before performing the following steps, read the following warnings and cautions.

	For low level and intermediate level disinfection, EchoNous validated its disinfection with intermediate level disinfection.
	Always disconnect the USB cable from Kosmos probes before cleaning and disinfecting.
	Always use protective eye wear and gloves when disinfecting any equipment.
	Before disinfecting, clean Kosmos probes by following the appropriate instructions to remove all gels, fluids, and particulates that may interfere with the disinfection process.
	Use only EchoNous-recommended disinfectants. Using a non-recommended disinfecting wipe can damage the Kosmos probe and void the warranty.

To disinfect Kosmos Probes (intermediate level):







1. After cleaning, choose an intermediate-level disinfectant from **Presaturated wipes**, and observe the recommended minimum wet contact time.
2. With a new wipe, clean the cable and the Kosmos probe, starting from the exposed cable, wiping toward the Kosmos probe head to avoid cross-contamination.
3. Observe the required wet contact time. Monitor the Kosmos probe for wet appearance. Use at least three wipes to ensure effective disinfection.
4. Before reusing the Kosmos probe, ensure the Kosmos probe is visibly dry.

	Check the Kosmos probe for damage, such as cracks, splitting, or sharp edges. If damage is evident, discontinue using the Kosmos probe, and contact your EchoNous representative.
---	---

Disinfecting (high-level)

Use the following steps to high-level disinfect the Kosmos probe whenever it has come into contact with intact mucous membranes or non-intact skin (semi-critical use). High-level disinfection of Kosmos probes typically uses an immersion method with high-level disinfectants or chemical sterilant.


Before performing the following steps, read the following warnings and cautions.

	Always disconnect Kosmos probes from AC mains during cleaning and disinfection.
	Before disinfection, clean the Kosmos probe by following the appropriate cleaning instructions in Cleaning to remove all gels, fluids, and particulates that may interfere with the disinfection process.
	Always use protective eye wear and gloves when disinfecting any equipment.
	When disinfecting Kosmos probes, do not allow any fluid to enter electrical connections or metal portions of the USB or Kosmos ECG patient cable connector.
	Do not attempt to disinfect Kosmos probes using a method that is not included in these instructions. This can damage the Kosmos probe and void the warranty.
	Use only EchoNous-recommended disinfectants. Using a non-recommended disinfecting solution or incorrect solution strength can damage the Kosmos probe and void the warranty.
	If the Kosmos probe has come into contact with intact mucous membranes or non-intact skin (semi-critical use), use the high-level cleaning and disinfection procedure.

To disinfect Kosmos probes (high level):

1. After cleaning, choose a high-level disinfectant that is compatible with Kosmos probes. For a list of compatible disinfectants, see **Disinfectant Solutions for Kosmos Probe Immersion**.

2. Test the solution strength by using a Cidex OPA test strip. Ensure that the solution is not older than 14 days (in an open container) or 75 days (from a just opened storage container).
3. If a pre-mixed solution is used, be sure to observe the solution expiration date.
4. Immerse Kosmos probe into the disinfectant as shown below. Kosmos probes may be immersed only up to the immersion point shown. No other part of the Kosmos probe, such as cable, strain relief, or connectors should be soaked or immersed in fluids.

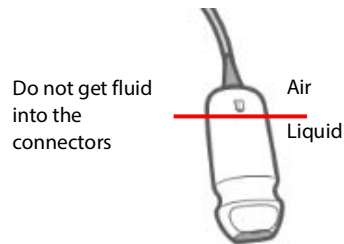




5. Refer to **Disinfectant Solutions for Kosmos Probe Immersion** for duration of immersion and contact temperature.
6. Do not immerse Kosmos probe longer than the minimum time needed for semi-critical level of disinfection.
7. Rinse Kosmos probe for at least one minute in clean water up to the point of immersion to remove chemical residue. Do not soak or immerse any other part of the Kosmos probe, such as the cable, strain relief, or connector.
8. Repeat, rinsing three times to ensure proper rinsing.
9. Air dry or use a soft sterile cloth to dry the Kosmos probe until visibly dry.
10. Wipe the strain relief and first 18 inches (45 cm) of the Kosmos probe cable with an approved wipe from the list in **Presaturated wipes**.
11. Examine the Kosmos probe for damage such as cracks, splitting, or sharp edges. If damage is evident, discontinue using the Kosmos probe, and contact your EchoNous representative.

Disinfectant Solutions for Kosmos Probe Immersion

Product	Company	Active Ingredients	Contact Condition
Cidex OPA Solution	Advanced Sterilization Product	Products 0.55% ortho phthaldehyde	12 minutes at 20°C

- Check the expiration date on the bottle to ensure the disinfectant has not expired. Mix or check that the disinfection chemicals have the concentration recommended by the manufacturer (for example, a chemical strip test).
- Check that the temperature of the disinfectant is within the manufacturer's recommended limits.

Guidelines for AR (Automated Reprocessors)

	Always disconnect the cable from the Kosmos probe before cleaning and disinfecting.
	Ensure cable insulation is intact before and after cleaning.
	The EMC suppressor on probes should be inside the trophon2 chamber below the cable clamp during disinfection.

All Kosmos probes are compatible with Nanosonic™ Trophon2 System. Refer to Trophon®2 User Guide for detailed instructions related to disinfection of ultrasound probes.






For questions related to compatibility with other AR systems, contact EchoNous support.

Kosmos ECG patient cable

Cleaning

The following cleaning instructions must be followed for the Kosmos ECG patient cable. The cable must be cleaned after each use. Cleaning the cable is an essential step before effective disinfection.


Before cleaning the Kosmos ECG patient cable, read the following warnings and cautions.

	Always disconnect the cable from the Kosmos probe before cleaning and disinfecting.
	After cleaning, you must disinfect the cable by following the appropriate instructions.
	Always wear protective eye wear and gloves when cleaning and disinfecting any equipment.
	Ensure cable insulation is intact before and after cleaning.
	Use only EchoNous-recommended wipes and solution. Using a non-recommended wipe can damage the cable.

To clean the Kosmos ECG patient cable:

1. After each use, disconnect the cable from Kosmos Torso.
2. Remove any accessories attached to, or covering, the cable, such as electrode pads.
3. At point of use, wipe the cables with an approved presaturated wipe from the list in **Presaturated wipes** to ensure effective cleaning.
4. Immerse the ECG clips and leadwires in a cleaning solution from the list in **Cleaning Detergent Solution for Kosmos ECG Patient Cable**, and soak for

at least 10 minutes. Refer to **Cleaning Detergent Solution for Kosmos ECG Patient Cable** for the solution concentration and contact time.

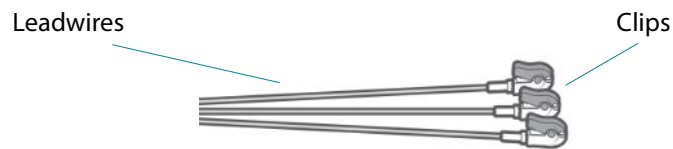






5. Place the cable with the solution in a ultrasonicator for at least 10 minutes.
6. Post sonicating, using a standard cleaning brush, vigorously brush all surfaces of the ECG clips while immersed in the Enzol solution until visibly clean.
7. Actuate any movable parts while immersed. In addition, flush crevices using a slip tip syringe filled with prepared cleaning detergent.
8. Remove the ECG clips from the Enzol solution, and run them under running water for 1 minute. Ensure no gel or any particulate matter is visible after this cleaning step.
9. Before continuing with disinfection, ensure the Kosmos ECG patient cable is visibly dry.

Cleaning Detergent Solution for Kosmos ECG Patient Cable

Product	Company	Active Ingredients	Contact Condition
Enzol	Advanced Sterilization Products	Borax decahydrate ≥ 5 - < 10 Subtilisin ≥ 1 - < 5	2 oz. per gallon solution 20 minutes immersion

Disinfecting the Kosmos ECG patient cable

Use the following steps to disinfect the Kosmos ECG patient cable. Before performing the following steps, read the following warnings and cautions.

	Always disconnect the USB cable from the Kosmos probe before cleaning and disinfecting.
	Always use protective eye wear and gloves when disinfecting any equipment.
	Before disinfecting, clean the Kosmos ECG patient cable by following the appropriate instructions to remove all gels, fluids, and particulates that may interfere with the disinfection process.
	Ensure cable insulation is intact before and after disinfection.
	Use only EchoNous-recommended disinfectants. Using a non-recommended disinfecting wipe can damage the Kosmos ECG patient cable.

To disinfect the Kosmos ECG patient cable:

1. After cleaning, choose an low-level disinfectant from the list in **Presaturated wipes** and follow the instructions on the disinfectant label for the minimum wet contact time.
2. With a new wipe, disinfect the Kosmos ECG patient cable, starting from the connector end to the clips.



3. Observe the required wet contact time. Monitor the Kosmos ECG patient cable for wet appearance.
4. Use at least three wipes to ensure effective disinfection.
5. Examine the cable for damage, such as insulation wearing or discoloration. If damage is evident, discontinue using the Kosmos ECG patient cable.
6. Before reusing the cable, ensure the cable is visibly dry.

Binaural headset

The following cleaning and disinfection method must be followed for Binaural Headset:

1. Disconnect the headset from Kosmos Bridge.
2. Using a wipe from an approved presaturated disinfectant wipe, carefully wipe all the areas of the headset. Choose an EchoNous-approved wipe from **Presaturated wipes**.
3. If necessary, clean the headset with additional wipes to remove all visible contaminants.

Recycling and disposal

	Do not incinerate or discard Kosmos in general waste at end of life. The lithium battery is a potential environmental and fire safety hazard.
	The lithium ion battery inside Kosmos Bridge may explode if exposed to very high temperatures. Do not destroy this unit by incinerating or burning. Return the unit to EchoNous or your local representative for disposal.

Kosmos Bridge contains lithium-polymer batteries, and the system should be disposed of in an environmentally responsible manner in compliance with federal and local regulations. EchoNous recommends taking Kosmos Bridge, and Kosmos probes to a recycling center which specializes in the recycling and disposal of electronic equipment.

In cases where Kosmos Bridge and/or a Kosmos probe has been exposed to biologically hazardous material, EchoNous recommends using biohazard

containers and in compliance with federal and local regulations. Kosmos Bridge and Kosmos probes should be taken to a waste center which specializes in the disposal of biohazard waste.

Troubleshooting

Preventive inspection, maintenance, and calibration

- Kosmos does not require any preventative maintenance or calibration.
- Kosmos does not contain any service able parts.
- The Kosmos battery is not replaceable.



If Kosmos is not functioning as designed and intended, contact EchoNous customer support.

Kosmos Bridge handle controls

- If you do not see the handle control checkbox on the Home screen, contact **EchoNous customer support** to get a newer version of the software/firmware.
- If you do see the handle control checkbox on the Home screen but are not able to activate the handle controls, it is probably a firmware issue. Contact **EchoNous customer support** to get new hardware.
- If you have the updated software and firmware, but still have intermittent issues where the handle controls do not work (or one or more buttons stop responding), try one or more of the following:
 - Check to see if the **Accept** checkbox is selected on the Home screen.
 - Disable the handle controls, and enable them again
 - Restart Kosmos Bridge, and enable the handle controls.
 - Try using the handle controls without gloves.
 - Try using the handle controls with moisturized hands.
 - Lightly tap the handle continuously for five to six seconds.
 - Make sure you are tapping the handle control buttons and not doing a long press.

Intentionally left blank.

Electrical safety

References

IEC 60601-2-37: 2015 Medical electrical equipment – Part 2-37: *Particular requirements for the basic safety and essential performance of ultrasonic medical diagnostic and monitoring equipment*

ANSI AAMI ES 60601-1: 2012 Medical electrical equipment. Part 1: *General requirements for basic safety and essential performance* – IEC 60601-1:2012, Edition 3.1

IEC 60601-1-2:2021 Medical electrical equipment – Parts 1-2: *General requirements for basic safety and essential performance - Collateral Standard: Electromagnetic disturbances - Requirements and tests*

IEC 62304:2015 Medical device software - *Software life-cycle processes*

ISO 14971:2021 Medical devices - *Application of risk management to medical devices*

10993-1:2020 Biological evaluation of medical devices - *Part 1: Evaluation and testing within a risk management process*



ANSI AAMI EC53:2013 *ECG Trunk Cables And Patient Leadwires*





Lang, Roberto M., et al. *Recommendations for chamber quantification: a report from the American Society of Echocardiography's Guidelines and Standards Committee and the Chamber Quantification Writing Group, developed in conjunction with the European Association of Echocardiography, a branch of the European Society of Cardiology*. *Journal of the American Society of Echocardiography* 18.12 (2005): 1440-1463.

Lang, Roberto M., et al. *Recommendations for cardiac chamber quantification by echocardiography in adults: an update from the American Society of Echocardiography and the European Association of Cardiovascular Imaging*. *European Heart Journal-Cardiovascular Imaging* 16.3 (2015): 233-271.




Ronneberger, Olaf, Philipp Fischer, and Thomas Brox. *U-net: Convolutional networks for biomedical image segmentation*. *International Conference on Medical image computing and computer-assisted intervention*. Springer, Cham, 2015.





Labeling symbols

Symbol	EchoNous Description	SDO Title Reference Number Standard
	Indicates device manufacturer. Includes name and address of the manufacturer	Manufacturer Ref. No. 5.1.1 ISO 15223-1 Medical devices - Symbols to be used with medical device labels, labeling and information to be supplied - Part 1: General requirements
U.S. FCC ID: 2AU8B-ECHKMOS Model P005247 	Tested to comply with FCC standards	None


	<p>Class II equipment</p>	<p>Class II equipment Ref. No. D.1-9 IEC 60601-1 Medical electrical equipment - Part 1: General requirements for basic safety and essential performance</p>
	<p>Safety cautions are identified with this mark on the device.</p>	<p>Caution Ref. No. D.1-10 IEC 60601-1 Medical electrical equipment - Part 1: General requirements for basic safety and essential performance</p>
	<p>Consult instructions for use</p>	<p>Operating instructions Ref. No. D.1-11 IEC 60601-1 Medical electrical equipment - Part 1: General requirements for basic safety and essential performance</p>
	<p>Do not dispose of this product in normal trash or landfill; refer to local regulations for disposal</p>	<p>Separate collection Annex IX Waste Electrical and Electronic Equipment (WEEE) Directive 2012/19/EU of the European Parliament</p>



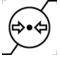

IPX7	Kosmos Torso, Kosmos Torso-One and Kosmos Lexsa are protected against temporary immersion in water.	IP Code for degree of protection IEC 60529 Degrees of protection provided by enclosures (IP Code)
IP22	Kosmos Bridge	IP Code for degree of protection IEC 60529 Degrees of protection provided by enclosures (IP Code)
REF	Part or model number	Catalog number Ref. No. 5.1.6 ISO 15223-1 Medical devices - Symbols to be used with medical device labels, labeling and information to be supplied - Part 1: General requirements
SN	Serial number	Serial number Ref. No. 5.1.7 ISO 15223-1 Medical devices - Symbols to be used with medical device labels, labeling and information to be supplied - Part 1: General requirements

	<p>Date of manufacture</p>	<p>Date of manufacture Ref. No. 5.1.3 ISO 15223-1 Medical devices - Symbols to be used with medical device labels, labeling and information to be supplied - Part 1: General requirements</p>
	<p>Acceptable temperature range XX is generic placeholder for specified temperatures</p>	<p>Temperature limit Ref. No. 5.3.7 ISO 15223-1 Medical devices - Symbols to be used with medical device labels, labeling and information to be supplied - Part 1: General requirements</p>
	<p>Acceptable humidity range XX is generic placeholder for specified percentages</p>	<p>Humidity limitation Ref. No. 5.3.8 ISO 15223-1 Medical devices - Symbols to be used with medical device labels, labeling and information to be supplied - Part 1: General requirements</p>


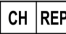
	<p>Acceptable atmospheric pressure</p> <p>range XX is generic placeholder for specified kPa</p>	<p>Atmospheric pressure limitation</p> <p>Ref. No. 5.3.9</p> <p>ISO 15223-1</p> <p>Medical devices - Symbols to be used with medical device labels, labeling and information to be supplied - Part 1: General requirements</p>
	<p>Stack box this way up</p>	<p>This way up</p> <p>Ref. No. 13</p> <p>ISO 780</p> <p>Packaging - Distribution packaging - Graphical symbols for handling and storage of packages</p>
	<p>Indicates direct current</p>	<p>Direct current</p> <p>Ref. No. D.1-4</p> <p>IEC 60601-1</p> <p>Medical electrical equipment - Part 1: General requirements for basic safety and essential performance</p>
	<p>Indicates alternating current</p>	<p>Alternating current</p> <p>Ref. No. D.1-1</p> <p>IEC 60601-1</p> <p>Medical electrical equipment - Part 1: General requirements for basic safety and essential performance</p>

Labeling symbols

<p>R-NZ</p>	<p>R-NZ Compliance Mark. AS/NZS 4268:2017, Radiocommunications Regulations (Radio Standards) Notice 2016.</p>	<p>None</p>
	<p>Regulatory Compliance Mark. AS/NZS 4268:2017, Radiocommunications (Short Range Devices) Standard 2014, Compilation No.2, December 2018. Radiocommunications (Electromagnetic Radiation - Human Exposure) Standard 2014, Compilation No. 1, November 2019.</p>	<p>None</p>
<p>LOT</p>	<p>Batch code</p>	<p>Batch code Ref. No. 5.1.5 ISO 15223-1 Medical devices - Symbols to be used with medical device labels, labeling and information to be supplied - Part 1: General requirements</p>

	<p>UL Classified.</p> <p>Medical - General medical equipment as to electrical shock, fire and mechanical hazards only in accordance with ANSI/AAMI ES 60601-1 (2005) + AMD (2012) / CAN/CSA-C22.2 No. 6060-1 (2008) + (2014).</p> <p>E509516</p>	<p>None</p>
<p>Rx Only</p>	<p>Caution: Federal law restricts this device to sale by or on the order of a physician.</p>	<p>Reference: USA FDA 21 CFR 801.109</p>
	<p>Probes are tested to Type BF protection</p>	<p>TYPE BF APPLIED PART</p> <p>Refer to D1.20</p> <p>IEC 60601-1</p> <p>Medical Electrical Equipment - Part 1: General requirement for basic safety and essential performance</p>
	<p>A manufacturer's indication that a device is in conformity with the applicable requirements set out in EU MDR 2017/745 for CE marking and the Notified Body reference number.</p>	<p>CE marking of conformity Article 20, Annex V EU MDR 2017/745</p>
	<p>Medical device</p>	<p>Symbol for Medical Device in compliance with EU MDR directive</p>

Labeling symbols

	UK Conformity Assessed	Symbol for UK Conformity Assessed. MHRA Department of Business, Energy & Industrial Strategy December 31, 2020
	Switzerland Representative	Symbol for Switzerland Representative MU600_00_016e_MB

Contact information

United States



EchoNous Inc.

8310 154th Avenue NE

Building B, Suite 200

Redmond, WA 98052

Technical Support (toll free): (844) 854 0800

Sales (toll free): (844) 854 0800

Email: support@EchoNous.com

Website: www.EchoNous.com

European Economic Area



Authorized Representative

Advena Ltd

Tower Business Centre

2nd Flr, Tower Street

Swatar, BKR 4013

Malta



Switzerland Authorized Representative

CH REP

QUNIQUE GmbH
Bahnhofweg 17
5610 Wohlen
Switzerland

UK Responsible Person Qserve

Group UK, Ltd
49 Greek St, London W1D 4EG,
United Kingdom

Australia Sponsor

LC & Partners Pty Ltd
Level 32, 101 Miller Street
North Sydney, NSW, 2060
Australia
Tel: +61 2 9959 2400

Brazil Authorized Representative

Detentor da Notificação:

VR Medical Importadora e Distribuidora de Produtos Médicos Ltda

Rua Batataes no 391, conjuntos 11, 12 e 13 - Jardim Paulista
São Paulo - SP - 01423-010
CNPJ: 04.718.143/0001-94
SAC: 0800-7703661
Farm. Resp: Cristiane Ap. de Oliveira Aguirre – CRF/SP: 21.079
Notificação ANVISA no: 80102519147

Suporte ao cliente da EchoNous

Entre em contato com o suporte ao cliente:

Telefone: 844-854-0800

Fax: 425-242-5553

E-mail: info@echonous.com

Site: www.echonous.com

Fabricante:

EchoNous, Inc.

8310 154th Ave NE, Edifício B, Suíte 200

Redmond, WA 98052

Estados Unidos da América

País de Origem: Estados Unidos da América

ANATEL: 00430-22-14521

Biological safety

ALARA education program

The guiding principle for the use of diagnostic ultrasound is defined by the “as low as reasonably achievable” (ALARA) principle. The decision as to what is reasonable has been left to the judgment and insight of qualified personnel (users). No set of rules can be formulated that would be sufficiently complete to dictate the correct response to every circumstance. By keeping ultrasound exposure as low as possible, while obtaining diagnostic images, users can minimize ultrasonic bioeffects.

Since the threshold for diagnostic ultrasound bioeffects is undetermined, users are responsible for controlling total energy transmitted into the patient. Reconcile exposure time with diagnostic image quality. To ensure diagnostic image quality and limit exposure time, Kosmos provides controls that can be manipulated during the exam to optimize the results of the exam.

The ability of the user to abide by the ALARA principle is important. Advances in diagnostic ultrasound, not only in the technology but in the applications of that technology, have resulted in the need for more and better information to guide users. The output display tables are designed to provide that important information.

There are a number of variables which affect the way in which the output display tables can be used to implement the ALARA principle. These variables include index values, body size, location of the bone relative to the focal point, attenuation in the body, and ultrasound exposure time. Exposure time is an especially useful variable, because it is controlled by the user. The ability to limit the index values over time supports the ALARA principle.

A generic ALARA education program is supplied with Kosmos (see enclosed ISBN 1-932962-30-1, Medical Ultrasound Safety).

Applying ALARA

The Kosmos imaging mode used depends upon the information needed. B-mode imaging provides anatomical information, while Color-mode imaging provides information about blood flow.

Understanding the nature of the imaging mode being used allows users to apply the ALARA principle with informed judgment. Additionally, the Kosmos probe frequency, Kosmos Bridge setup values, scanning techniques, and experience allow users to meet the definition of the ALARA principle.

The decision as to the amount of acoustic output is, in the final analysis, up to the user. This decision must be based on the following factors: type of patient, type of exam, patient history, ease or difficulty of obtaining diagnostically useful information, and the potential localized heating of the patient due to transducer surface temperatures. Prudent use of Kosmos occurs when patient exposure is limited to the lowest index reading for the shortest amount of time necessary to achieve acceptable diagnostic results.

Although a high index reading does not mean that a bioeffect is actually occurring, a high index reading should be taken seriously. Every effort should be made to reduce the possible effects of a high index reading. Limiting exposure time is an effective way to accomplish this goal.

There are several system controls that the operator can use to adjust the image quality and limit the acoustic intensity. These controls are related to the techniques that a user might use to implement ALARA.

Output display and display accuracy

OUTPUT DISPLAY

Kosmos displays the two bioeffect indices prescribed by IEC 60601-2-37. Medical electrical equipment. Part 2-37: Particular requirements for the safety of ultrasonic medical diagnostic and monitoring equipment.

The thermal index (TI), provides a measure of the expected temperature increase.

Thermal index

TI is an estimate of the temperature increase of soft tissue or bone. There are three TI categories: TIS, TIB, and TIC. However, since Kosmos is not intended for transcranial applications, the TI for cranial bone at the surface (TIC) is not available for display on the system. The following TI categories are available for display:

- TIS: Soft tissue thermal index. The main TI category. Used for applications that do not image bone.
- TIB: Bone thermal index (bone located in a focal region).

MECHANICAL INDEX

MI is the estimated likelihood of tissue damage due to cavitation. The absolute maximum limits of the MI is 1.9 as set by the Guidance for Industry and FDA Staff - Marketing Clearance of Diagnostic Ultrasound Systems and Transducers (2019).

ISPTA

The Ispta is the Spatial Peak Temporal Average Intensity. The absolute maximum limit of Ispta is 720 mW/cm² as set by the Guidance for Industry and FDA Staff - Marketing Clearance of Diagnostic Ultrasound Systems and Transducers (2019).

OUTPUT DISPLAY ACCURACY

Output display accuracy of the bioeffect indices, MI and TI, is dependent on the uncertainty and precision of the measurement system, engineering assumptions within the acoustic model used to calculate the parameters, and variability in the acoustic output of the systems. EchoNous also compares both internal and third-

party acoustic measurements and confirms that both measurements are within recommended display quantization of 0.2 as outlined by the standards.



All MI and TI values displayed on Kosmos will not exceed the maximum global values (listed in the Track 3 acoustic output tables) by more than 0.2.

The accuracy of the MI and TI indices are as follows:

- MI: accurate to within $\pm 25\%$ or $+0.2$, whichever value is larger
- TI: accurate to within $\pm 30\%$ or $+0.2$, whichever value is larger

See Kosmos Torso and Kosmos Torso-One acoustic output tables, **TABLE 9-1.** through **TABLE 9-7.** See Kosmos Lexsa acoustic output tables **TABLE 9-8.** through **TABLE 9-14.**

Kosmos Torso and Kosmos Torso-One Acoustic Output Tables

See next page

TABLE 9-1. Transducer: Kosmos Torso and Kosmos Torso-One, Operating Mode: B-Mode, Combined acoustic output table: Reportable mode 1 (B-mode) Cardiac, body type 2, 16 cm

Index label	MI	TIS		TIB	
		At surface	Below surface	At surface	Below surface
Maximum index value	1.11	0.56		0.56	
Index component value		1: 0.30 2: 0.26	1: 0.30 2: 0.26	1: 0.30 2: 0.26	1: 0.30 2: 0.26
Acoustic parameters	$p_{r,\alpha}$ at z_{MI} (MPa)	1: 1.58			
	P (mW)		1: 41.03 2: 37.03	1: 41.03 2: 37.03	
	$P_{1 \times 1}$ (mW)		1: 30.42 2: 27.46	1: 30.42 2: 27.46	
	z_s (cm)		1: 4.27 2: 4.23		
	z_b (cm)				1: 3.93 2: 3.87
	z_{MI} (cm)	1: 4.20			
	$z_{pii,\alpha}$ (cm)	1: 4.20			
	f_{awf} (MHz)	1: 2.03	1: 2.03 2: 2.03	1: 2.03 2: 2.03	
	p_{rr} (Hz)	1: 1589.5			
	s_{rr} (Hz)	1: 28.4			
Other information	η_{pps}	1: 1			
	$I_{pa,\alpha}$ at $z_{pii,\alpha}$ (W/cm ²)	1: 91.28			
	$I_{spta,\alpha}$ at $z_{pii,\alpha}$ or $z_{sij,\alpha}$ (mW/cm ²)	25.13			
	I_{spta} at z_{pii} or z_{sij} (mW/cm ²)	42.50			
	p_r at z_{pii} (MPa)	1: 2.13			
	Operating control conditions	Exam	Cardiac		
	BMI Setting	2			
	Depth	16 cm			

NOTE 1 Only one operating condition per index.
 NOTE 2 Data should be entered for "at surface" and "below surface" both in the columns related to TIS or TIB.
 NOTE 3 Information need not be provided regarding TIC for an TRANSDUCER ASSEMBLY not intended for transcranial or neonatal cephalic uses.
 NOTE 4 If the requirements of 201.12.4.2a) are met, it is not required to enter any data in the columns related to TIS or TIB or TIC.
 NOTE 5 If the requirements of 201.12.4.2b) are met, it is not required to enter any data in the column related to MI.
 NOTE 6 Unshaded cells should have a numerical value. The equipment setting related to the index has to be entered in the operating control section.
 NOTE 7 The depths z_{pii} and $z_{pii,\alpha}$ apply to NON-SCANNING MODES, while the depths z_{sij} and $z_{sij,\alpha}$ apply to SCANNING MODES.

TABLE 9-2. Transducer: Kosmos Torso and Kosmos Torso-One, Operating Mode: M-Mode, Acoustic output reporting table: Reportable mode 3 M-mode (Cardiac, Body type: medium, 12 cm Depth)

Index label	MI	TIS		TIB	
		At surface	Below surface	At surface	Below surface
Maximum index value	0.43	5.32E-02		0.11	
Index component value		5.32E-02	2.15E-02	5.32E-02	0.11
Acoustic parameters	$p_{r,\alpha}$ at z_{MI} (MPa)	0.70			
	P (mW)		4.55	4.55	
	P_{1x1} (mW)		4.11	4.11	
	z_s (cm)		5.37		
	z_b (cm)				4.80
	z_{MI} (cm)	5.37			
	$z_{pii,\alpha}$ (cm)	5.37			
	f_{awf} (MHz)	2.72	2.72		2.68
	p_{rr} (Hz)	800			
Other information	s_{rr} (Hz)	N/A			
	n_{pps}	1			
	$I_{pa,\alpha}$ at $z_{pii,\alpha}$ (W/cm^2)	52.08			
	$I_{spta,\alpha}$ at $z_{pii,\alpha}$ or $z_{sii,\alpha}$ (mW/cm^2)	16.71			
	I_{spta} at z_{pii} or z_{sii} (mW/cm^2)	31.29			
	p_r at z_{pii} (MPa)	45.72			
Operating control Conditions					

NOTE 1 Only one operating condition per index.
 NOTE 2 Data should be entered for "at surface" and "below surface" both in the columns related to TIS or TIB.
 NOTE 3 If the requirements of 201.12.4.2a) are met, it is not required to enter any data in the columns related to TIS or TIB.
 NOTE 4 If the requirements of 201.12.4.2b) are met, it is not required to enter any data in the column related to MI.
 NOTE 5 Unshaded cells should have a numerical value. The equipment setting related to the index has to be entered in the operating control section.
 NOTE 6 The depths z_{pii} and $z_{pii,\alpha}$ apply to NON-SCANNING MODES, while the depths z_{sii} and $z_{sii,\alpha}$ apply to SCANNING MODES.

TABLE 9-3. Transducer: Kosmos Torso and Kosmos Torso-One, Operating Mode: M-Mode, Acoustic output reporting table: Reportable mode 4 M-mode (Cardiac, Body type: medium, 14 cm Depth)

Index label	MI	TIS		TIB	
		At surface	Below surface	At surface	Below surface
Maximum index value	0.39	5.33E-02		9.70E-02	
Index component value		5.33E-02	2.12E-02	5.33E-02	9.70E-02
Acoustic parameters	$p_{r,\alpha}$ at z_{MI} (MPa)	0.63			
	P (mW)		4.60		4.60
	P_{1x1} (mW)		4.14		4.14
	z_s (cm)		5.50		
	z_b (cm)				4.97
	z_{MI} (cm)	5.50			
	$z_{pii,\alpha}$ (cm)	5.50			
	f_{awf} (MHz)	2.70	2.70		2.67
Other information	p_{rr} (Hz)	800			
	s_{rr} (Hz)	N/A			
	n_{pps}	1			
	$I_{pa,\alpha}$ at $z_{pii,\alpha}$ (W/cm ²)	41.86			
	$I_{spta,\alpha}$ at $z_{pii,\alpha}$ or $z_{sii,\alpha}$ (mW/cm ²)	13.64			
	I_{spta} at z_{pii} or z_{sii} (mW/cm ²)	38.22			
	p_r at z_{pii} (MPa)	1.06			
Operating control conditions					

NOTE 1 Only one operating condition per index.
 NOTE 2 Data should be entered for "at surface" and "below surface" both in the columns related to TIS or TIB.
 NOTE 3 If the requirements of 201.12.4.2a) are met, it is not required to enter any data in the columns related to TIS or TIB.
 NOTE 4 If the requirements of 201.12.4.2b) are met, it is not required to enter any data in the column related to MI.
 NOTE 5 Unshaded cells should have a numerical value. The equipment setting related to the index has to be entered in the operating control section.
 NOTE 6 The depths z_{pii} and $z_{pii,\alpha}$ apply to NON-SCANNING MODES, while the depths z_{sii} and $z_{sii,\alpha}$ apply to SCANNING MODES.

TABLE 9-4. Transducer: Kosmos Torso and Kosmos Torso-One, Operating Mode: BC-Mode (Max MI, 12cm depth, small ROI, image top)

Index label	MI	TIS		TIB		TIC
		At surface	Below surface	At surface	Below surface	
Maximum index value	1.56	0.37		0.37		0.64
Index component value		1: 6.47E-02 2: 0.30	1: 6.47E-02 2: 0.30	1: 6.47E-02 2: 0.30	1: 6.47E-02 2: 0.30	
Acoustic parameters	$p_{r,\alpha}$ at z_{MI} (MPa)	2: 2.50				
	P (mW)		1: 5.89 2: 27.52		1: 5.89 2: 27.52	1: 5.89 2: 27.52
	P_{1x1} (mW)		1: 5.02 2: 24.07		1: 5.02 2: 24.07	
	z_s (cm)			1: N/A 2: N/A		
	z_b (cm)				1: N/A 2: N/A	
	z_{MI} (cm)	2: 1.90				
	$z_{pii,\alpha}$ (cm)	2: 2.00				
	f_{awf} (MHz)	2: 2.65	1: 2.71 2: 2.65		1: 2.71 2: 2.65	1: 2.71 2: 2.65
	p_{rr} (Hz)	2: 1248.9				
	s_{rr} (Hz)	2: 31.2				
n_{pps}	2: 10					
Other information	$I_{pa,\alpha}$ at $z_{pii,\alpha}$ (W/cm^2)	2: 282				
	$I_{spta,\alpha}$ at $z_{pii,\alpha}$ or $z_{sii,\alpha}$ (mW/cm^2)	160.04				
	I_{spta} at z_{pii} or z_{sii} (mW/cm^2)	233.06				
	p_r at z_{pii} (MPa)	2: 2.85				
Operating Control Conditions	Component 1: UTP 4					
	Component 2: UTP 275					

NOTE 1 Only one operating condition per index.
 NOTE 2 Data should be entered for "at surface" and "below surface" both in the columns related to TIS or TIB.
 NOTE 3 If the requirements of 201.12.4.2a) are met, it is not required to enter any data in the columns related to TIS or TIB.
 NOTE 4 If the requirements of 201.12.4.2b) are met, it is not required to enter any data in the column related to MI.
 NOTE 5 Unshaded cells should have a numerical value. The equipment setting related to the index has to be entered in the operating control section.
 NOTE 6 The depths z_{pii} and $z_{pii,\alpha}$ apply to NON-SCANNING MODES, while the depths z_{sii} and $z_{sii,\alpha}$ apply to SCANNING MODES.

TABLE 9-5. Transducer: Kosmos Torso and Kosmos Torso-One, Operating Mode: BC-Mode (Max TIS/TIB, ISPTA, 12cm depth, large ROI, image top)

Index label	MI	TIS		TIB		TIC
		At surface	Below surface	At surface	Below surface	
Maximum index value	0.98	0.96		0.96		1.74
Index component value		1: 5.66E-02 2: 0.90	1: 5.66E-02 2: 0.90	1: 5.66E-02 2: 0.90	1: 5.66E-02 2: 0.90	
Acoustic parameters	$p_{r,\alpha}$ at z_{MI} (MPa)	2: 1.58				
	P (mW)		1: 5.15 2: 86.25	1: 5.15 2: 86.25		1: 5.15 2: 86.25
	P_{1x1} (mW)		1: 4.39 2: 72.84	1: 4.39 2: 72.84		
	z_s (cm)			1: N/A 2: N/A		
	z_b (cm)				1: N/A 2: N/A	
	z_{MI} (cm)	2: 4.24				
	$z_{pii,\alpha}$ (cm)	2: 4.24				
	f_{awf} (MHz)	2: 2.59	1: 2.71 2: 2.59	1: 2.71 2: 2.59		1: 2.71 2: 2.59
	p_{rr} (Hz)	2: 3824.6				
s_{rr} (Hz)	2: 25.5					
n_{pps}	2: 10					
$I_{pa,\alpha}$ at $z_{pii,\alpha}$ (W/cm^2)	2: 153					
$I_{spta,\alpha}$ at $z_{pii,\alpha}$ or $z_{sii,\alpha}$ (mW/cm^2)	69.29					
I_{spta} at z_{pii} or z_{sii} (mW/cm^2)	151.32					
p_r at z_{pii} (MPa)	2: 2.23					
Operating Control Conditions	Component 1: UTP 4					
	Component 2: UTP 277					
<p>NOTE 1 Only one operating condition per index. NOTE 2 Data should be entered for "at surface" and "below surface" both in the columns related to TIS or TIB. NOTE 3 If the requirements of 201.12.4.2a) are met, it is not required to enter any data in the columns related to TIS or TIB. NOTE 4 If the requirements of 201.12.4.2b) are met, it is not required to enter any data in the column related to MI. NOTE 5 Unshaded cells should have a numerical value. The equipment setting related to the index has to be entered in the operating control section. NOTE 6 The depths z_{pii} and $z_{pii,\alpha}$ apply to NON-SCANNING MODES, while the depths z_{sii} and $z_{sii,\alpha}$ apply to SCANNING MODES.</p>						

TABLE 9-6. Transducer: Kosmos Torso and Kosmos Torso-One, Acoustic output reporting table, Operating Mode: PW Doppler (Max MI, TIS, TIB)

Index Label	MI	TIS		TIB		
		At Surface	Below Surface	At Surface	Below Surface	
Maximum Index Value	0.42	3.04		3.04		
Index Component Value		0.49	3.04	3.04	3.04	
Acoustic Parameters	$p_{r,\alpha}$ at z_{MI} (MPa)	0.59				
	P (mW)		50.93	50.93		
	P_{Tx1} (mW)		37.76	37.76		
	z_s (cm)		1.93			
	z_b (cm)				1.87	
	z_{MI} (cm)	1.93				
	$z_{pii,\alpha}$ (cm)	1.93				
	f_{awf} (MHz)	2.03	2.03		2.03	
	p_{rr} (Hz)	14468				
	s_{rr} (Hz)	N/A				
Other Information	n_{pps}	1				
	$I_{pa,\alpha}$ at $z_{pii,\alpha}$ (W/cm^2)	12.14				
	$I_{spta,\alpha}$ at $z_{pii,\alpha}$ or $z_{sii,\alpha}$ (mW/cm^2)	429.69				
	I_{spta} at z_{pii} or z_{sii} (mW/cm^2)	553.54				
	p_r at z_{pii} (MPa)	0.68				
Operating Control Conditions	PRF	14468 Hz				
	Gate Size	4 mm				
	Focal Depth	20mm				

NOTE 1 Only one operating condition per index.
 NOTE 2 Data should be entered for "at surface" and "below surface" both in the columns related to TIS or TIB.
 NOTE 3 Information need not be provided regarding TIC for an TRANSDUCER ASSEMBLY not intended for transcranial or neonatal cephalic uses.
 NOTE 4 If the requirements of 201.12.4.2a) are met, it is not required to enter any data in the columns related to TIS or TIB or TIC.
 NOTE 5 If the requirements of 201.12.4.2b) are met, it is not required to enter any data in the column related to MI.
 NOTE 6 Unshaded cells should have a numerical value. The equipment setting related to the index has to be entered in the operating control section.
 NOTE 7 The depths z_{pii} and $z_{pii,\alpha}$ apply to NON-SCANNING MODES, while the depths z_{sii} and $z_{sii,\alpha}$ apply to SCANNING MODES.

TABLE 9-7. Transducer: Kosmos Torso and Kosmos Torso-One, Acoustic Output reporting table, Operating Mode: CW Doppler (Max MI, TIS, TIB)

Index Label	MI	TIS		TIB	
		At Surface	Below Surface	At Surface	Below Surface
Maximum Index Value	0.07	0.49		2.43	
Index Component Value		0.47	0.49	0.47	2.43
Acoustic Parameters	$p_{r,\alpha}$ at z_{MI} (MPa)	0.0976			
	P (mW)		62.48	62.48	
	P_{1x1} (mW)		50.17		50.17
	z_s (cm)		1.27		
	z_b (cm)				1.27
	z_{MI} (cm)	0.9			
	$z_{pii,\alpha}$ (cm)	1.27			
	f_{awf} (MHz)	1.95	1.95		1.95
	pr (Hz)	N/A			
	srr (Hz)	N/A			
Other Information	n_{pps}	1			
	$I_{pa,\alpha}$ at $z_{pii,\alpha}$ (W/cm^2)	N/A			
	$I_{spta,\alpha}$ at $z_{pii,\alpha}$ or $z_{sii,\alpha}$ (mW/cm^2)	279.77			
	I_{spta} at z_{pii} or z_{sii} (mW/cm^2)	331.51			
	p_r at z_{pii} (MPa)	0.10			
Operating Control Conditions	Focal Depth	4 cm			
	CW Mode				

NOTE 1 Only one operating condition per index.
 NOTE 2 Data should be entered for "at surface" and "below surface" both in the columns related to TIS or TIB.
 NOTE 3 Information need not be provided regarding TIC for an TRANSDUCER ASSEMBLY not intended for transcranial or neonatal cephalic uses.
 NOTE 4 If the requirements of 201.12.4.2a) are met, it is not required to enter any data in the columns related to TIS or TIB or TIC.
 NOTE 5 If the requirements of 201.12.4.2b) are met, it is not required to enter any data in the column related to MI.
 NOTE 6 Unshaded cells should have a numerical value. The equipment setting related to the index has to be entered in the operating control section.
 NOTE 7 The depths z_{pii} and $z_{pii,\alpha}$ apply to NON-SCANNING MODES, while the depths z_{sii} and $z_{sii,\alpha}$ apply to SCANNING MODES.

Kosmos Lexsa Maximum Acoustic Output Summary

TABLE 9-8. Transducer: Kosmos Lexsa Acoustic output reporting table, Operating Mode: B-Mode (Max MI, ISPTA, MSK, 3cm depth)

Index label	MI	TIS		TIB		TIC
		At surface	Below surface	At surface	Below surface	
Maximum index value	0.77	5.39E-03		5.39E-03		1.25E-02
Index component value		5.39E-03	5.39E-03	5.39E-03	5.39E-03	
Acoustic Parameters	$p_{r,\alpha}$ at z_{MI} (MPa)	2.01				
	P (mW)		0.52		0.52	0.52
	P_{1x1} (mW)		0.15		0.15	
	z_s (cm)			1.57		
	z_b (cm)					1.57
	z_{MI} (cm)	1.43				
	$z_{pii,\alpha}$ (cm)	1.57				
	f_{awf} (MHz)	6.77	7.44		7.44	7.44
	p_{rr} (Hz)	1820.0				
Other Information	s_{rr} (Hz)	28.0				
	n_{pps}	1				
	$I_{pa,\alpha}$ at $z_{pii,\alpha}$ (W/cm ²)	1.7E+02				
	$I_{spta,\alpha}$ at $z_{pii,\alpha}$ or $z_{sij,\alpha}$ (mW/cm ²)	1.62				
	I_{spta} at z_{pii} or z_{sij} (mW/cm ²)	3.58				
	p_r at z_{pii} (MPa)	2.24				
Operating Control Conditions	UTP 71					

NOTE 1 Only one operating condition per index.
 NOTE 2 Data should be entered for "at surface" and "below surface" both in the columns related to TIS or TIB.
 NOTE 3 If the requirements of 201.12.4.2a) are met, it is not required to enter any data in the columns related to TIS or TIB.
 NOTE 4 If the requirements of 201.12.4.2b) are met, it is not required to enter any data in the column related to MI.
 NOTE 5 Unshaded cells should have a numerical value. The equipment setting related to the index has to be entered in the operating control section.
 NOTE 6 The depths z_{pii} and $z_{pii,\alpha}$ apply to NON-SCANNING MODES, while the depths z_{sij} and $z_{sij,\alpha}$ apply to SCANNING MODES.

TABLE 9-9. Transducer: Kosmos Lexsa Acoustic output reporting table, Operating Mode: B-Mode (Max TIS, TIB, MSK, 10cm depth)

Index label	MI	TIS		TIB		TIC
		At surface	Below surface	At surface	Below surface	
Maximum index value	0.19	9.16E-03		9.16E-03		2.05E-02
Index component value		9.16E-03	9.16E-03	9.16E-03	9.16E-03	
Acoustic Parameters	$p_{r,\alpha}$ at z_{MI} (MPa)	0.53				
	P (mW)		0.85	0.85		0.85
	$P_{I \times I}$ (mW)		0.25	0.25		
	z_s (cm)		1.63			
	z_b (cm)				1.63	
	z_{MI} (cm)	1.63				
	$z_{pii,\alpha}$ (cm)	1.63				
	f_{awf} (MHz)	7.69	7.69	7.69	7.69	7.69
	p_{rr} (Hz)	1300.0				
	s_{rr} (Hz)	20.0				
Other Information	n_{pps}	1				
	$I_{pa,\alpha}$ at $z_{pii,\alpha}$ (W/cm ²)	17.0				
	$I_{spta,\alpha}$ at $z_{pii,\alpha}$ or $z_{sii,\alpha}$ (mW/cm ²)	1.36				
	I_{spta} at z_{pii} or z_{sii} (mW/cm ²)	3.23				
	p_r at z_{pii} (MPa)	0.82				
Operating Control Conditions	UTP 87					

NOTE 1 Only one operating condition per index.
 NOTE 2 Data should be entered for "at surface" and "below surface" both in the columns related to TIS or TIB.
 NOTE 3 If the requirements of 201.12.4.2a) are met, it is not required to enter any data in the columns related to TIS or TIB.
 NOTE 4 If the requirements of 201.12.4.2b) are met, it is not required to enter any data in the column related to MI.
 NOTE 5 Unshaded cells should have a numerical value. The equipment setting related to the index has to be entered in the operating control section.
 NOTE 6 The depths z_{pii} and $z_{pii,\alpha}$ apply to NON-SCANNING MODES, while the depths z_{sii} and $z_{sii,\alpha}$ apply to SCANNING MODES.

TABLE 9-10. Transducer: Kosmos Lexsa Acoustic output reporting table, Operating Mode: BC, CPD-Mode (Max MI, Vascular, 4cm depth, large ROI)

Index label	MI	TIS		TIB		TIC
		At surface	Below surface	At surface	Below surface	
Maximum index value	1.37	7.72E-02		7.72E-02		0.29
Index component value		1: 2.35E-03 2: 7.48E-02	1: 2.35E-03 2: 7.48E-02	1: 2.35E-03 2: 7.48E-02	1: 2.35E-03 2: 7.48E-02	
Acoustic Parameters	$p_{r,\alpha}$ at z_{MI} (MPa)	2: 2.88				
	P (mW)		1: 0.26 2: 11.93	1: 0.26 2: 11.93		1: 0.26 2: 11.93
	$P_{1 \times 1}$ (mW)		1: 6.90E-02 2: 3.56	1: 6.90E-02 2: 3.56		
	z_s (cm)			1: N/A 2: N/A		
	z_b (cm)				1: N/A 2: N/A	
	z_{MI} (cm)	2: 0.96				
	$z_{pii,\alpha}$ (cm)	2: 1.57				
	f_{awf} (MHz)	2: 4.42	1: 7.15 2: 4.42	1: 7.15 2: 4.42		1: 7.15 2: 4.42
Other Information	pr (Hz)	2: 8236.4				
	srr (Hz)	2: 21.4				
	n_{pps}	2: 12				
	$I_{pa,\alpha}$ at $z_{pii,\alpha}$ (W/cm ²)	2: 23.3				
	$I_{spta,\alpha}$ at $z_{pii,\alpha}$ or $z_{sij,\alpha}$ (mW/cm ²)	29.58				
	I_{spta} at z_{pii} or z_{sij} (mW/cm ²)	48.42				
	p_r at z_{pii} (MPa)	2: 0.95				
Operating Control Conditions	Component 1: UTP 225					
	Component 2: UTP 339 (16V)					

NOTE 1 Only one operating condition per index.
 NOTE 2 Data should be entered for "at surface" and "below surface" both in the columns related to TIS or TIB.
 NOTE 3 Information need not be provided regarding TIC for an TRANSDUCER ASSEMBLY not intended for transcranial or neonatal cephalic uses.
 NOTE 4 If the requirements of 201.12.4.2a) are met, it is not required to enter any data in the columns related to TIS or TIB or TIC.
 NOTE 5 If the requirements of 201.12.4.2b) are met, it is not required to enter any data in the column related to MI.
 NOTE 6 Unshaded cells should have a numerical value. The equipment setting related to the index has to be entered in the operating control section.
 NOTE 7 The depths z_{pii} and $z_{pii,\alpha}$ apply to NON-SCANNING MODES, while the depths z_{sij} and $z_{sij,\alpha}$ apply to SCANNING MODES.

TABLE 9-11. Transducer: Kosmos Lexsa Acoustic output reporting table, Operating Mode: BC, CPD-Mode (Max ISPTA, Vascular, 4cm depth, small ROI, image top)

Index label	MI	TIS		TIB		TIC
		At surface	Below surface	At surface	Below surface	
Maximum index value	1.37	6.50E-02		6.50E-02		7.98E-02
Index component value		1: 3.23E-03 2: 6.18E-02	1: 3.23E-03 2: 6.18E-02	1: 3.23E-03 2: 6.18E-02	1: 3.23E-03 2: 6.18E-02	
$p_{r,\alpha}$ at z_{MI} (MPa)	2: 2.88					
P (mW)		1: 0.36 2: 2.94		1: 0.36 2: 2.94		1: 0.36 2: 2.94
$P_{I \times I}$ (mW)		1: 9.49E-02 2: 2.94		1: 9.49E-02 2: 2.94		
z_5 (cm)			1: N/A 2: N/A			
z_b (cm)					1: N/A 2: N/A	
z_{MI} (cm)	2: 0.96					
$z_{pii,\alpha}$ (cm)	2: 1.57					
f_{awf} (MHz)	2: 4.42	1: 7.15 2: 4.42		1: 7.15 2: 4.42		1: 7.15 2: 4.42
p_{rr} (Hz)	2: 2026.6					
s_{rr} (Hz)	2: 28.1					
η_{pps}	2: 12					
$I_{pa,\alpha}$ at $z_{pii,\alpha}$ (W/cm ²)	2: 23.3					
$I_{spta,\alpha}$ at $z_{pii,\alpha}$ or $z_{sii,\alpha}$ (mW/cm ²)	48.65					
I_{spta} at z_{pii} or z_{sii} (mW/cm ²)	79.44					
p_r at z_{pii} (MPa)	2: 0.95					
Operating Control Conditions	Component 1: UTP 225					
	Component 2: UTP 339 (16V)					

NOTE 1 Only one operating condition per index.
 NOTE 2 Data should be entered for "at surface" and "below surface" both in the columns related to TIS or TIB.
 NOTE 3 If the requirements of 201.12.4.2a) are met, it is not required to enter any data in the columns related to TIS or TIB.
 NOTE 4 If the requirements of 201.12.4.2b) are met, it is not required to enter any data in the column related to MI.
 NOTE 5 Unshaded cells should have a numerical value. The equipment setting related to the index has to be entered in the operating control section.
 NOTE 6 The depths z_{pii} and $z_{pii,\alpha}$ apply to NON-SCANNING MODES, while the depths z_{sii} and $z_{sii,\alpha}$ apply to SCANNING MODES.

TABLE 9-12. Transducer: Kosmos Lexsa Acoustic output reporting table, Operating Mode: BC, CPD-Mode (Max TIS, TIB)

Index label	MI	TIS		TIB		TIC
		At surface	Below surface	At surface	Below surface	
Maximum index value	0.94	0.10		0.10		0.29
Index component value		1: 1.91E-03 2: 0.10	1: 1.91E-03 2: 0.10	1: 1.91E-03 2: 0.10	1: 1.91E-03 2: 0.10	
Acoustic Parameters	$p_{r,\alpha}$ at z_{MI} (MPa)	2: 2.34				
	P (mW)		1: 0.22 2: 11.60		1: 0.22 2: 11.60	1: 0.22 2: 11.60
	P_{1x1} (mW)		1: 5.62E-02 2: 3.46		1: 5.62E-02 2: 3.46	
	z_s (cm)			1: N/A 2: N/A		
	z_b (cm)				1: N/A 2: N/A	
	z_{MI} (cm)	2: 0.93				
	$z_{pii,\alpha}$ (cm)	2: 1.40				
	f_{awf} (MHz)	2: 6.22	1: 7.15 2: 6.22		1: 7.15 2: 6.22	1: 7.15 2: 6.22
	Other Information	p_{rr} (Hz)	2: 8830.3			
s_{rr} (Hz)		2: 17.8				
n_{pps}		2: 16				
$I_{pa,\alpha}$ at $z_{pii,\alpha}$ (W/cm ²)		2: 73.7				
$I_{spta,\alpha}$ at $z_{pii,\alpha}$ or $z_{sii,\alpha}$ (mW/cm ²)		29.56				
I_{spta} at z_{pii} or z_{sii} (mW/cm ²)		54.39				
p_r at z_{pii} (MPa)	2: 1.51					
Operating Control Conditions	Component 1: UTP 225					
	Component 2: UTP 161					

NOTE 1 Only one operating condition per index.
 NOTE 2 Data should be entered for "at surface" and "below surface" both in the columns related to TIS or TIB.
 NOTE 3 Information need not be provided regarding TIC for an TRANSDUCER ASSEMBLY not intended for transcranial or neonatal cephalic uses.
 NOTE 4 If the requirements of 201.12.4.2a) are met, it is not required to enter any data in the columns related to TIS or TIB or TIC.
 NOTE 5 If the requirements of 201.12.4.2b) are met, it is not required to enter any data in the column related to MI.
 NOTE 6 Unshaded cells should have a numerical value. The equipment setting related to the index has to be entered in the operating control section.
 NOTE 7 The depths z_{pii} and $z_{pii,\alpha}$ apply to NON-SCANNING MODES, while the depths z_{sii} and $z_{sii,\alpha}$ apply to SCANNING MODES.

TABLE 9-13. Transducer: Kosmos Lexsa Acoustic output reporting table, Operating Mode: PW Doppler (Max MI)

Index label	MI	TIS		TIB		TIC
		At surface	Below surface	At surface	Below surface	
Maximum index value	0.35	0.19		0.47		0.26
Index component value		0.19	0.06	0.19	0.47	
Acoustic Parameters	$p_{r,\alpha}$ at z_{MI} (MPa)	0.88				
	P (mW)		6.45		6.45	6.45
	P_{1x1} (mW)		6.45		6.45	
	z_s (cm)			2.6		
	z_b (cm)				2.6	
	z_{MI} (cm)	1.22				
	$z_{pii,\alpha}$ (cm)	1.24				
	f_{awf} (MHz)	6.26	6.26	6.26	6.26	6.26
Other Information	p_{rr} (Hz)	15625				
	s_{rr} (Hz)	N/A				
	n_{pps}	1				
	$I_{pa,\alpha}$ at $z_{pii,\alpha}$ (W/cm^2)	23.9				
	$I_{spta,\alpha}$ at $z_{pii,\alpha}$ or $z_{sji,\alpha}$ (mW/cm^2)	338.3				
	I_{spta} at z_{pji} or z_{sji} (mW/cm^2)	575.2				
p_r at z_{pji} (MPa)	1.14					
Operating Control Conditions	PRF	15625				
	Gate Size	5mm				
	Gate Focal Depth	10mm				

NOTE 1 Only one operating condition per index.
 NOTE 2 Data should be entered for "at surface" and "below surface" both in the columns related to TIS or TIB.
 NOTE 3 If the requirements of 201.12.4.2a) are met, it is not required to enter any data in the columns related to TIS or TIB.
 NOTE 4 If the requirements of 201.12.4.2b) are met, it is not required to enter any data in the column related to MI.
 NOTE 5 Unshaded cells should have a numerical value. The equipment setting related to the index has to be entered in the operating control section.
 NOTE 6 The depths z_{pji} and $z_{pii,\alpha}$ apply to NON-SCANNING MODES, while the depths z_{sji} and $z_{sji,\alpha}$ apply to SCANNING MODES.

TABLE 9-14. Transducer: Kosmos Lexsa Acoustic output reporting table, Operating Mode: PW Doppler (Max TIS, TIB, TIC)

Index label	MI	TIS		TIB		TIC
		At surface	Below surface	At surface	Below surface	
Maximum index value	0.15	0.66		1.64		0.64
Index component value		0.66	0.26	0.66	1.64	
Acoustic Parameters	$p_{r,\alpha}$ at z_{MI} (MPa)	0.38				
	P (mW)		22.23		22.23	22.23
	P_{1x1} (mW)		22.23		22.23	
	z_s (cm)			2.6		
	z_b (cm)					2.6
	z_{MI} (cm)	2.58				
	$z_{pii,\alpha}$ (cm)	2.58				
	f_{awf} (MHz)	6.25	6.25	6.25	6.25	6.25
Other Information	p_{rr} (Hz)	7621				
	s_{rr} (Hz)	N/A				
	n_{pps}	1				
	$I_{pa,\alpha}$ at $z_{pii,\alpha}$ (W/cm^2)	5.42				
	$I_{spta,\alpha}$ at $z_{pii,\alpha}$ or $z_{sji,\alpha}$ (mW/cm^2)	127.8				
	I_{spta} at z_{pji} or z_{sji} (mW/cm^2)	539.19				
	p_r at z_{pji} (MPa)	0.73				
Operating Control Conditions	PRF	7621				
	Gate Size	5mm				
	Gate Focal Depth	50mm				

NOTE 1 Only one operating condition per index.
 NOTE 2 Data should be entered for "at surface" and "below surface" both in the columns related to TIS or TIB.
 NOTE 3 If the requirements of 201.12.4.2a) are met, it is not required to enter any data in the columns related to TIS or TIB.
 NOTE 4 If the requirements of 201.12.4.2b) are met, it is not required to enter any data in the column related to MI.
 NOTE 5 Unshaded cells should have a numerical value. The equipment setting related to the index has to be entered in the operating control section.
 NOTE 6 The depths z_{pji} and $z_{pii,\alpha}$ apply to NON-SCANNING MODES, while the depths z_{sji} and $z_{sji,\alpha}$ apply to SCANNING MODES.

Measurement accuracy

Measurement accuracy for distance and area in B-mode images are as follows:

- Axial measurement accuracy: Axial distance measurements in 2D imaging modes shall be accurate to within +/- 2% of the displayed value (or 1 mm, whichever is larger).
- Lateral distance measurement accuracy: Lateral distance measurements in 2D imaging modes shall be accurate to within +/- 2% of the displayed value (or 1 mm, whichever is larger).
- Diagonal measurement accuracy: Diagonal distance measurements in 2D imaging modes shall be accurate to within +/- 2% of the displayed value (or 1 mm, whichever is larger).
- Area measurement accuracy: Area measurement accuracy in 2D imaging modes shall be +/-4% of the nominal value.

Measurement accuracy for distance and time in M-mode images are as follows:

- M-mode distance measurement: M-mode distance measurements shall be accurate to within +/- 3% of the displayed value.
- M-mode time measurement accuracy: M-mode time measurements shall be accurate to within +/- 2% of the displayed value.

Kosmos AI-assisted EF Workflow measurements accuracy:

- The accuracy of the Kosmos EF calculations depends on the correct selection of ED/ES frames and accurate tracing of the LV endocardial border. It is important to review the initial ED/ES frames and LV contours provided by the Kosmos AI algorithms, confirm their accuracy, and edit them, as required.
 - Ensure that the selected ED/ES frames accurately represent the corresponding end-diastolic and end-systolic cardiac phases in the A4C and A2C clips. Use the editing tool to select a more appropriate frame, as required.
 - Ensure that the LV contours accurately follows the LV endocardium. Use the editing tool to properly trace and adjust the LV contours.

- When possible, acquire both A4C and A2C clips to obtain a biplane A4C/A2C EF, which is more accurate than the single plane A4C EF.

Control effects

Kosmos does not provide the user with direct control of acoustic output power. Kosmos has been designed to automatically adjust the output to ensure that acoustic limits are not exceeded in any imaging mode. Since there is no direct user control for output, the user should rely on controlling exposure time and scanning technique to implement the ALARA principle.

Related references

- U.S. Dept. of Health and Human Services, Food and Drug Administration, Guidance for Industry and FDA Staff - Marketing Clearance of Diagnostic Ultrasound Systems and Transducers (2019)
- IEC 60601-2-37:2015 Medical electrical equipment - Part 2-37: Particular requirements for the basic safety and essential performance of ultrasonic medical diagnostic and monitoring equipment
- IEC 62359:2017 Ultrasonics - Field characterization - Test methods for the determination of thermal and mechanical indices related to medical diagnostic ultrasonic fields
- NEMA UD 2-2004 (R2009) Acoustic Output Measurement Standard for Diagnostic Ultrasound Equipment Revision 3

Transducer surface temperature rise

TABLE 9-15. summarizes the expected maximum temperature rise for Kosmos. The values are based on a statistical sample test of production-equivalent systems and were measured in accordance with IEC 60601-2-37. The values listed in the table are determined with 90% confidence, that 90% of the systems will result in a temperature rise less than or equal to that stated in the table.

TABLE 9-15. Surface temperature rise

Test	Temperature rise (°C)
Still air	16.02
Simulated use	9.85

ECG supplemental information

- Recommended ECG electrodes: Use a fluid-resistant, foam-backed electrode, such as 3M™ Red Dot™ Clear Plastic Monitoring Electrode 2235.
- Kosmos uses single ECG filter from 0.65 Hz - 47.5 Hz.
- Kosmos, with a fully charged battery, provides about 90 minutes of continuous operation.
- The Kosmos HR calculation is accurate to within $\pm 10\%$ or $\pm 5/\text{min}$, whichever is greater for regular HRs in the specified range per 60601-2-27 HR Accuracy Requirement.
- Kosmos HR range (adult): 30/min to 200/min.
- Kosmos HR range (pediatric): 30/min to 250/min.
- Noise suppression: Right leg drive max. voltage 2.12Vrms.
- Method of HR averaging: Data is analyzed for R-wave peaks in approx 2.5 seconds sampling periods. If required, two sampling periods are combined to capture a minimum of three R-wave peaks. The HR is updated after every sampling period.
- Kosmos provides the following sweep speeds: 25 mm/sec, 50 mm/sec, 75 mm/sec, and 100 mm/sec.
- When calculating HR, Kosmos is capable of rejecting tall T-waves (as false QRS peaks) up to amplitudes that are up to 75% of QRS amplitude.

Ergonomics



Repetitive ultrasound scanning may cause occasional discomfort in your thumbs, fingers, hands, arms, shoulders, eyes, neck, back, or other parts of your body. However, if you experience symptoms such as constant or recurring discomfort, soreness, pain, throbbing, aching, tingling, numbness, stiffness, burning sensation, muscle fatigue/weakness, or limited range of motion, do not ignore these warning signs. Promptly see a qualified health professional. Symptoms such as these can be linked with Work Related Musculoskeletal Disorders (WRMSDs). WRMSDs can be painful and may result in potentially disabling injuries to the nerves, muscles, tendons, or other parts of the body. Examples of WRMSDs include bursitis, tendonitis, tenosynovitis, carpal tunnel syndrome, and De Quervain syndrome.

While researchers are not able to definitively answer many questions about WRMSDs, there is a general agreement that certain factors are associated with their occurrence, including preexisting medical and physical conditions, overall health, equipment, and body position while performing work, frequency of work, and duration of work.

Kosmos is intended for quick-look applications by qualified health professionals. It is not intended for continual use in radiology or other departments. If you need to use the device for a continual period, take the following precautions:

- Position yourself comfortably, either with a chair with appropriate lower-back support or by sitting or standing upright.
- Minimize twisting, relax your shoulders, and support your arm with a cushion.
- Hold Kosmos Torso, Kosmos Torso-One or Kosmos Lexsa lightly, keep your wrist straight, and minimize the pressure applied to the patient.
- Take regular breaks.

Electromagnetic compatibility



The System complies with the Electromagnetic Compatibility requirements of AS/NZ CISPR 11:2015 and EN IEC 60601-1-2:2014. However, electronic and mobile communications equipment may transmit electromagnetic energy through air and there is no guarantee that interference will not occur in a particular installation or environment. Interference may result in artifacts, distortion, or degradation of the ultrasound image. If the System is found to cause or respond to interference, try re-orienting the System or the affected device, or increasing the separation distance between the devices. Contact EchoNous customer support or your EchoNous distributor for further information.



EchoNous does not recommend the use of high-frequency electromedical devices in proximity to its systems. EchoNous equipment has not been validated for use with high-frequency electrosurgical devices or procedures. Use of high-frequency electrosurgical devices in proximity to its systems may lead to abnormal system behavior or shutdown of the system. To avoid the risk of a burn hazard, do not use Kosmos probes with high-frequency surgical equipment. Such a hazard may occur in the event of a defect in the high-frequency surgical neutral electrode connection.



The System contains sensitive components and circuits. Failure to observe proper static control procedures may result in damage to the System. Any faults should be reported to EchoNous customer support or your EchoNous distributor for repair.

The **System** is intended for use in the electromagnetic environment specified below. The user of the **System** should assure that it is used in such an environment.

Electromagnetic emissions

TABLE 9-16. Guidance and manufacturer's declaration: electromagnetic emissions

Emissions test	Compliance	Electromagnetic environment: guidance
RF emissions CISPR 11	Group 1	The System uses RF energy only for its internal function. Therefore, its RF emissions are very low and are not likely to cause any interference in nearby electronic equipment.
RF emissions CISPR 11	Class A	
Harmonic emissions IEC 61000-3-2	Class A	The System is suitable for use in all establishments other than domestic and those directly connected to the public low-voltage power supply network that supplies buildings used for domestic purposes.
Voltage fluctuations/ flicker emissions IEC 61000-3-3	Complies	

The **System** has Class A compliance in meaning it is suitable for use in all establishments other than domestic and those directly connected to the public low-voltage power supply network that supplies buildings used for domestic purposes. If the **System** is found to cause or respond to interference follow the guidelines in the warning section above.

Electromagnetic immunity


TABLE 9-17. Guidance and manufacturer's declaration: electromagnetic immunity

Immunity test	Compliance level	Electromagnetic environment: guidance
Electrostatic discharge (ESD) IEC 61000-4-2	±8 kV contact ±2 kV, ±4 kV, ±8 kV, ±15 kV air	Floors should be wood, concrete or ceramic tile. If floors are covered with synthetic material, the relative humidity should be at least 30%.
Electrical fast transient/burst IEC 61000-4-4	±2 kV at 100 kHz repetition frequency on Power Supply Lines	Mains power quality should be that of a typical commercial or hospital environment.
Surge IEC 61000-4-5	±0.5 kV, ±1 kV Line to Line ±0.5 kV, ±1 kV, ±2 kV Line to ground	Mains power quality should be that of a typical commercial or hospital environment.
Voltage dips, short interruptions and voltage variations on power supply input lines IEC 61000-4-11	0% U_t ; 0.5 Cycle at 0 deg, 45 deg, 90 deg, 135 deg, 180 deg, 225 deg, 270 deg and 315 deg. 0% U_t ; 1 cycle and 70% U_t 25/30 cycles single phase at 0 deg	Mains power quality should be that of a typical commercial or hospital environment.
Power frequency (50/60 Hz) magnetic field IEC 61000-4-8	8 A/m at 30 kHz in CW modulation 65 A/m at 134.2 kHz in 2.1 kHz Pulse modulation 75 A/m at 13.56 MHz in 50 kHz Pulse modulation	Power frequency magnetic fields should be at levels characteristic of a typical location in a typical commercial or hospital environment.

TABLE 9-17. Guidance and manufacturer's declaration: electromagnetic immunity

<p>^{2,3}Conducted RF IEC 61000-4-6</p>	<p>3 Vrms⁶ 0.15 MHz - 80 MHz 6 Vrms in ISM and Amateur radio bands between 0.15 MHz - 80 MHz 80% AM at 1 kHz</p>	<p>Portable and mobile RF communications equipment should be used no closer to any part of the system, including cables, than the recommended separation distance calculated from the equation applicable to the frequency of the transmitter</p> <p>Recommended separation distance</p> $d = 1.2 \sqrt{P}$
---	---	--

TABLE 9-17. Guidance and manufacturer's declaration: electromagnetic immunity

<p>Radiated RF IEC 61000-4-3</p>	<p>3 V/m 80 MHz- 2.7 GHz 80% AM at 1 kHz</p>	<p>$d=1.2 \sqrt{P}$ 80MHz to 800MHz $d=2.3 \sqrt{P}$ 800MHz to 2.5GHz</p> <p>Where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer and d is the recommended separations distance in meters (m).</p> <p>Field strengths from fixed RF transmitters, as determined by an electromagnetic site survey⁴, should be less than the compliance level in each frequency range⁵.</p> <p>Interference may occur in the vicinity of equipment marked with the following symbol.</p> 
<p>1 UT is the AC mains voltage prior to application of the test level 2 At 80MHz and 800 MHz, the higher frequency range applies 3 These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people. 4 Field strengths from fixed transmitters, such as base stations for radio (cellular/cordless) telephones and land mobile radios, amateur radio, AM and FM radio broadcast and TV broadcast cannot be predicted theoretically with accuracy. To assess the electromagnetic environment due to fixed RF transmitters, an electromagnetic site survey should be considered. If the measured field strength in the location in which the system is used exceeds the applicable RF compliance level above, the system should be observed to verify normal operation. If abnormal performance is observed, additional measures may be necessary, such as re-orientating or relocating the system. 5 Over the frequency range 150kHz to 80MHz, field strengths should be less than 3V/m. 6 Conducted RF energy may cause noise in the ECG waveform. If noise is detected on the ECG waveform, disconnect the system from AC power.</p>		

- ⚠ When using the optional mobile stand, the **System** can be susceptible to ESD and may require manual intervention. If ESD results in a **System** error, unplug the probe and plug back in to restore operation.
- ⚠ Conducted RF energy may cause noise in the ECG waveform. If noise is detected on the ECG waveform, disconnect Kosmos from AC power.
- ⚠ Using cables or accessories other than those specified for use with the system, may result in increased emissions or decreased immunity of the system.

Separation distances

TABLE 9-18. Separation distances

Recommended separation distances between portable and mobile RF communications equipment and the EchoNous System			
Rated maximum output power of transmitter W	Separation distance according to frequency of transmitter		
	150 kHz to 80 MHz	80 MHz to 800 MHz	800 MHz to 2,5 GHz
	$d=1.2 \sqrt{P}$	$d=1.2 \sqrt{P}$	$d=2.3 \sqrt{P}$
0.01	0.12	0.12	0.23
0.1	0.38	0.38	0.73
1	1.2	1.2	2.3
10	3.8	3.8	7.3
100	12	12	23

For transmitters rated at a maximum output power not listed above, the recommended separation distance *d* in meters (m) can be estimated using the equation applicable to the frequency of the transmitter, where *P* is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer.

NOTE 1: At 80 MHz and 800 MHz, the separation distance for the higher frequency range applies.

NOTE 2: These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.

Certificate and compliance

For details on certificate-specified and compliant mark (including number of certificate and authorization), perform the following steps:

- ★ From the Home screen, tap **SETTINGS** --> **About** --> **Regulatory**.

Intentional radiator

FCC Intentional Radiator Certification contains:

- FCC ID: 2AU8B-ECHKMOS
- IC ID: 25670-ECHKMOS

Kosmos contains an intentional radiator approved by the FCC under the FCC ID numbers, as shown above. Kosmos complies with Part 15 of the FCC rules. Operation is subject to the following two conditions: (1) Kosmos may not cause harmful interference and (2) Kosmos must accept any interference received, including interference that may cause undesirable operation.

NO MODIFICATION: Modifications to Kosmos shall not be made without the written consent of EchoNous, Inc. Unauthorized modifications may void the authority granted under Federal Communications Commission rules permitting the operation of this device.

Operations in the 5.15-5.25GHz band are restricted to indoor usage only.

Class B device

Kosmos has been tested and found to comply with the limits for a class B digital device, pursuant to part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses and can radiate radio frequency energy and if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna
- Increase the separation between the equipment and receiver
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected

- Consult the dealer or an experienced radio/TV technician for help

Kosmos has been verified to comply with the limits for a class B computing device, pursuant to FCC rules. In order to maintain compliance with FCC regulations, shielded cables must be used with this equipment. Operation with non-approved equipment or unshielded cables is likely to result in interference to radio and TV reception. The user is cautioned that changes and modifications made to the equipment without the approval of manufacturer could void the user's authority to operate this equipment.

Industry Canadian statement

This device complies with Industry Canada licence-exempt RSS standard(s). Operation is subject to the following two conditions: (1) this device may not cause interference, and (2) this device must accept any interference, including interference that may cause undesired operation of the device

Le présent appareil est conforme aux CNR d'Industrie Canada applicables aux appareils radio exempts de licence. L'exploitation est autorisée aux deux conditions suivantes: (1) l'appareil ne doit pas produire de brouillage, et (2) l'utilisateur de l'appareil doit accepter tout brouillage radioélectrique subi, même si le brouillage est susceptible d'en compromettre le fonctionnement.

Operation in the band 5150-5250 MHz bands are restricted to indoor use to reduce potential for harmful interference to co-channel mobile satellite systems.

CAN ICES-3 (B)/NMB-3(B)

Standards

HIPAA

Kosmos includes security settings that help you to meet the applicable security requirements listed in the HIPAA standard. Users are ultimately responsible for

Standards

ensuring the security and protection of all electronic protected health information collected, stored, reviewed, and transmitted on the system.

The Health Insurance Portability and Accountability Act, Pub.L. No. 104-191 (1996). 45 CFR 160, General Administrative Requirements.

45 CFR 164, Security and Privacy

DICOM

Kosmos conforms to the DICOM standard as specified in the Kosmos DICOM Conformance Statement, available at www.echonous.com. This statement provides information about the purpose, characteristics, configuration, and specifications of the network connections supported by the system.

--End of section--

Intentionally left blank.

System Specifications

Feature	Height (mm)	Width (mm)	Depth (mm)	Weight (g)	Cable (m)	Operating Frequency (MHz)	Scanning Depth (cm)
Kosmos Torso	150*	56	35	290 (with cable)	1.8	1.5 - 4.5	4 - 30
Kosmos Torso-One	150*	56	35	275 (with cable)	1.8	1.5 - 4.5	4 - 30
Kosmos Lexsa	155	56	35	280 (with cable)	1.5	3 - 10.5	1 - 10
Kosmos Bridge	146	216	59	652	N/A	N/A	N/A
Kosmos ECG patient cable	N/A	N/A	N/A	35	0.86	N/A	N/A
Kosmos Power Supply	117.5	53.5	34.2	260	1.5	N/A	N/A

*excluding cable (the hard plastic housing length)




Environmental operating and storage conditions

Kosmos Bridge and probes are intended to be used and stored in normal ambient conditions inside a medical facility.

Operating, charging, transport, and storage condition ranges

	Operating	Transport/Storage
Temperature (°C)	0C to +40C	-20 C to +60 C
Relative humidity (non-condensing)	15% to 95%	15% to 95%
Pressure	62 kPa to 106 kPa	62 kPa to 106 kPa

Mode of operation

	After storage at extreme temperatures, check the Kosmos probe surface temperature before applying to a patient. A cold or hot surface may burn the patient.
	Only operate, charge, and store Kosmos Bridge and probes within the approved environmental parameters.
	When used in high ambient temperatures (such as 40 deg C), the Kosmos safety feature may disable scanning to maintain safe touch temperature.

Kosmos Bridge enforces scanning limits to maintain safe user contact temperatures.

Power supply (charger)

Rated input: 100-240V~, 50-60Hz, 1.5A

Watts: 60

Volts out: 5V, 5.8V, 8.9V, 11.9V, 15V, 20V

Current out (Amps): 4.6A, 4.6A, 4.4A, 4A, 3.6A, 3A

Internal batteries

Kosmos Bridge

Li-Ion main battery: 3.6V, 6.4 Ah

Li-Ion coin cell battery: 3V, 5.8mAh

Battery charging time: The time to charge the battery from 0% to 90% of capacity is ~3 hours

Battery life: A fully charged battery will provide ~90 minutes of uninterrupted scanning

The performance may vary based on scanning modes used.

-- End of Section --

Intentionally left blank.

Wireless networking

Functions

You can connect Kosmos to an IT network to perform the following:

- Storing exam data (static images and clips) acquired by Kosmos in Picture Archiving and Communication System (PACS) by DICOM communication.
- Setting Kosmos time correctly by inquiring the network time service.

Connection specifications

Hardware specification

802.11 a/b/g/n/ac, Bluetooth 4.0 or later.

Software Specification

Kosmos is connected to PACS by the DICOM standard. For details, refer to the DICOM Conformance Statement that is on the EchoNous website.

Network for connecting the device



It is important to configure the device on a secure network, behind a firewall with secure WIFI protocol (e.g. WPA2) to ensure security of the device and patient data transferred over the network.

Specifications for the connection

Hardware specification

802.11 a/b/g/n, Bluetooth 4.0 or later.

Software specifications

Kosmos is connected to PACS by DICOM standard. Refer to the DICOM Conformance Statement of this device for details.

When available, this device connects to the network time server at startup.

Security

This device has no listening ports open to the WLAN interface. A network entity cannot initiate a connection to Kosmos from the WLAN. However, Kosmos can initiate a connection to servers on the WLAN and beyond.

The Kosmos USB port can only be used to export data to a USB memory stick. Computer access to the device through the USB port is blocked.

The following TCP/IP ports are used for outgoing communication to the WLAN:

- Port for DICOM communication (specified by the user in the system settings; typically port 104, 2762, or 11112)
- Port 443 for encrypted traffic to HTTPS time/web servers
- Port 80 for HTTP web servers

Anti-virus software is not installed on this device.

IT network failure recovery measures

Connection to an IT network may become, at times, unreliable, and this may lead to failure to perform the functions described in **Functions**. As a result, the following hazardous situations may occur:

Network failure	Impact on equipment	Hazard	Countermeasures
IT network becomes unstable	Unable to transmit exam data to PACS	Delay of diagnosis	Kosmos has internal memory, and exam data is stored in it. After the IT network has returned to stable, the user can reinitiate the transfer of data.
	Delay of transmission to a PACS		
	Incorrect data transmitted to a PACS	Misdiagnosis	Integrity of the data is ensured by the TCP/IP and DICOM protocols used by Kosmos.
	Unable to get the time from a time server	Incorrect exam data	Kosmos has the capability of entering data and time manually.
	Incorrect time data		Kosmos always indicates the date and the time on the main screen.

Firewall has broken down	Attack via network	Manipulation of exam data	Kosmos closes unnecessary network ports.
	Infection by computer virus	Leak of exam data	Kosmos prevents a user from loading software and executing it.

- Connection of equipment to an IT network that includes other systems could result in previously unidentified risks to patients, operators, or third parties. Before connecting the equipment to an uncontrolled IT Network, make sure that all potential risks resulting from such connections were identified and evaluated, and suitable countermeasures were put in place. IEC 80001-1:2010 provides guidance for addressing these risks.
- When a setting of the IT network to which Kosmos is connected has been changed, check that the change does not affect it, and take measures, if necessary. Changes to the IT network include:
 - Changing the network configuration (IP address, router, and so on)
 - Connecting additional items
 - Disconnecting items
 - Updating equipment
 - Upgrading equipment
- Any changes to the IT network could introduce new risks requiring additional evaluation to be performed.

-- End of Section --

Term	Description
A2C	Apical 2 chamber.
A4C	Apical 4 chamber.
ACEP	American College of Emergency Physicians
Annotation	Annotations are text notes, arrows, and/or measurements that a clinician may add to an image or clip. An annotation appears as an overlay on the image/clip.
Archive	After a report is generated, the patient information is updated in the hospital's EMR/PACS system. The device needs to have a secure connection for data transfer. Once an exam is archived, it cannot be edited. At this point, it is safe to purge the exam from KOSMOS to create more room for new studies.
Arrow	An arrow is an arrow icon that a clinician may put on a certain location of an image/clip to highlight something. This displays as an overlay on the image/clip.
Auscultation	Auscultation is listening to the internal sounds of the body, usually using a stethoscope, for the purpose of examining the circulatory and respiratory systems (heart and breath sounds) as well as the gastrointestinal system (bowel sounds).
BMI	Body mass index.
B-mode	The Kosmos probe array scans a plane through the body and produces a 2D image on the screen. This is also called B-mode imaging.
Calculation	Calculations are estimations made from specific sets of measurements.
Caliper	You perform most measurements by using calipers that you drag into position. The active caliper has a round highlighted handle.

Term	Description
CapSense	<p>The Cypress CapSense technology detects the presence of a finger on or near a touch surface.</p> <p>The Kosmos Bridge handle contains two CapSense buttons and one slider that you can feel and activate without looking.</p>
Cine	<p>A cine is a period of images, stored digitally as a sequence of individual frames. It is recorded at high frame rates and may contain more frames than were displayed during the examination.</p>
Clip	<p>A clip is a short sequences of multiple frames like a movie.</p>
CO	<p>Cardiac output, calculated as: $CO = SV \times HR$.</p>
Completed exam	<p>Once an exam is completed, you won't be able to add images to the exam. You can add/edit/delete any annotations that have been saved as overlays on images/clips until the exam is archived. Once archived, you cannot edit anything. If the clinician does not complete an exam, Kosmos will automatically complete the exam when Kosmos is turned off.</p>
CW	<p>Continuous-Wave Doppler</p>
DA	<p>Digital auscultation.</p>
DICOM	<p>Digital Imaging and Communications in Medicine. DICOM is the most universal and fundamental standard in digital medical imaging. It's an all-encompassing data transfer, storage, and display protocol built and designed to cover all functional aspects of contemporary medicine. PACS functionality is DICOM driven.</p>

Term	Description
ECG	Electrocardiogram. Electrocardiography is the process of recording the electrical activity of the heart over a period of time using electrodes placed over the skin. These electrodes detect the tiny electrical changes on the skin that arise from the heart muscle's electro physiologic pattern of depolarizing and re-polarizing during each heartbeat.
ED	End-diastolic.
EDV	End-diastolic volume.
EF	Ejection fraction, calculated as (a percentage): $EF = (EDV - ESV) / EDV * 100$
ES	End-systolic.
ESV	End-systolic volume.
Exam	An exam contains all the objects, images, clips, and reports that are saved during a clinical examination of a patient with Kosmos, which usually maps to a patient's visit.
FOV	Field of view is the two-dimension space of B-mode image acquisition.
Frozen state	The state Kosmos gets into when you tap the Freeze button in live imaging. During the frozen state, you can add annotations to one frame of the cine and save the still image. The measurements only stay on one frame of the cine, but the annotations will persist in the whole cine. When you save a clip from the cine, annotations are saved as overlays on the clip, but the measurement won't be saved in the clip. That is because usually measurements are relevant to only one frame of a cine instead of the whole series of frames.
HR	Heart rate.
Image	An image is a single frame of an ultrasound view captured by Kosmos.
LV	Left ventricle.

Term	Description
M-line	A line that appears in B-mode for which M-mode provides the trace.
Measurement	A measurement is a distance or area measurement on images with no inference to underlying anatomy. A measurement overlay shows the tool (such as a caliper or ellipse) and the measured values.
MWL	Modality Worklist
PACS	Picture Archiving and Communication Systems. PACS refer to medical systems (hardware and software) built to run digital medical imaging. The main components of PACS include digital image acquisition devices, digital image archives, and workstations. The PACS settings in this document refer to the settings of connecting to digital image archives.
Physical coordinates	The position in the field of view expressed in terms of physical dimensions either in millimeters or radians with respect to a designated point of reference.
Picture	You can use the Kosmos camera to take pictures of a wound or injury as part of the exam.
PIMS	Patient Information Management Systems.
Ping test	A ping test is used to test a TCP/IP connection. If the test is successful, the connection between the Kosmos and PACS archive is working.
PW	Pulsed-Wave Doppler
Report	A report consists of details of an exam, along with the notes entered by the clinician.
Review	This is the state of Kosmos where you can review and edit patient data if it has not been archived.
ROI	Region of Interest. The ROI refers to the bounded region in the field of view where color flow information is depicted.

Term	Description
Scan	A scan is a system preset where system parameters are optimized for scanning a certain organ, such as heart or lungs. Scans can include multiple images, clips, and reports that you can save. The scan preset drives calculations, measurements, and reports.
Snackbar	The snackbar is a brief message that displays on the bottom of many Kosmos screens. You don't have to act on the messages, and they automatically go away after a short period of time.
Study	<p>A study is a collection of one or more series of medical images and presentation states that are logically related for diagnosing a patient. Each study is associated with one patient. A study may include composite Instances that are created by a single modality, multiple modalities, or by multiple devices of the same modality.</p> <p>In Kosmos, the term "exam" means "study" in the DICOM world. An exam contains all the objects, images, clips, and reports that are saved during a clinical examination of a patient with Kosmos, which usually maps to a patient's visit.</p>
SV	<p>Stroke volume, calculated as:</p> $SV = EDV - ESV$
Verify	This is used to conduct a DICOM C-Echo, which sends a signal to the PACS archive using a DICOM protocol to confirm that the PACS archive is working and available on the network.

Intentionally left blank.

Auto EF Clinical Performance and Non-Clinical Testing

Auto EF Clinical Performance Testing

A prospective study was conducted to evaluate the difference between the left ventricle ejection fraction (LVEF; %EF) automatically generated by Kosmos' Auto EF algorithm and manually calculated by cardiologists from clips acquired by cardiac sonographers.

Study Design

153 participants were scanned by the cardiac sonographers and the two views were obtained using a Kosmos ultrasound system with Auto EF software. The A2C and A4C views were used to calculate the %EF, using Simpson's biplane method, at point-of-care. A range of body-mass index (BMI) was included in the dataset, with 22.8% of the patients overweight ($25 \leq \text{BMI} < 30 \text{ kg/m}^2$) and 31.6% obese ($\text{BMI} > 30 \text{ kg/m}^2$). In addition, 19% of patients had reduced EF ($30\% \leq \text{EF} < 53\%$), and 22% of the patients had severely reduced EF ($\text{EF} < 30\%$). All studies were traced by three (3) sonographers using the biplane Simpson's method to establish the reference standard.

The study continued enrollment until four sonographers had completed scans of 17 patients each. Enrolled patients were evenly stratified into four groups based on BMI to ensure a sufficient distribution of patients by sex and BMI. Following the patient exams, three (3) independent readers manually calculated %EF from scans containing both A4C and A2C views ($N = 141$) which acted as the ground truth.

The performance goal is the Root Mean Square Deviation (RMSD) < 10 %EF between LVEF manually measured by experts and Kosmos' Auto EF on biplane scans (A4C and A2C) acquired by cardiac sonographers.

Results

Results of the hypothesis testing of Kosmos Auto EF software evaluating the performance of the algorithm against its objective performance goal of < 10 % RMSD is shown below. The performance goal was met.

Endpoint	RMSD (95% CI)	p-value	Pearson Correlation	Bias (95% CI)
LVEF calculations	4.57 %EF (5.129, 5.153)	< 0.0001	0.96	1.54 %EF (6.90, -9.98)

Software Verification and Validation Testing

Software documentation generated as part of EchoNous' design process include:

1. Software/Firmware Description
2. Device Hazard Analysis
3. Software Requirement Specifications
4. Architecture Design Chart
5. Software Design Specifications
6. Traceability
7. Software Development Environment Description
8. Verification and Validation Documentation
9. Revision Level History
10. Unresolved Anomalies
11. Cybersecurity

A comprehensive risk analysis was generated for the software with detailed description of the hazards, their causes and severity, as well as acceptable methods for control of the identified hazards. EchoNous developed a description, with test protocols including pass/fail criteria and report of results, of acceptable verification and validation activities at the unit, integration, and system level.

Algorithm Testing

Comprehensive non-clinical performance testing of the deep-learning algorithms used in the device was performed to support their clinical performance. Specifically, the performance testing evaluated the performance of the following software functionality:

- EF Workflow

Overall, the non-clinical performance testing results provide evidence in support of the functionality of Auto EF's algorithm.

- End of Section -

Intentionally left blank.

AI FAST Clinical Performance and Non-Clinical Testing

AI FAST Clinical Performance Testing

A prospective study was conducted to evaluate the sensitivity and precision of AI FAST's view identification and object labeling algorithms.

Study Design

32 subjects were recruited across two sites (16 subjects per site). Each subject was scanned by a cardiac sonographer and a sonographer with abdominal experience using the following views: SUP (suprapubic), RUQ (right upper quadrant), LUQ (left upper quadrant), AS ([transverse] aortic sweep), IVC ([longitudinal] inferior vena cava), Aorta, SUB (subcostal 4-chamber), SUB2 (subcostal 2-chamber), A4C (apical 4-chamber), A2C (apical 2-chamber), PLAX (parasternal long axis), PSAX (parasternal short axis), and Lung.

The participants in this study comprised of a well-rounded representation of demographic factors such as age, gender, BMI, ethnicity, and race. Enrolled patients were evenly stratified into four groups based on BMI to ensure a sufficient distribution of patients by sex and BMI

Five (5) radiologists, from a pool of nine (9), independently and collectively reviewed the algorithm's predictions on anatomical structure labeling and view identification using extracted frames. These radiologists were used to establish ground truth to compare the algorithm's performance and were blinded to assessments from others on the panel. The results from the expert-panel reads were used for the statistical analysis.

Two (2) prospectively defined primary endpoints were evaluated to demonstrate the efficacy of the anatomical object labeling and view identification of AI FAST's algorithms in clinical settings.

Results

Results of the hypothesis testing of Kosmos AI FAST software evaluating the performance of the algorithm against its objective performance goal of < 20% false detection rate (FDR) is shown below. The performance goal was met.

Endpoint	FDR (95% CI)	Recall	Precision
Object Labeling	0.9% (0.0 – 1.5%)	93.8%	99.6%
View Identification	3.4% (0.0 – 4.3%)	89.9%	96.6%

Note, recall is defined as how well the algorithm can identify positive instances, also known as sensitivity. Precision is how well the algorithm can correctly predict positive instance.

Software Verification and Validation Testing

Software documentation generated as part of EchoNous' design process include:

1. Software/Firmware Description
2. Device Hazard Analysis
3. Software Requirement Specifications
4. Architecture Design Chart
5. Software Design Specifications
6. Traceability
7. Software Development Environment Description
8. Verification and Validation Documentation
9. Revision Level History
10. Unresolved Anomalies
11. Cybersecurity

A comprehensive risk analysis was generated for the software with detailed description of the hazards, their causes and severity, as well as acceptable methods for control of the identified hazards. EchoNous developed a description, with test protocols including pass/fail criteria and report of results, of acceptable verification and validation activities at the unit, integration, and system level.

Algorithm Testing

Comprehensive non-clinical performance testing of the deep-learning algorithms used in the device was performed to support their clinical performance. Specifically, the performance testing evaluated the performance of the following software functionality:

- Abdominal Object Detection and View Identification

Overall, the non-clinical performance testing results provide evidence in support of the functionality of AI FAST's algorithms.

- End of Section -

Intentionally left blank.