
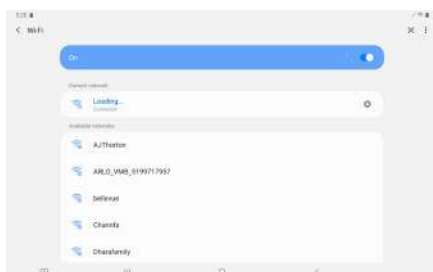
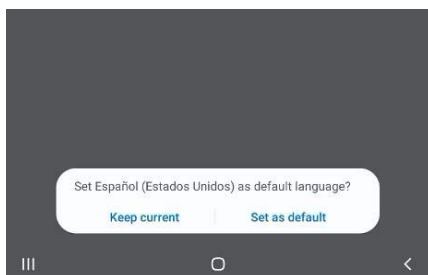


LANGUAGE AND WI-FI SELECTION

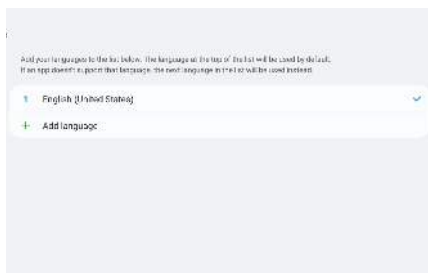

1 CHANGING DEFAULT LANGUAGE

The default language on the EchoNous System software is English. To change the default language:

- From the EchoNous System homepage, tap the Three-dot Menu > Settings > System Settings > General Management > Language and Input > Language
- Select Add language and choose the language you would like
- Select Set as default

2 CONNECTING TO A WIRELESS NETWORK

To connect to a wireless network for completing registration of the product:

- From the EchoNous System homepage, tap the Three-dot Menu > Settings > System Settings > Connections
- Press Wi-Fi and click on the slider to turn on the Wi-Fi. Select a network from the list and enter details as needed to securely establish a network connection.
- Note: Remember to switch off Wi-Fi when not in use.