

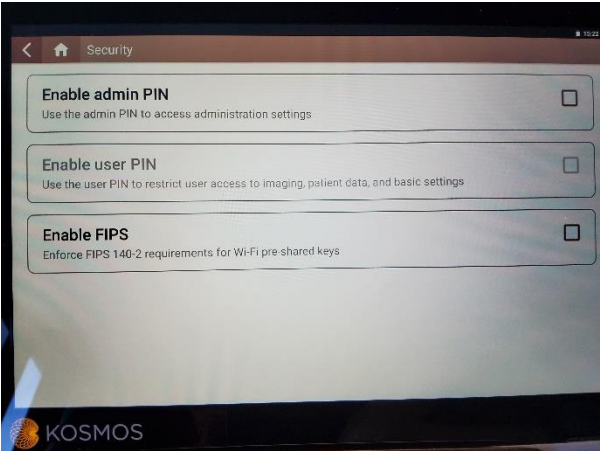
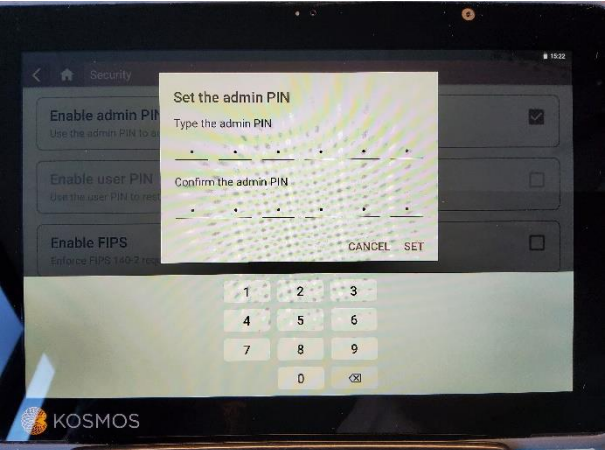
Summary

This technical bulletin provides instructions for enabling AI FAST workflows on your Kosmos Torso and Torso-One probes via USB flash drive. ***This procedure will need to be performed on every Kosmos bridge tablet where you plan to use the AI FAST enabled probe(s).***


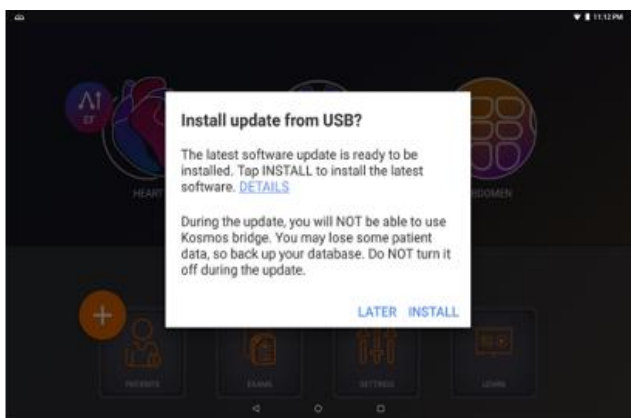
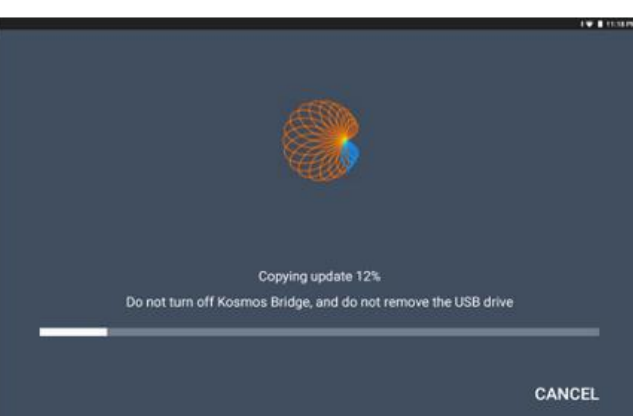
Contents

One (1) USB Drive containing 7.1.0.x software and Kosmos AI FAST probe license(s).

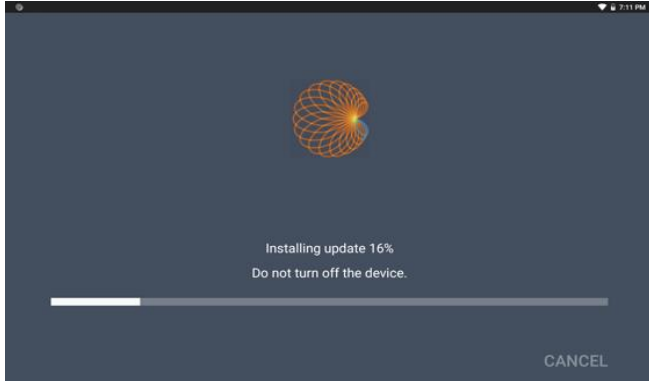
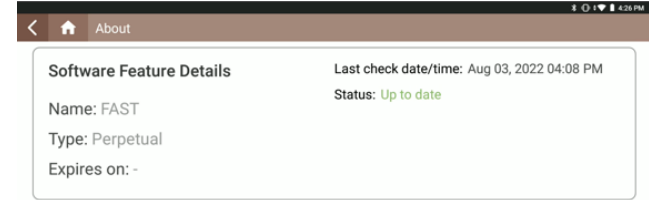
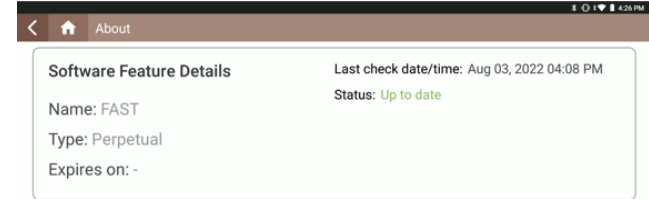
Software Update and License Installation

<p>STEP 1</p>	<p><i>NOTE: Follow Step 1 only if the device contains software version 7.0.0.125. For software versions 4.x and 6.x, begin at Step 2. The software version can be viewed at the following location: Settings > About</i></p> <p>Enable the Admin PIN by selecting the check box at the following location: Settings > Admin > Security > Enable admin PIN</p> <p>Enter 000000 as the admin PIN and touch SET.</p>	 
----------------------	--	--

Technical Bulletin

<p>STEP 2</p>	<p>Plug the USB flash drive into the USB-C port on the right side of the Kosmos Bridge.</p> <p>The system will recognize the USB drive and begin to verify its contents. This step can take 2 to 3 minutes to complete.</p>	
<p>STEP 3</p>	<p>Once the contents of the USB drive have been verified, you will be prompted to install the software.</p> <p>Press INSTALL.</p>	
<p>STEP 4</p>	<p>Kosmos Bridge will begin copying the update package from the USB drive. The progress of the install is indicated by the progress bar on the screen.</p>	

Technical Bulletin

<p>STEP 5</p>	<p>Once the update package has been copied, the installation process will immediately start.</p> <p>After installation is complete, the Kosmos Bridge will reboot, and a message will appear stating the update is complete.</p> <p>It is important to leave the USB drive plugged into the bridge until the software has installed and the device has fully rebooted.</p>	
<p>STEP 6</p>	<p>Verify the FAST software feature is enabled in: Settings > About > Software Feature Details</p> <p>The name of the feature will be shown (FAST), as well as the license type and expiration.</p>	
<p>STEP 7</p>	<p>Disable the Admin PIN feature by deselecting the box at the following location. Settings > Admin > Security > Enable admin PIN</p>	

If you have any questions regarding the software installation or feature implementation process, contact EchoNous Customer Service by telephone at **(844) 854-0800** or via email at **service@echonous.com**.