

## Kosmos on iOS Software v1.1

Date: February 24th, 2023

Distribution: Internal (sales and clinical educators) and External (customers)

## Kosmos on iOS Software v1.1 Release

The iOS v1.1 software release provides the following features and improvements for Kosmos on iOS:

- Initial release of Kosmos on iOS with Torso-One and Lexsa functionality
  - B-mode
  - M-mode
  - Color Doppler

## Compatible Tablets

Please visit <https://echonous.com/product/kosmos-ultraportable-ultrasound/> for a current list of tablets that are compatible with Kosmos.

## System Software Availability

The v1.1 system software is available for customers in the following countries:

- USA
- Nepal
- Oman
- Philippines
- Trinidad & Tobago
- UAE
- Egypt
- Jordan
- Israel
- Iraq
- South Africa
- Lebanon
- Hong Kong

## Driverkit Installation Instructions

**Step 1:** Go to your iPad Settings and update to iPadOS 16.

**Settings → General → Software Updates**

**Step 2:** Download “EchoNous Kosmos: Ultrasound App” from the Apple App Store.

**Step 3:** Once downloaded, open the Kosmos App and tap “Click here to enable driver”. You will be routed to your iPad’s settings. Swipe to enable the EchoNous Probe Driver.

**Step 4:** Go back to the Kosmos App and connect the Kosmos probe.

Visit the Apple App Store to download and install the latest update for the EchoNous Kosmos: Ultrasound App. If you have any questions, contact EchoNous Customer Service by telephone at (844) 854-0800 or via email at [service@echonous.com](mailto:service@echonous.com).