



Quick Guide

Kosmos Lexsa


For Bridge, Android and iOS

Kosmos on Android

Features vary between iOS and Android Software. Contact your EchoNous representative for more information regarding your software.

Homescreen

- 1 Imaging (MSK, Nerve, Vascular, and Lungs)
- 2 Patients, Exams, Settings and Tutorial Videos

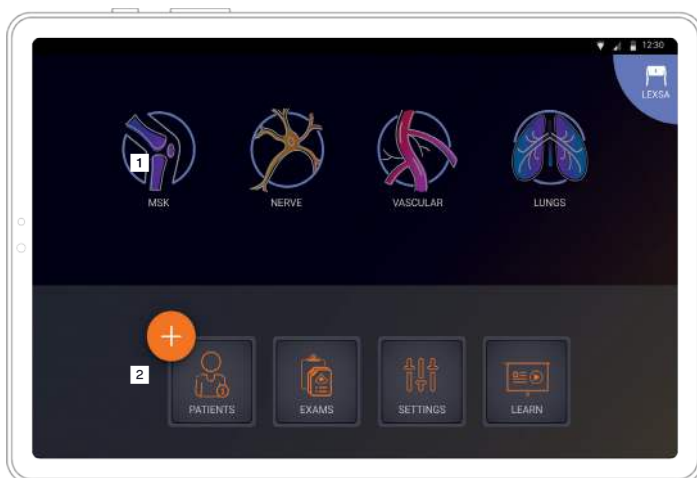

Note: EchoNous Kosmos Ultrasound App required for Android users.




App: EchoNous Kosmos: Ultrasound App
EchoNous, Inc.



Download EchoNous Kosmos: Ultrasound App from the Google Play Store.


Launch the App, connect a compatible probe and you are ready to scan.

Kosmos on Android: Imaging

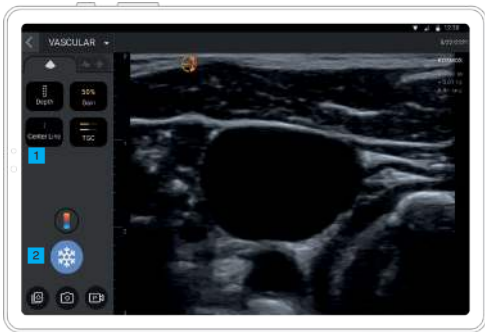

B-Mode

- 1 Center line
- 2 Freeze

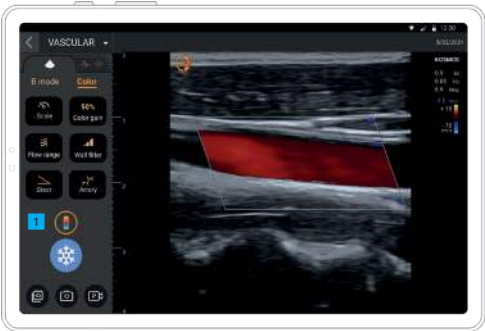

Color Doppler


- 1 Color mode is available in MSK, Nerve and Vascular presets.

Kosmos on iOS

Features vary between iOS and Android Software. Contact your EchoNouS representative for more information regarding your software.

Homescreen

- 1 Imaging (MSK, Nerve, Vascular, and Lungs)
- 2 Patients, Exams, Settings and Tutorial Videos

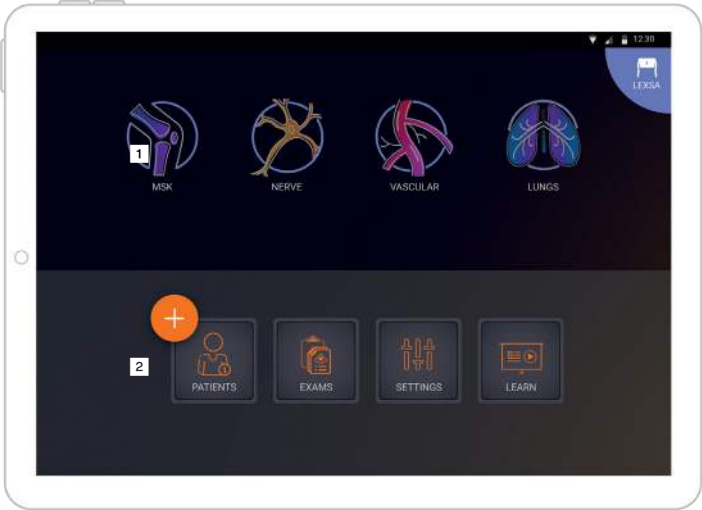

Note: Echonous Kosmos Ultrasound App required for iOS users.




Download EchoNouS Kosmos: Ultrasound App from the Apple App Store.
Launch the App, connect a compatible probe and you are ready to scan.

Kosmos on iOS: Imaging



B-Mode

- 1 Center line
- 2 Freeze

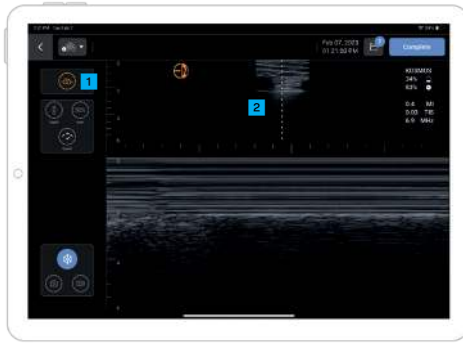


M-Mode

- 1 M-Mode is only available in the Lungs preset
- 2 M-lin



Color Doppler

- 1 Color mode is available in MSK, Nerve and Vascular presets.

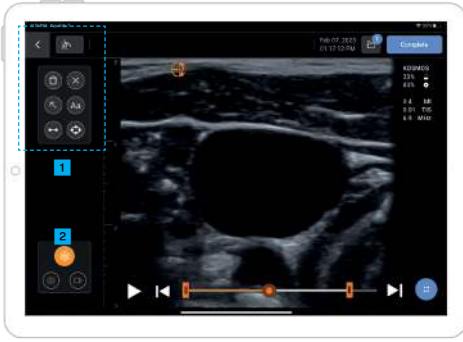

Freeze

- 1 Annotations
- 2 Unfreeze

Contact

Customer Support

Call 1-844-854-0800
Email support@echonous.com

KOSMOS is intended to be used by qualified and trained healthcare professionals in the clinical assessment of the cardiac and pulmonary systems and the abdomen by acquiring, processing, displaying, measuring, and storing ultrasound images

Refer to user guide before use

⚠ Caution: Federal (United States) law restricts this device to sale by or on the order of a physician.

Note: Not all features are available in all markets. Reference the user guide of your region or country for all available features before use.

Features may vary based on devices.