

Kosmos Chemical Compatibility

The cleaning and disinfection chemicals listed here have been evaluated for compatibility with the device materials. We will continue to update the table below as we evaluate additional cleaning and disinfection chemicals for compatibility with the device.

Cleaning and/or Disinfectant Agent	Active Ingredient	Kosmos Bridge	Kosmos Probes	ECG Cables
Warm Soapy water	None	A	A	A
PDI Healthcare Sani-Cloth® Plus Germicidal Disposable Wipes	n-Alkyl dimethyl ethylbenzyl ammonium chlorides (0.125%), n-Alkyl dimethyl Benzyl Ammonium Chlorides (0.125%), Isopropanol (10-20%), 2-Butoxyethanol (<5%)	A	A	A
1PDI Healthcare Super Sani-Cloth® Germicidal Disposable Wipes	n-Alkyl dimethyl ethylbenzyl ammonium chlorides (0.25%), n-Alkyl dimethyl Benzyl Ammonium Chlorides (0.25%), Isopropyl Alcohol (Isopropanol) (55.5%)	A	A	A
1PDI Healthcare Sani-Colth® Bleach Germicidal Disposable Wipes	Sodium Hypochlorite(0.63%)	A	A	C
2Metrex CaviWipes™ (KavoWipes)	Diisobutylphenoxyethoxyethyl dimethyl benzyl ammonium chloride (0.28%), Isopropanol (17.2%)	A	A	A
1Metrex CaviCide™	Diisobutylphenoxyethoxyethyl dimethyl benzyl ammonium chloride (0.28%), Isopropanol (17.2%)	A	A	A
1Metrex CaviWipes1™	Didecyl dimethyl ammonium Chloride (0.76%), Ethanol(7.50%), Isopropanol (15%)	A	A	A
1Metrex CaviCide1™	Didecyl dimethyl ammonium Chloride (0.76%), Ethanol(7.50%), Isopropanol (15%)	A	A	A
ASP Cidex® OPA	ortho-Phthalaldehyde(0.55%), Dipotassium Hydrogen phosphate, Potassium Dihydrogen Phosphate, Benzotriazole, citric acid, D&C green dye #5, N-(Hydroxyethyl)-ethylenediaminetriacetic acid(HEDTA) (99.45%)	N/A	A	N/A
ASP Enzol® Enzymatic Detergent	Subtilisins (<5%)	A	A	A
Hydrogen Peroxide 3% solution	Hydrogen Peroxide (3%) , water (97%)	A	A	A
Kyorin Pharmaceutical Milton	Sodium Hypochlorite(1.1%)	A	A	C
Tristel Duo ULT™	Sodium Chlorite (0.5%), Citric Acid (5.26%), Chlorine Dioxide (0.032%)	N/A	A	N/A

Key: **1**Product on List N: Disinfectants for Use Against SARS-CoV-2 **2**Per Metrex released note, approved for Covid 19

A: Compatible

B: Caution - Not recommended.
May cause minor cosmetic changes with extended use.

C: Not compatible - Do not use. Extended exposure may lead to product damage.

N/A: Compatibility data not available

Note: Not all features are available in all markets. Reference the user guide of your region or country for all available features. **⚠ Caution:** Federal (United States) law restricts this device to sale by or on the order of a physician.

Electronic versions of the Kosmos User Guides may be viewed and downloaded at echonous.com/resources.

Printed versions of the Kosmos User Guides may be requested by emailing service@echonous.com or by calling 1-844-854-0800.

For questions, please contact EchoNous® Customer Support:
Call: 1-844-854-0800
Fax: 425-242-5553
Email: support@echonous.com
Web: echonous.com

Australia Sponsor: LC & Partners Pty Ltd, North Sydney, NSW, 2060
Tel: +61 2 9959 2400